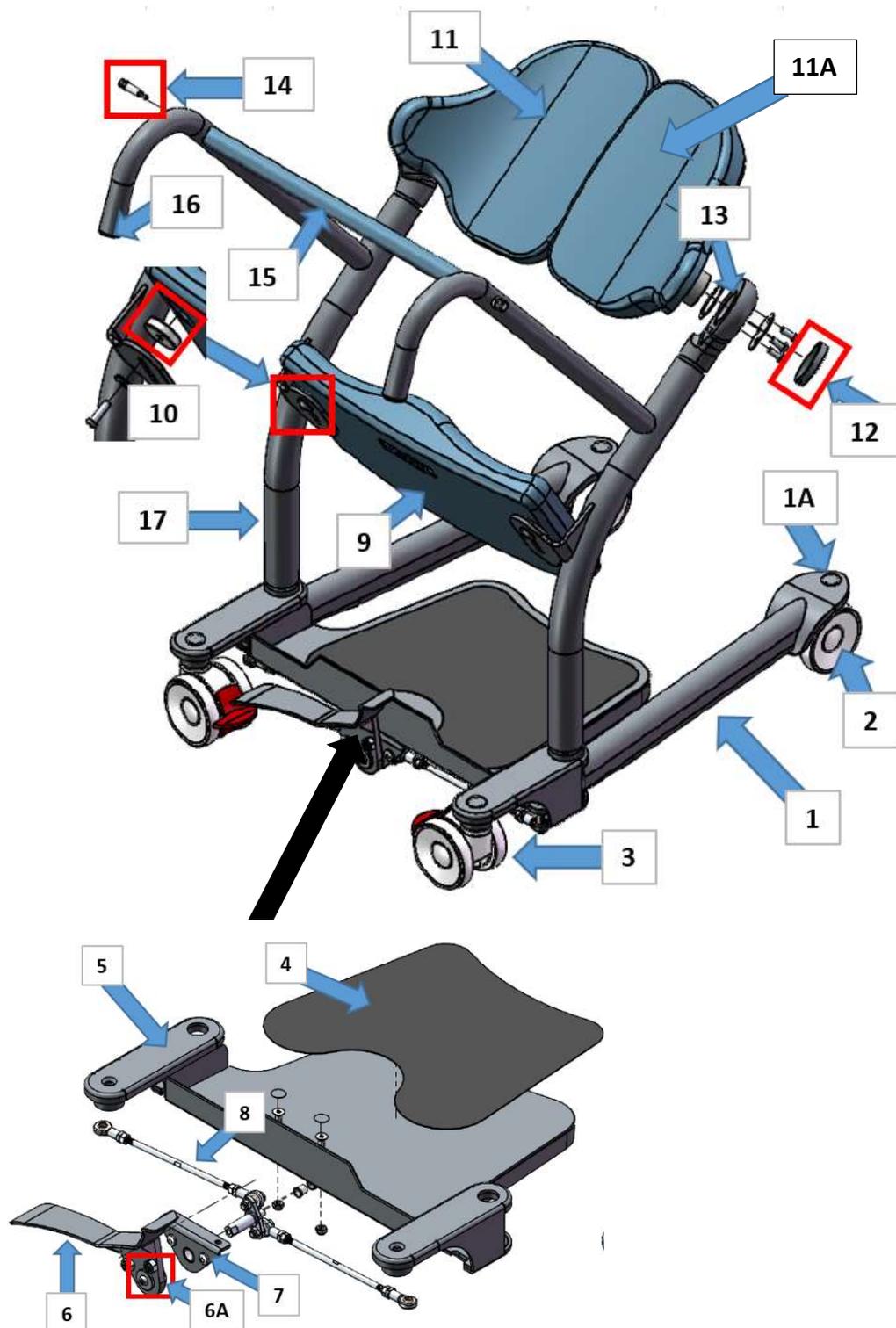


# Aspire STAR Patient Mover - Spare Parts Manual

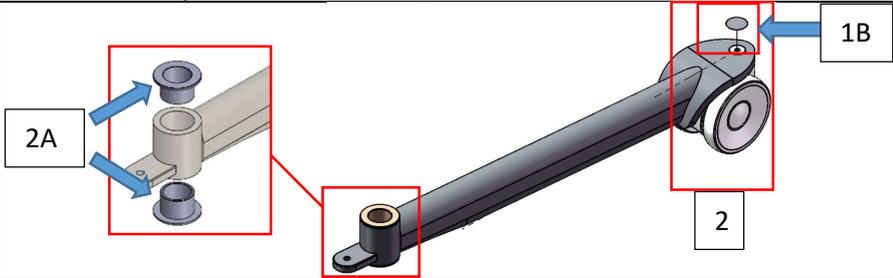
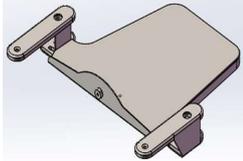


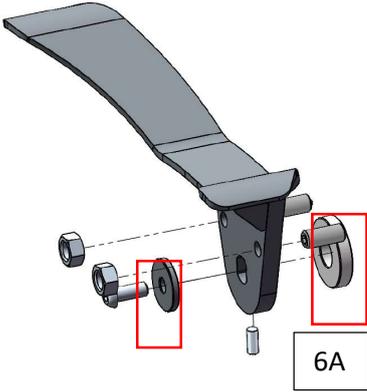
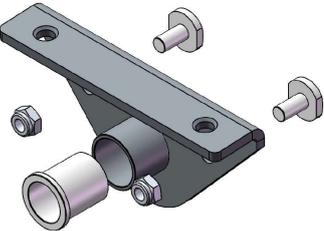
Exploded Diagram

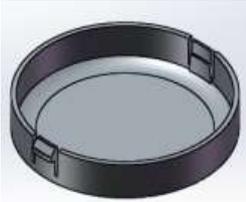


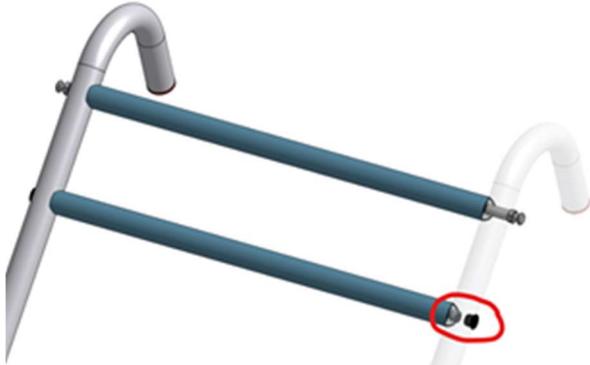
Product Code	Product Description
LSS390580	Standing & Transport Aid - Aspire STAR Mover - Adjustable Legs - 185kg

## Spare Parts List

Ref #	Item Number	Item Description	Type
1	GSP393010	Replacement Leg Frame	Spare Part
1A	GSP393007	Adhesive Sticker – 20mm for Footplate	Spare Part
1B	GSP393008	Adhesive Sticker – 30mm for Castor Bolt	
2	GSP393002	Castor and Mounting Hardware – Swivel – Front	Spare Part
2A	GSP393012	Bronze Bush - for Leg Frame	Spare Part
			
3a	GSP393000	Castor and Mounting Hardware – Locking – Rear <b>Compatible with STAR movers prior to SN 2305000</b>	Spare Part
3b	GSP393001	Castor and Mounting Hardware – Total Locking – Rear <b>Compatible with STAR movers from PO777658, SN 2208001 (remove existing spacer on frame)</b> <b>STAR movers from PO910505, SN 2305001 already have TOTAL LOCKING castors installed and do not have a spacer</b>	Spare Part
			
4	GSP393015	Anti-Skid Mat for Footplate	Spare Part
			
5	GSP393020	Footplate with Castor Bracket Assembly * PRIOR 2112288	Spare Part
	GSP393022	Footplate with Castor Bracket Assembly * FROM 2112288	Spare Part
			
6	GSP393025	Foot Pedal Assembly	Spare Part
6A	GSP393030	Bush Caps – for Foot Pedal	Spare Part

			
7	GSP393035	Mounting and Spacer Kit – for Foot Pedal	Spare Part
			
8	GSP393040	Tie Rods and Linkage Assembly	Spare Part
8a	GSP393042	Leg connecting rod holding assembly (Bolt+Spacer+Nut)	Spare Part
			
9	GSP393045	Knee Pad	Spare Part
			
10	GSP393050	Spacer and Hardware Kit – for Knee Pad	Spare Part
	GSP393051	Rectification Spacer Kit *PRIOR 21090000	Spare Part
			
11	GSP393055	Swivel Seat Pad and Metal Mounting Hardware – Left	Spare Part

11A	GSP393060	Swivel Seat Pad and Metal Mounting Hardware – Right	Spare Part
			
12	GSP393065	Plastic End Cap – for Swivel Seat Pad	Spare Part
			
13	GSP393070	Bush Swivel Plastic Cap – for Swivel Seat Pad	Spare Part
	GSP393072	Screws - for Swivel Seat Pad - PKT/3	
			
14	GSP393075	Mounting Sling Pin – for Grab Handle Bar	Spare Part
			
15	GSP393080	Grab Handle Bar	Spare Part
			
16	GSP393085	End Caps – for Push Handles	Spare Part
			

17	GSP393090	Handle Frame – Left	Spare Part
	GSP393095	Handle Frame – Right	Spare Part
	GSP393096	Handle Frame - Left - (After SRN: 202311001)	Spare Part
	GSP393097	Handle Frame - Right - (After SRN: 202311001)	
		**Replace both Left and Right sides if replacing on Star Mover prior SRN: 202311001	
			
18	GSP393076	Hex Head Screw - M8 X 25mm - for second handle hole on upright (After SRN: 202311001)	Spare Part
18a	GSP393077	Silicone Plug - 13mm - for second handle hole on upright (After SRN: 202311001)	Spare Part
			
19	GSP393023	Complete Base Assembly - Incl Legs - w/o Castors	Spare Part
Use this Item for replacing faulty Base			
