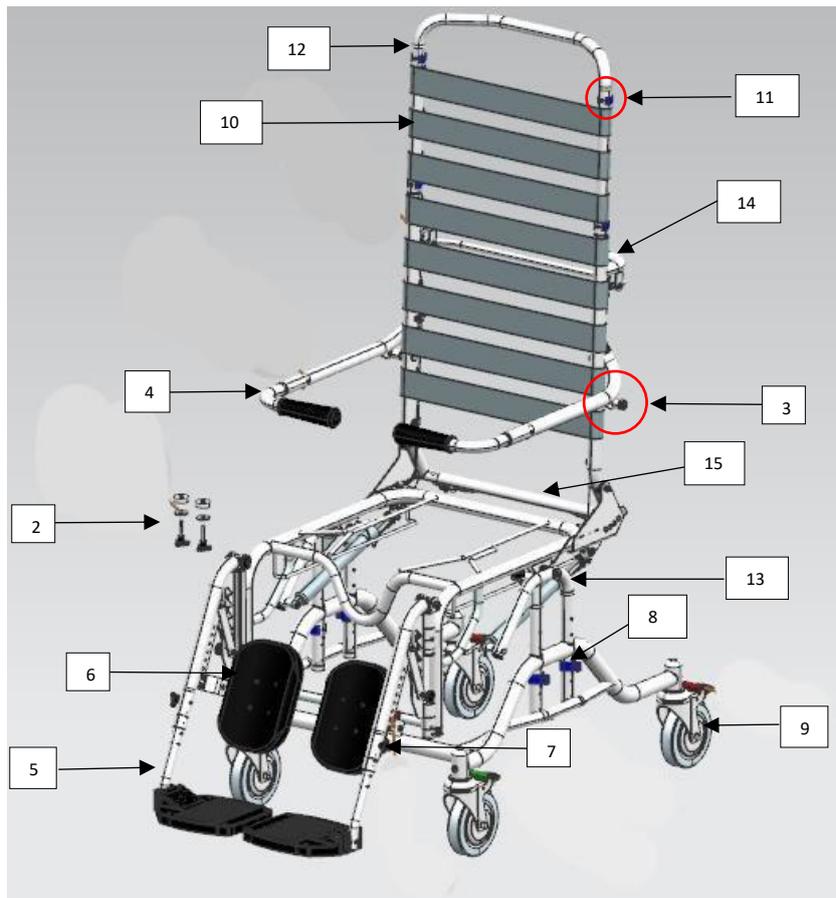


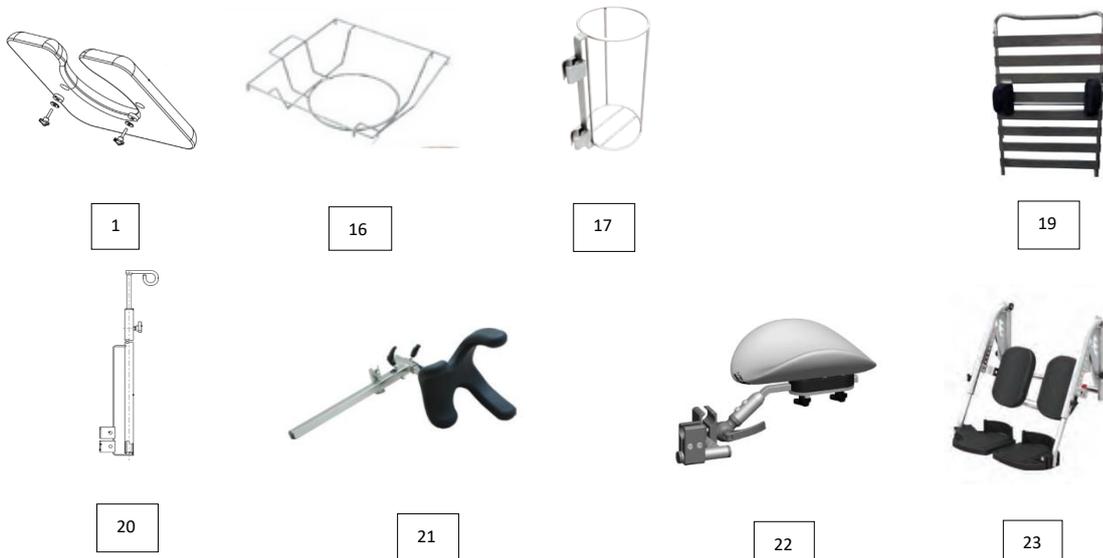
**Aspire Tilt in Space Shower Commode w/SA Footrest
Spare Parts Manual**



Aspire Tilt in Space 460mm / 530mm SA Commode – Exploded Parts Diagram

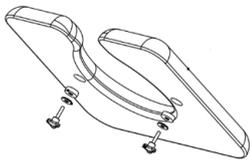
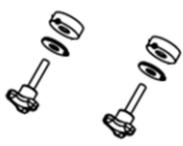
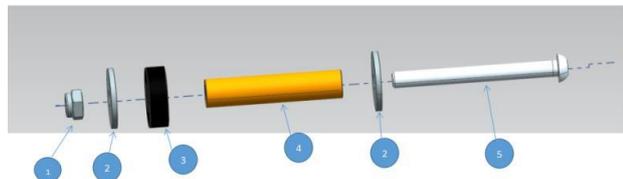


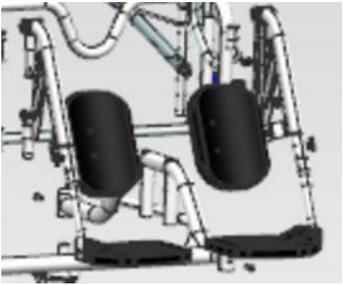
Accessories

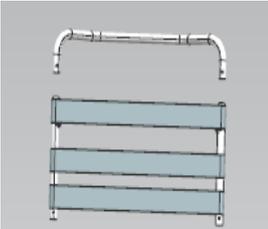


Product Code	Product Description
BTC066030	Tilt in Space Commode – Attendant Propelled – 460mm
BTC066040	Tilt in Space Commode – Attendant Propelled – 530mm

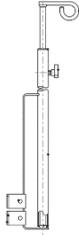
Spare Parts List

Ref #	Item Number	Item Description	Type
1	BTC066600	Shower Commode Seat - Open 460mm	Accessory
1	BTC066700	Shower Commode Seat – Closed 460mm	Accessory
1	BTC066750	Shower Commode Seat – Open 530mm	Accessory
			
2	GSP342405	Tri Nut Assembly – M6 - Used to fix seat to frame – Small Packet	Spare Part
			
3	GSP342404	Armrest Mounting Kit – for Tilt in Space Shower Commode	Spare Part
			
3b	GSP342443	Hardware Bolts and Spacers – OLD – With Side Frame Rectification Kit*	Spare Part
	GSP342457	Hardware Bolts and Spacers – NEW**	Spare Part
<p>* Side Frame Rectification Kit – GSP342477 (DISCONTINUED) ** Post serial number 362102001 for TIS Shower Commode 460mm Post serial number 372103026 for TIS Shower Commode 530mm</p>			
			
3a	GSP342463M	Armrest Tube Replacement – LEFT – TIS – Matte Finish	Spare Part
	GSP342464M	Armrest Tube Replacement – RIGHT – TIS – Matte Finis	Spare Part
3a	GSP342463S	Armrest Tube Replacement – LEFT – TIS – Shiny Finish	Spare Part
	GSP342464S	Armrest Tube Replacement – RIGHT – TIS – Shiny Finish	Spare Part
			

4	GSP342409M	Safety Swing Armrest (only) – Left – Matte Finish	Spare Part
4	GSP342411M	Safety Swing Armrest (only) – Right – Matte Finish	Spare Part
4	GSP342409S	Safety Swing Armrest (only) – Left – Shiny Finish	Spare Part
4	GSP342411S	Safety Swing Armrest (only) – Right – Shiny Finish	Spare Part
			
5	GSP342419M	Leg Rest with Calf Pad – Left – Matte Finish	Spare Part
5	GSP342420M	Leg Rest with Calf Pad – Right – Matte Finish	Spare Part
5	GSP342419S	Leg Rest with Calf Pad – Left – Shiny Finish	Spare Part
5	GSP342420S	Leg Rest with Calf Pad – Right – Shiny Finish	Spare Part
			
6	GSP342440	Calf Pad only – without Leg Rest	Spare Part
			
7	GSP342442	Hand Wheel – Swing Away Leg Rest	Spare Part
			
8	GSP342441	C-Clip Pin – Height Adjustment	Spare Part
			

9	GSP342388	Braking Castor – for Tilt in Space Shower Commode * Discontinue	Spare Part
9	GSP342389	Directional Castor – for Tilt in Space Shower Commode	Spare Part
9	GSP342388T	Braking Castor w Solid Pintle - TENTE – for TIS Shower Commodes	Spare Part
			
10	GSP342445	Backrest Strap 1pce with Rivets – 460mm	Spare Part
10	GSP342446	Backrest Strap 1pce with Rivets – 530mm	Spare Part
			
11	GSP342444	C- Clip Pin – Backrest Connections	Spare Part
			
12	GSP342452M	Backrest Top Section complete inc straps – 460mm – Matte Finish	Spare Part
12	GSP342451M	Backrest Top Section complete inc straps – 530mm – Matte Finish	Spare Part
12	GSP342452S	Backrest Top Section complete inc straps – 460mm – Shiny Finish	Spare Part
12	GSP342451S	Backrest Top Section complete inc straps – 530mm – Shiny Finish	Spare Part
			
13	GSP342393	Cable for Tilt Gas Strut	Spare Part
13	GSP342471	Tilt Gas Strut & Mounting Hardware - 200N	Spare Part
			
14	GSP342475	Tilt Handle & Hardware – 460mm	Spare Part
14	GSP342474	Tilt Handle & Hardware – 530mm	Spare Part

			
15	GSP342473	Recline Pull Cable (Green) & Hardware – 460mm	Spare Part
15	GSP342472	Recline Pull Cable (Green) & Hardware – 530mm	Spare Part
			
16	BTC066400	Pan/Bowl Carrier – 460mm	Accessory
			
17	BTC067200	Oxygen Bottle Carrier	Accessory
			
19	WCA867010	Lateral Support	Accessory
			
20	BTC067103	Adjustable IV Pole	Accessory

			
21	BTC067050	Advanced Headrest	Accessory
			
22	BTC067105	Stump Support & Hardware	Accessory
			
23	BTC067110M	Elevating Leg Rest – Left – Tilt ONLY – Matte Finish	Accessory
23	BTC067115M	Elevating Leg Rest – Right – Tilt ONLY – Matte Finish	Accessory
23	BTC067110S	Elevating Leg Rest – Left – Tilt ONLY – Shiny Finish	Accessory
23	BTC067115S	Elevating Leg Rest – Right – Tilt ONLY – Shiny Finish	Accessory
			