

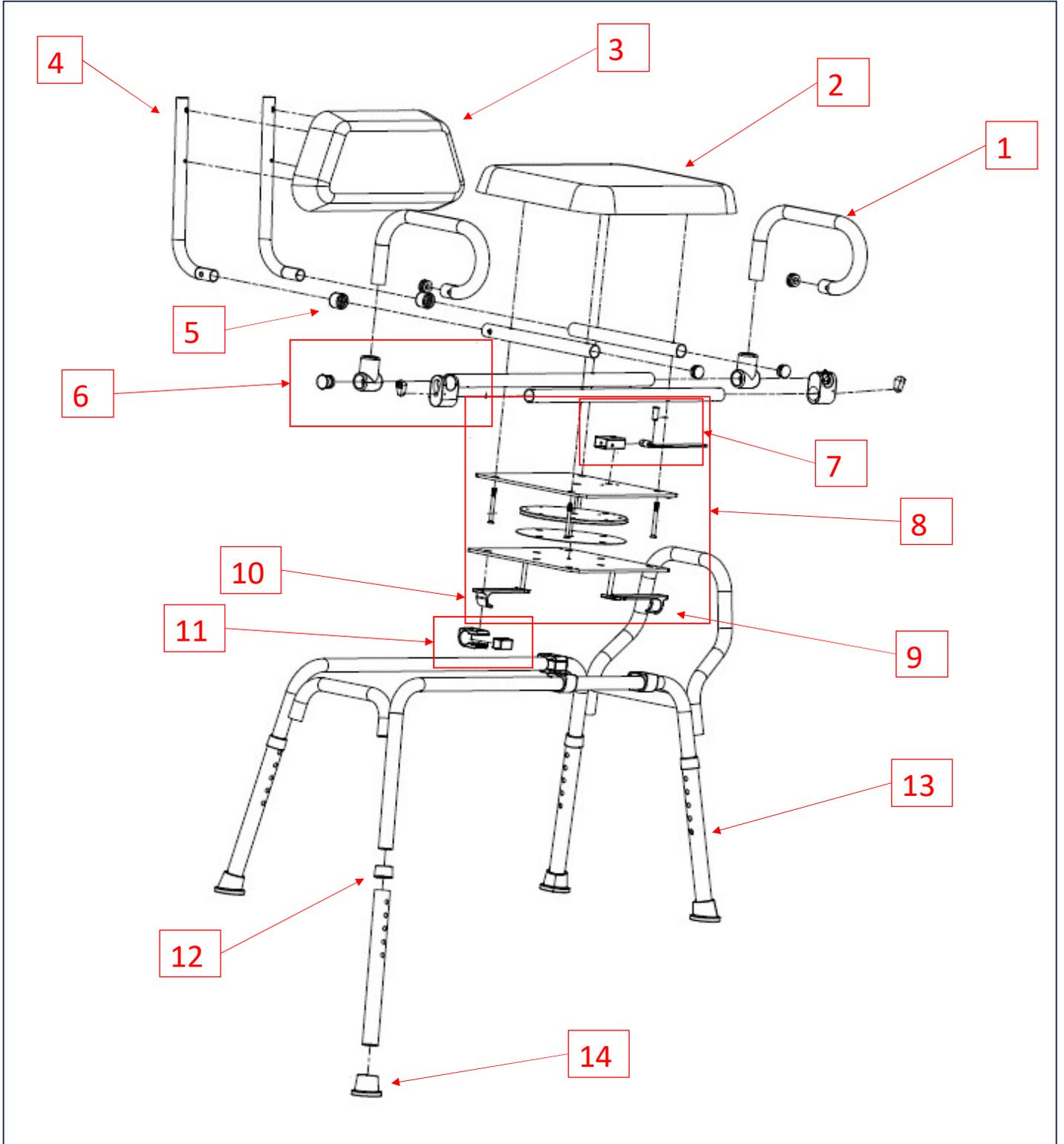
Spare Parts Manual

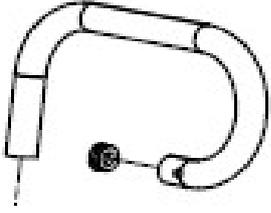
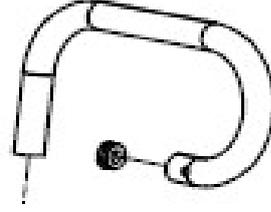
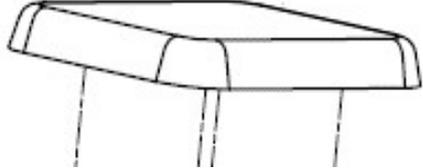
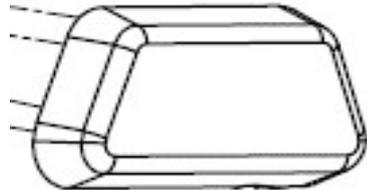
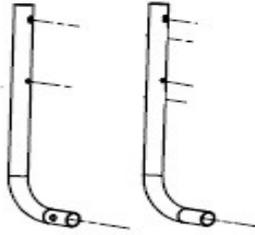
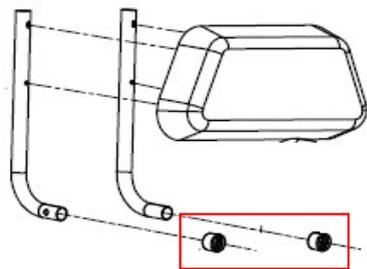


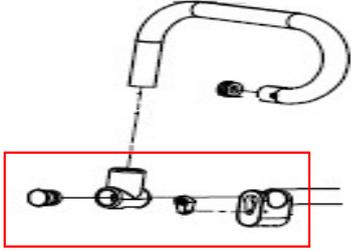
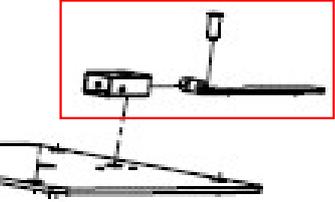
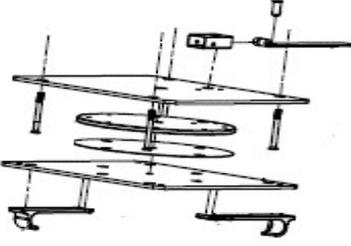
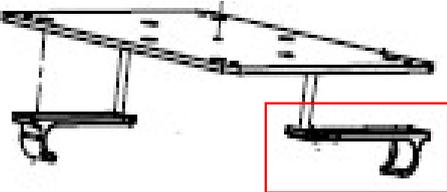
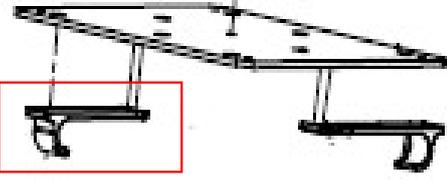
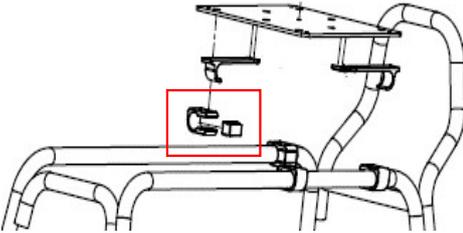
Item Number	Item Description
BTS150310	Transfer Bench - Aspire - Aluminium w/Rotating Seat - 136kg

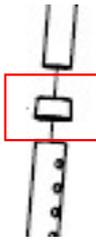
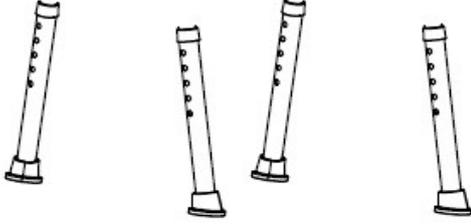
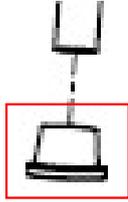
This manual is for internal use only

Exploded Diagram



1 Spare Parts				
Ref	Item Number	Item Description	Image	Type
1	GSP047000	Armrest Tube - Padded - Left - Aspire Transfer Bench w/Rotating Seat		Spare Part
1	GSP047002	Armrest Tube - Padded - Right - Aspire Transfer Bench w/Rotating Seat		Spare Part
2	GSP047004	Seat - w/Mounting Hardware - Aspire Transfer Bench w/Rotating Seat		Spare Part
3	GSP047006	Backrest - w/Mounting Hardware - Aspire Transfer Bench w/Rotating Seat		Spare Part
4	GSP047008	Backrest Tube - Pair - Aspire Transfer Bench w/Rotating Seat		Spare Part
5	GSP047010	Silencer Bushings - for Backrest Tubes - Pair - Aspire Transfer Bench w/Rotating Seat		Spare Part

1 Spare Parts				
6	GSP047012	Armrest Swivel Coupling Assembly - suits Left or Right - Aspire Transfer Bench w/Rotating Seat		Spare Part
7	GSP047016	Lever - Swivel Seat - Complete - w/Mounting Hardware - Aspire Transfer Bench w/Rotating Seat		Spare Part
8	GSP047018	Swivel Seat Mechanism - Complete Assembly - Aspire Transfer Bench w/Rotating Seat		Spare Part
9	GSP047020	Fixed Hook - Long - w/Mounting Hardware - Aspire Transfer Bench w/Rotating Seat		Spare Part
10	GSP047022	Fixed Hook - Short - w/Mounting Hardware - Aspire Transfer Bench w/Rotating Seat		Spare Part
11	GSP047024	Sliding Block Assembly - w/Mounting Hardware - Aspire Transfer Bench w/Rotating Seat		Spare Part

1 Spare Parts				
12	GSP047026	Silencer Bushing - for Leg - Aspire Transfer Bench w/Rotating Seat		Spare Part
13	GSP047028	Legs - w/Rubber Feet - Set of 4 - Aspire Transfer Bench w/Rotating Seat		Spare Part
14	GSP047030	Rubber Foot / Ferrule - Aspire Transfer Bench w/Rotating Seat		Spare Part