

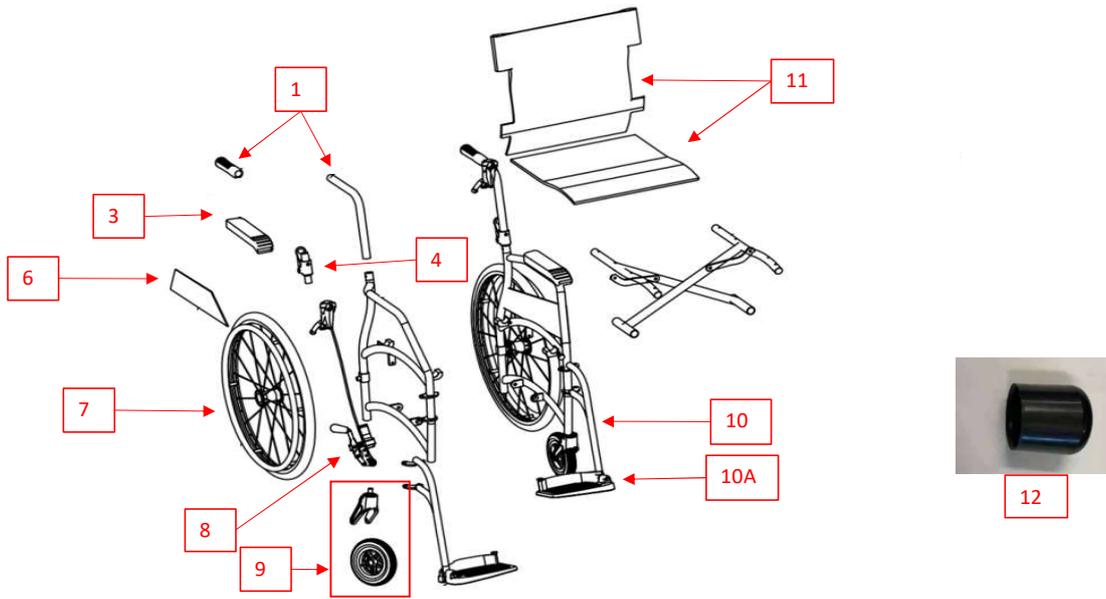
Aspire VIDA X Wheelchairs Spare Parts Manual



Item Number	Item Description
MWS449867	WChair Folding - Aspire VIDA X SP - 450mm - 135kg - Black
MWS449868	WChair Folding - Aspire VIDA X SP - 450mm - 135kg - Purple

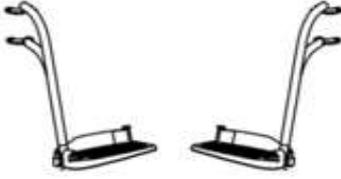
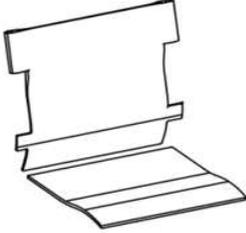
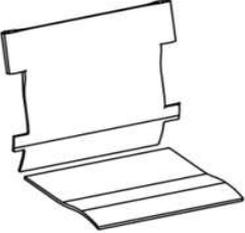
This manual is for internal use only

1 Castors, Seating & Armrest



Ref	Item Number	Item Description	Image	Type
1	GSP489200	Backrest Cane w/Handgrip - Aspire VIDA X Wheelchair - SP - Black - PAIR		Spare Part
1	GSP489202	Backrest Cane w/Hand Grip - Aspire VIDA X Wheelchair - SP - Purple - PAIR		Spare Part
3	GSP489100	Armrest Pad - Aspire DASH AP / SP / VIDA X Wheelchair - PAIR		Spare Part
4	GSP489125	Mounting Hardware - for Backrest Cane - Aspire DASH AP / SP / VIDA X Wheelchair - PAIR		Spare Part

1 Castors, Seating & Armrest				
6	GSP489150	Side Arm Fabric Guard - Aspire DASH AP / SP / VIDA X Wheelchair - PAIR		Spare Part
7	GSP489204	22in Rear Wheel w/Tyre - Aspire VIDA X Wheelchair		Spare Part
7A	GSP489214	Quick Release Axle - Aspire VIDA X Wheelchair		Spare Part
8	GSP489115	Brake Handle and Cable Assembly - Aspire DASH SP / VIDA X Wheelchair - PAIR		Spare Part
9	GSP489120	Front Fork and Castor Assembly - Aspire DASH AP / SP / VIDA X Wheelchair - LEFT		Spare Part
9	GSP489121	Front Fork and Castor Assembly - Aspire DASH AP / SP / VIDA X Wheelchair - RIGHT		Spare Part

1 Castors, Seating & Armrest				
10	GSP489206	Swing Away Footrest Assembly - Aspire VIDA X Wheelchair - SP - Black - PAIR		Spare Part
10	GSP489208	Swing Away Footrest Assembly - Aspire VIDA X Wheelchair - SP - Purple - PAIR		Spare Part
10A	GSP489160	Foot Plates and Calf Strap - Aspire DASH AP / SP / VIDA X Wheelchair - LEFT		Spare Part
10A	GSP489162	Foot Plates and Calf Strap - Aspire DASH AP / SP / VISA X Wheelchair - RIGHT		Spare Part
11	GSP489210	Seat and Back Upholstery - Aspire VIDA X Wheelchair - SP - Black		Spare Part
11	GSP489212	Seat and Back Upholstery - Aspire VIDA X Wheelchair - SP - Purple		Spare Part

1 Castors, Seating & Armrest				
12	GSP489165	End Cap - Aspire DASH AP / SP / VIDA X Wheelchair		Spare Part