

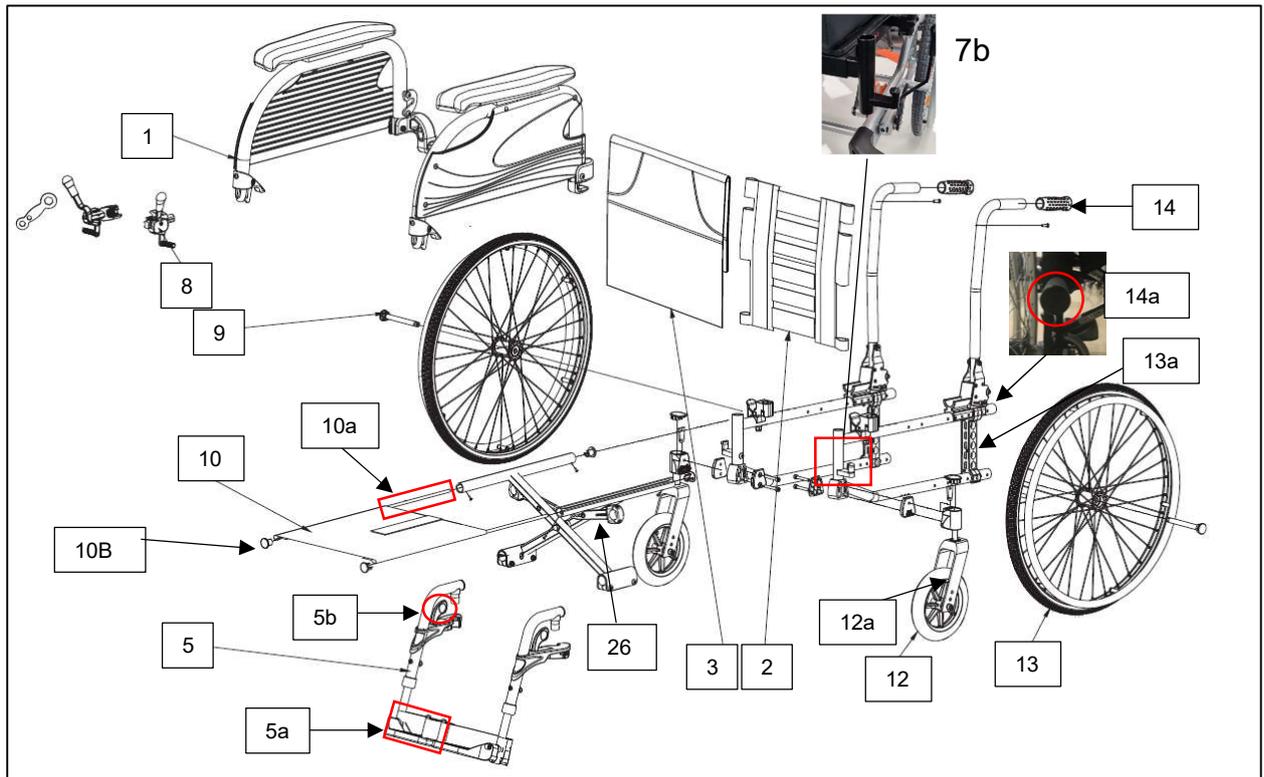
**Evoke 2 / Evoke 2 HD
Wheelchair Spare Parts Manual**



Evoke 2 / Evoke 2 HD

Aidacare Code	Aidacare Description
MWS449500	Aspire Evoke 2 – Standard - 350mm
MWS449510	Aspire Evoke 2 – Standard – 400mm
MWS449520	Aspire Evoke 2 – Standard – 450mm
MWS449530	Aspire Evoke 2 – Standard – 500mm
MWS449540	Aspire Evoke 2 – HD – 570mm
MWS449550	Aspire Evoke 2 – HD – 610mm

Exploded Diagram



Additional Spare Parts and Accessories



15



16



17



18



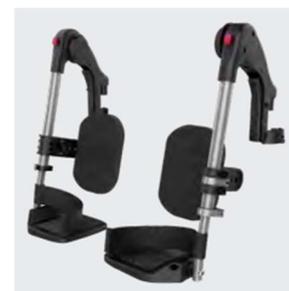
19



20



6



7



7a



1a



1b



4



6b



11



21



22



23



24

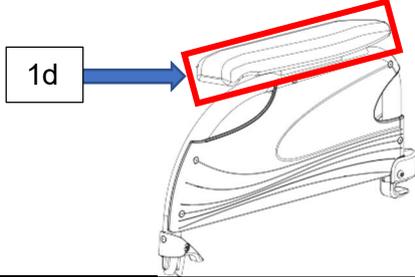
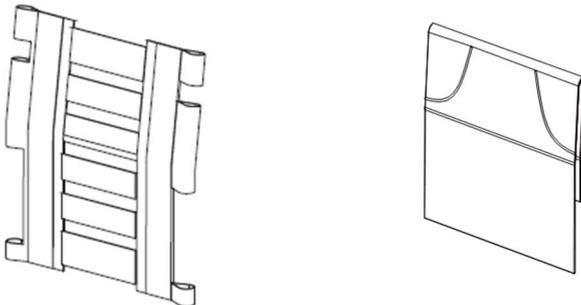


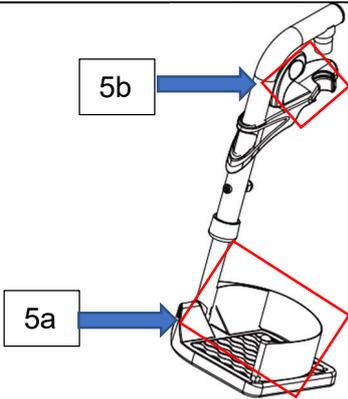
27

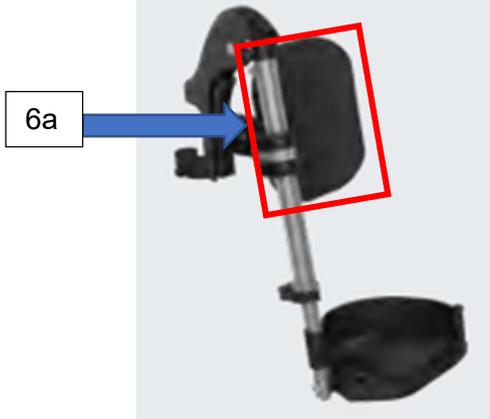


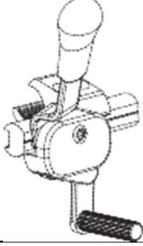
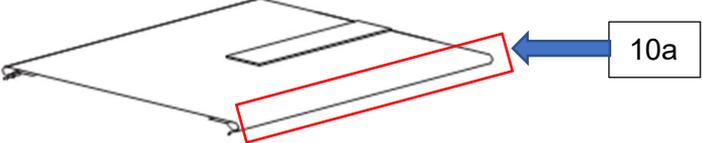
28

Spare Part List

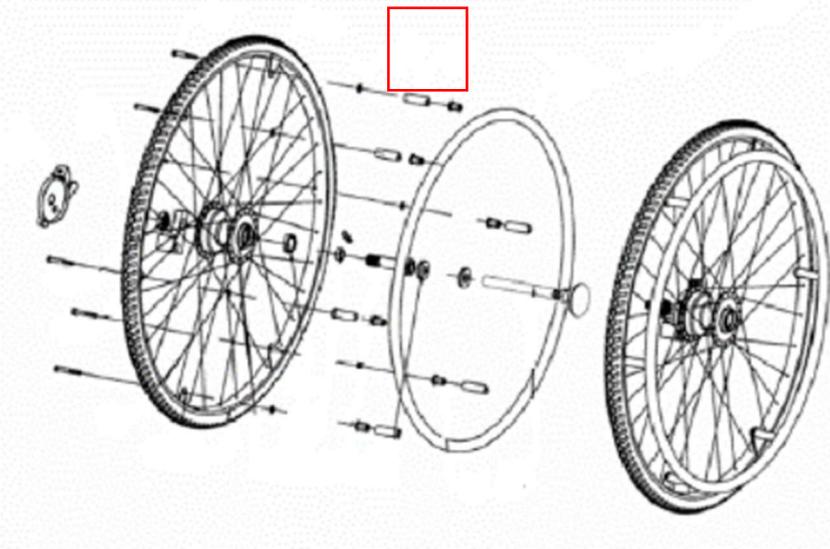
No.	Item Number	Item Description	Type
1	GSP492372	Flip Up Armrest Assembly – Left	Spare Part
1	GSP492374	Flip Up Armrest Assembly – Right	Spare Part
1d	GSP492384	Armrest Pad – Left	Spare Part
1d	GSP492385	Armrest Pad - Right	Spare Part
			
1a	WCA848410	Gutter Armrest Pad – Left	Accessory
1a	WCA848420	Gutter Armrest Pad – Right	Accessory
			
1b	WCA848390	Height Adjustable Armrest – Left	Accessory
1b	WCA848400	Height Adjustable Armrest – Right	Accessory
1c	GSP492375	Height Adjustment Button	Spare Part
			
2/3	GSP492822	Backrest Upholstery – Tension Adjustable / Padded – 350mm	Spare Part
2/3	GSP492824	Backrest Upholstery – Tension Adjustable / Padded – 400mm	Spare Part
2/3	GSP492826	Backrest Upholstery – Tension Adjustable / Padded – 450mm	Spare Part
2/3	GSP492828	Backrest Upholstery – Tension Adjustable / Padded – 500mm	Spare Part
2/3	GSP492830	Backrest Upholstery – Tension Adjustable / Padded – 570mm	Spare Part
2/3	GSP492832	Backrest Upholstery – Tension Adjustable / Padded – 610mm	Spare Part
			

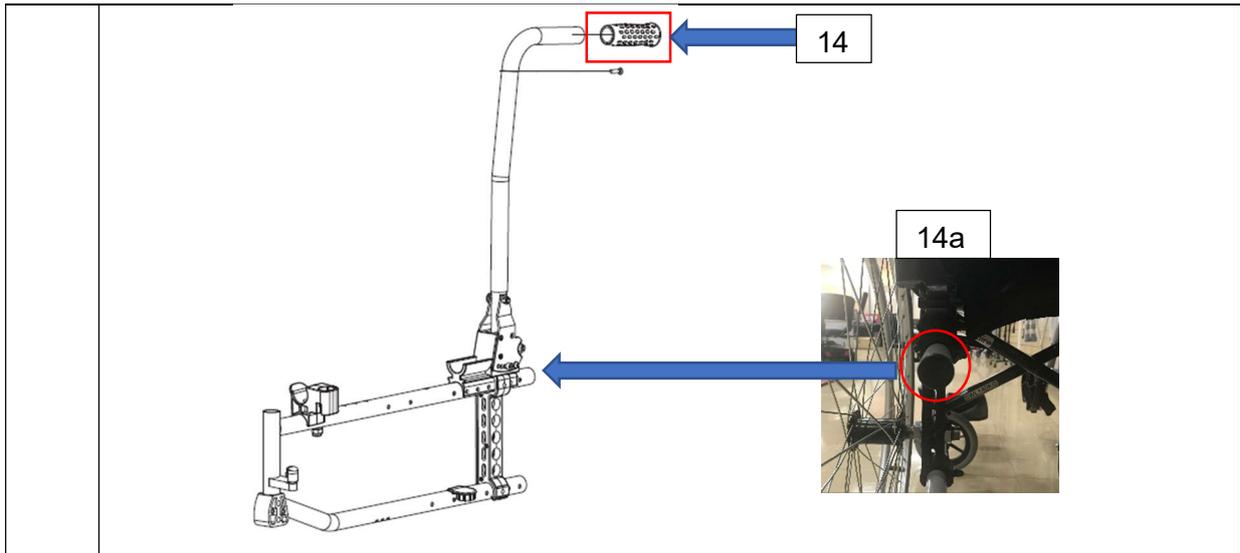
4	WCA848330	Contoured Padded Overlay – 350mm	Accessory
4	WCA848340	Contoured Padded Overlay – 400mm	Accessory
4	WCA848350	Contoured Padded Overlay – 450mm	Accessory
4	WCA848360	Contoured Padded Overlay – 500mm	Accessory
4	WCA848370	Contoured Padded Overlay – 570mm	Accessory
4	WCA848380	Contoured Padded Overlay – 610mm	Accessory
			
5	GSP492408	Swing Away Legrest – Wide - 350mm	Spare Part
5	GSP492414	Swing Away Legrest – Wide - 400mm	Spare Part
5	GSP492418	Swing Away Legrest – Wide - 450mm	Spare Part
5	GSP492422	Swing Away Legrest – Wide - 500mm	Spare Part
5	GSP492424	Swing Away Legrest – Wide – 570 to 610mm	Spare Part
5a	GSP492428	Heel Strap – Wide - 350 to 400mm	Spare Part
5a	GSP492430	Heel Strap – Wide – 450 to 610mm	Spare Part
5b	GSP492860	Quick Release Stopper for Swing Away Legrest	Spare Part
			
6	WCA848100	Elevating Legrest – Standard – Left – 320 to 410mm	Accessory
6	WCA848110	Elevating Legrest – Standard – Right – 320 to 410mm	Accessory
6	WCA848120	Elevating Legrest – Standard – Left – 450 to 510mm	Accessory
6	WCA848130	Elevating Legrest – Standard – Right – 450 to 510mm	Accessory
6	WCA848140	Elevating Legrest – Standard – Left – 570 to 610mm	Accessory
6	WCA848150	Elevating Legrest – Standard – Right – 570 to 610mm	Accessory
6	WCA848160	Elevating Legrest – Advanced – Left – 320 to 410mm	Accessory
6	WCA848170	Elevating Legrest – Advanced – Right – 320 to 410mm	Accessory
6	WCA848180	Elevating Legrest – Advanced – Left – 450 to 510mm	Accessory
6	WCA848190	Elevating Legrest – Advanced – Right – 450 to 510mm	Accessory
6a	GSP492433	Calf Pad Assembly – Left – 410 to 510mm	Spare Part
6a	GSP492435	Calf Pad Assembly – Right – 410 to 510mm	Spare Part
6a	GSP492366	Calf Pad Assembly – Left – 320 to 360mm	Spare Part
6a	GSP492365	Calf Pad Assembly – Right – 320 to 360mm	Spare Part

				
6b	WCA848580	Calf Strap		Accessory
				
7	WCA848210	Elevating Legrest – Articulating – Right - 570 to 610mm		Accessory
7	WCA848200	Elevating Legrest – Articulating – Left - 570 to 610mm		Accessory
				
7a	GSP492870	Soft Pad for Leg Rest, Left		Spare Part
7a	GSP492875	Soft Pad for Leg Rest, Right		Spare Part
				
7b	GSP492880	Elevating Legrest - Locking Arm		Spare Part

			
8	GSP492392	Brake Assembly – Hand Brake – Left	Spare Part
8	GSP492394	Brake Assembly – Hand Brake – Right	Spare Part
8	WCA848310	Brake Extension Lever – Left	Accessory
8	WCA848320	Brake Extension Lever – Right	Accessory
			
9	GSP492462	Quick Release Axle	Spare Part
			
10	GSP492774	Seat Upholstery – 350mm - PRIOR SN EV2_PO725025_0001-0088	Spare Part
10	GSP492776	Seat Upholstery – 400mm - PRIOR SN EV2_PO725025_0001-0088	Spare Part
10	GSP492778	Seat Upholstery – 450mm - PRIOR SN EV2_PO725025_0001-0088	Spare Part
10	GSP492780	Seat Upholstery – 500mm - PRIOR SN EV2_PO725025_0001-0088	Spare Part
10	GSP492782	Seat Upholstery – 570mm - PRIOR SN EVH2_PO725025_0089--0111	Spare Part
10	GSP492784	Seat Upholstery – 610mm - PRIOR SN EVH2_PO725025_0089--0111	Spare Part
10	GSP492774	Seat Upholstery – 350mm - AFTER SN EV2_PO725025_0001-0088	Spare Part
10	GSP492777	Seat Upholstery – 400mm - AFTER SN EV2_PO725025_0001-0088	Spare Part
10	GSP492779	Seat Upholstery – 450mm - AFTER SN EV2_PO725025_0001-0088	Spare Part
10	GSP492781	Seat Upholstery – 500mm - AFTER SN EV2_PO725025_0001-0088	Spare Part
10	GSP492783	Seat Upholstery – 570mm - AFTER SN EVH2_PO725025_0089--0111	Spare Part
10	GSP492785	Seat Upholstery – 610mm - AFTER SN EVH2_PO725025_0089--0111	Spare Part
10a	GSP492771	Rods for Seat Upholstery – LONG	Spare Part
10a	GSP492773	Rods for Seat Upholstery – SHORT	Spare Part
			

10B	GSP492890	Welded Extension Tube - for Seat Upholstery - suits Evoke 2 WChairs (500mm and above)	Spare Part
10B	GSP492892	Welded Extension Tube - for Seat Upholstery - suits Evoke 2 WChairs (smaller than 500mm)	Spare Part
11	GSP492468	Seat Cushion – 350mm	Spare Part
11	GSP492470	Seat Cushion – 400mm	Spare Part
11	GSP492472	Seat Cushion – 450mm	Spare Part
11	GSP492474	Seat Cushion – 500mm	Spare Part
11	GSP492476	Seat Cushion – 570mm	Spare Part
11	GSP492478	Seat Cushion – 610mm	Spare Part
12	GSP492446	Front Wheel – Castor - PRIOR SN EV2_PO725025_0001-0088	Spare Part
12	GSP492447	Front Wheel – Castor - AFTER SN EV2_PO725025_0001-0088	Spare Part
12a	GSP492454	Front Wheel Fork – Prior SN EV2_PO725025_0001_0088	Spare Part
12a	GSP492455	Front Wheel Fork Assly – Left – After SN EV2_PO725025_0001_0088	Spare Part
12a	GSP492457	Front Wheel Fork Assly – Right – After SN EV2_PO725025_0001_0088	Spare Part
12b	GSP492451	End Cap for Front Wheel Fork - PRIOR SN EV2_PO725025_0001-0088	Spare Part
12b	GSP492453	End Cap for Front Wheel Fork - AFTER SN EV2_PO725025_0001-0088	Spare Part
13	GSP492440	Rear Wheel – 24 Inch	Spare Part

			
13a	GSP492445	Wheelchair Rear Wheel Spacer and Screw	Spare Part
			
13a	GSP492854	Rear Axle Mounting Plate with Holder	Spare Part
			
14	GSP492842	Hand Grip	Spare Part
14a	GSP492856	Rear Frame Tip Plastic Cover	Spare Part



15	GSP492858	Tool Kit	Spare Part
----	-----------	----------	------------



16	WCA848220	Stump Support – Left – 320 to 450mm	Accessory
16	WCA848230	Stump Support – Right – 320 to 450mm	Accessory
16	WCA848240	Stump Support – Left – 500 to 610mm	Accessory
16	WCA848250	Stump Support -Right – 500 to 610mm	Accessory
16	WCA848260	Stump Support – CoG Extension Bracket	Accessory

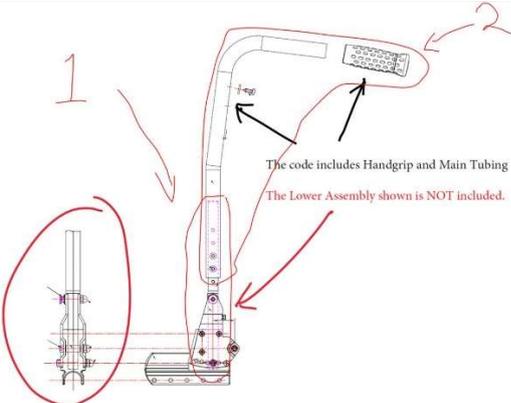


17	WCA848270	Anti Tip Bar	Accessory
----	-----------	--------------	-----------



18	WCA848280	IV Pole	Accessory
----	-----------	---------	-----------

			
19	WCA848290	Oxygen Bottle Holder	Accessory
			
20	WCA848300	Lap Belt – Metal Seatbelt Buckle	Accessory
			
21	WCA848430	Lateral Chest Support Assembly – Left	Accessory
21	WCA848440	Lateral Chest Support Assembly – Right	Accessory
21.1	GSP492857	Plastic Bush – for Lateral Chest Support	Spare Part
		 	21.1

22	WCA848470	Extended Back Canes – Pair – 19/21 inch	Accessory
			
23	WCA848480	Headrest Assembly – for 400/450mm * NOT SUITABLE FOR 350mm	Accessory
23	WCA848481	Headrest Assembly – for 500/570/610mm	Accessory
23A	GSP492868	Mounting Clamps for Headrest - PAIR	Spare Part
			
24	WCA848500	Drum Brake Kit – 24inch	Accessory
			
25(1)	GSP492841	Lower Mounting Assembly – PAIR	Spare Part
25(2)	GSP492839	Backrest Cane – Standard – EACH	Spare Part
		 <p>The code includes Handgrip and Main Tubing The Lower Assembly shown is NOT included.</p>	

26	GSP492900	Hook Cross Bracket – 350mm	Spare Part
26	GSP492905	Hook Cross Bracket – 400mm	Spare Part
26	GSP492910	Hook Cross Bracket – 450 / 500mm	Spare Part
26	GSP492915	Hook Cross Bracket – 570mm	Spare Part
26	GSP492917	Hook Cross Bracket – 610mm	Spare Part
			

27	WCA848594	Wheel & Brake Kit – Aspire – 12in Wheels + Attendant Brakes – Suits 350mm to 500mm Evoke2, Assist2 Wheelchairs only
28	WCA848592	Attendant Brakes – Brakes Only (No Wheels) – Suits Evoke2, Assist2, Transit2 Wheelchairs
		
<div style="display: flex; justify-content: space-around; width: 100%;"> 27 28 </div>		