


Spare Parts Diagram - Aspire Hybrid Mattress Pumps - V1

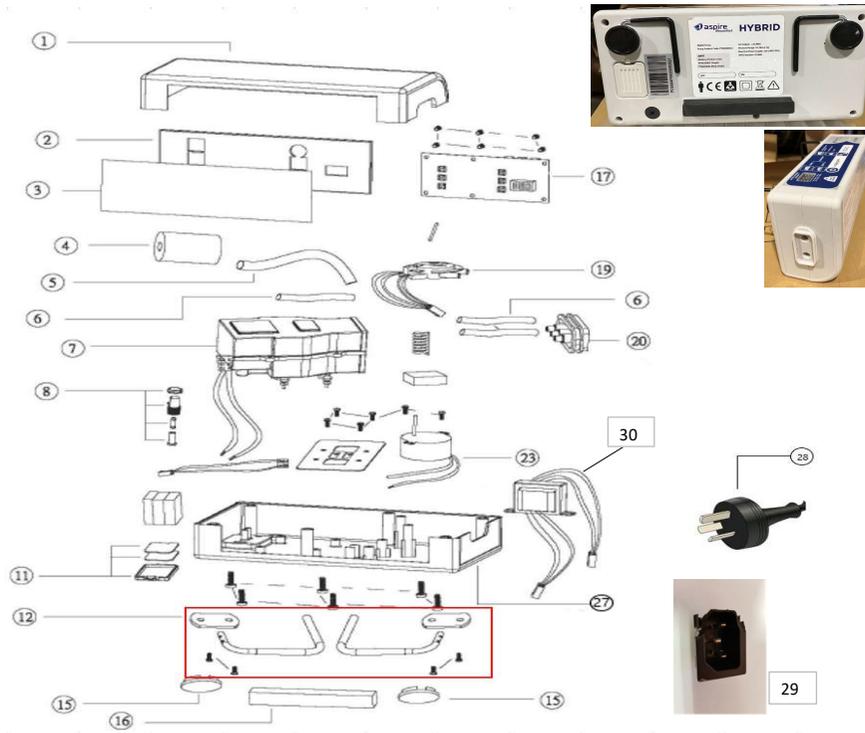

Code	Product Description
PTM638030	Aspire Lifecomfort Hybrid Mattress - Digital Pump - V1

No	Code	Spare Parts
1	GSP494320	Top Case - Aspire Hybrid / Hybrid Acute Mattress Pump - V1
2	GSP494250	Front Panel - Aspire Hybrid / Hybrid Acute Mattress Pump - V1
3	GSP494380	Membrane Panel - Aspire Hybrid / Hybrid Acute Mattress Pump - V1
4	GSP494340	Air Buffer - Aspire Hybrid / Hybrid Acute Mattress Pump - V1
5	GSP494160	Internal Air Tubing - Aspire Hybrid/Hyvrud Acute Mattress Pump - 7.2mm - V1/V2
6	GSP494165	Internal Air Tubing - Aspire Hybrid/Hyvrud Acute Mattress Pump - 5.5mm - V1/V2
7	GSP494130	Compressor - Aspire Hybrid/Hybrid Acute Mattress Pump - V1
8	GSP494210	Fuse - Aspire Hybrid/Hybrid Acute Mattress Pump - V1
11	GSP494100	Pump Filter Cap - Aspire Hybrid/Hybrid Acute Mattress Pump - V1
11	GSP494110	Pump Filter - Aspire Hybrid/Hybrid Acute Mattress Pump - V1
12	GSP494260	Pump Bracket Assembly - Aspire Hybrid / Hybrid Acute Mattress Pump - V1
15	GSP494240	Plastic Feet for Pump - Aspire Hybrid/Hybrid Acute Mattress Pump - V1
16	GSP494120	Rear Foam Pad for Pump - Aspire Hybrid/Hybrid Acute Mattress Pump - V1
17	GSP494200	Controlled Circuit Board - Aspire Hybrid/Hybrid Acute Mattress Pump - V1
19	GSP494360	Air Valve - Aspire Hybrid / Hybrid Acute Mattress Pump - V1/V2
20	GSP494230	Air Valve Assembly - Aspire Hybrid/Hybrid Acute Mattress Pump - V1/V2
23	GSP494150	Motor - Aspire Hybrid/Hybrid Acute Mattress Pump - V1/V2
27	GSP494330	Base Case - Aspire Hybrid/Hybrid Acute Mattress Pump - V1
28	GSP494190	Main Power Cable - Aspire Hybrid/Hybrid Acute Mattress Pump - V1/V2
29	GSP494195	Lead Connector Power Socket - Aspire Hybrid/Hybrid Acute Mattress Pump - V1
30	GSP494155	Transformer - Aspire Hybrid / Hybrid Acute Mattress Pump - V1/V2