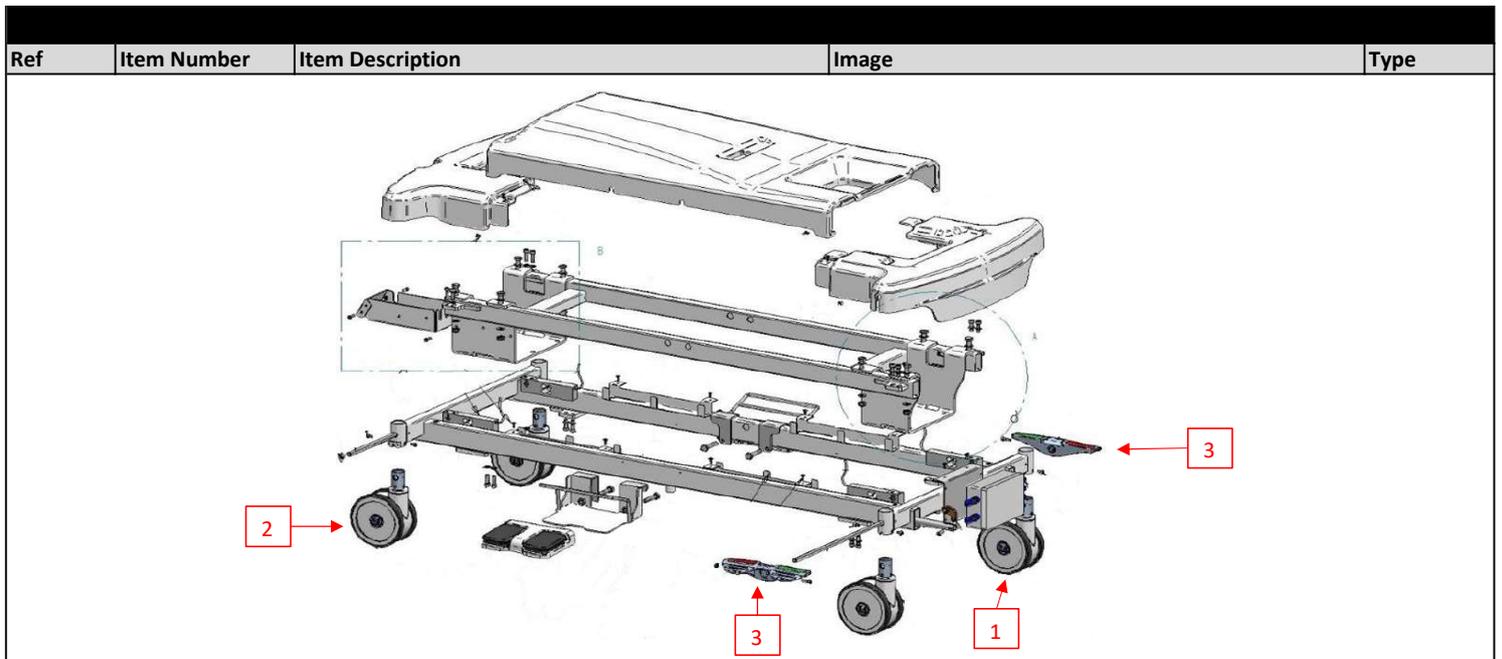


Malvestio Aria 3960B Bed Spare Parts Manual

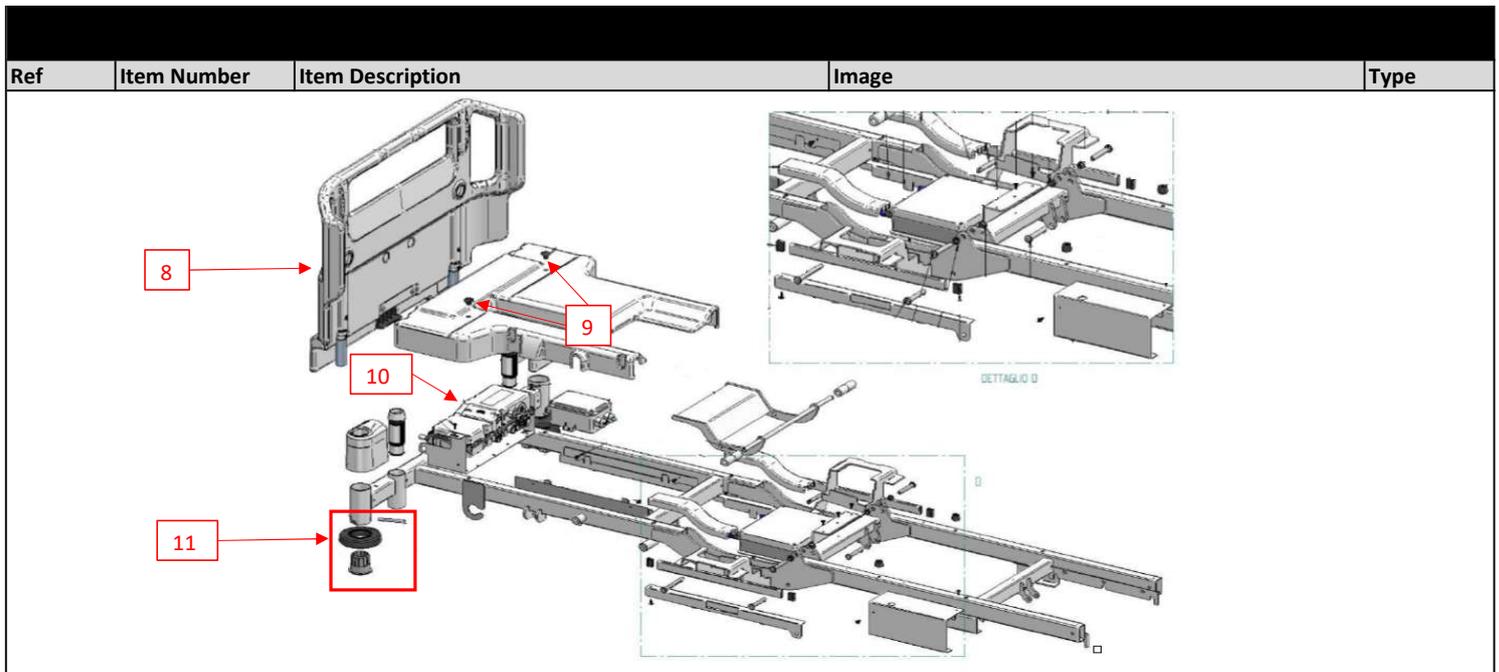


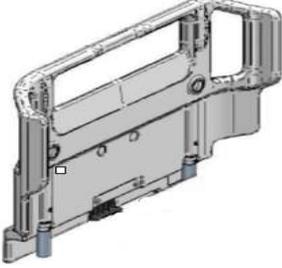
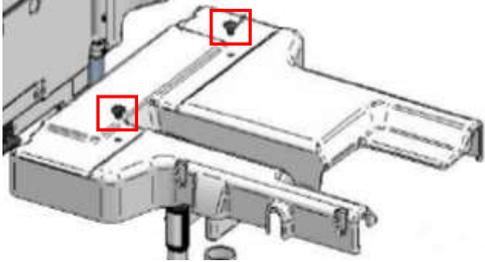
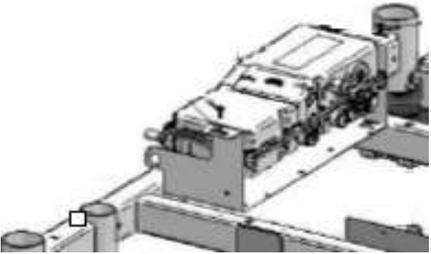
Item Number	Item Description
BEB050160	Malvestio 3960B Aria Bed

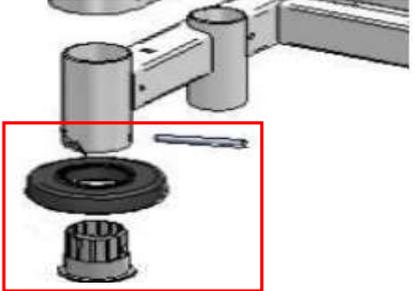
This manual is for internal use only



Ref	Item Number	Item Description	Image	Type
1	GSP872640	Total Brake Castor - Malvestio Aria Bed		Spare Part
2	GSP498115	Directional Braking Castor - Malvestio Aria Bed		Spare Part
3	GSP872570	Brake pedal - Malvestio Aria Bed		Spare Part

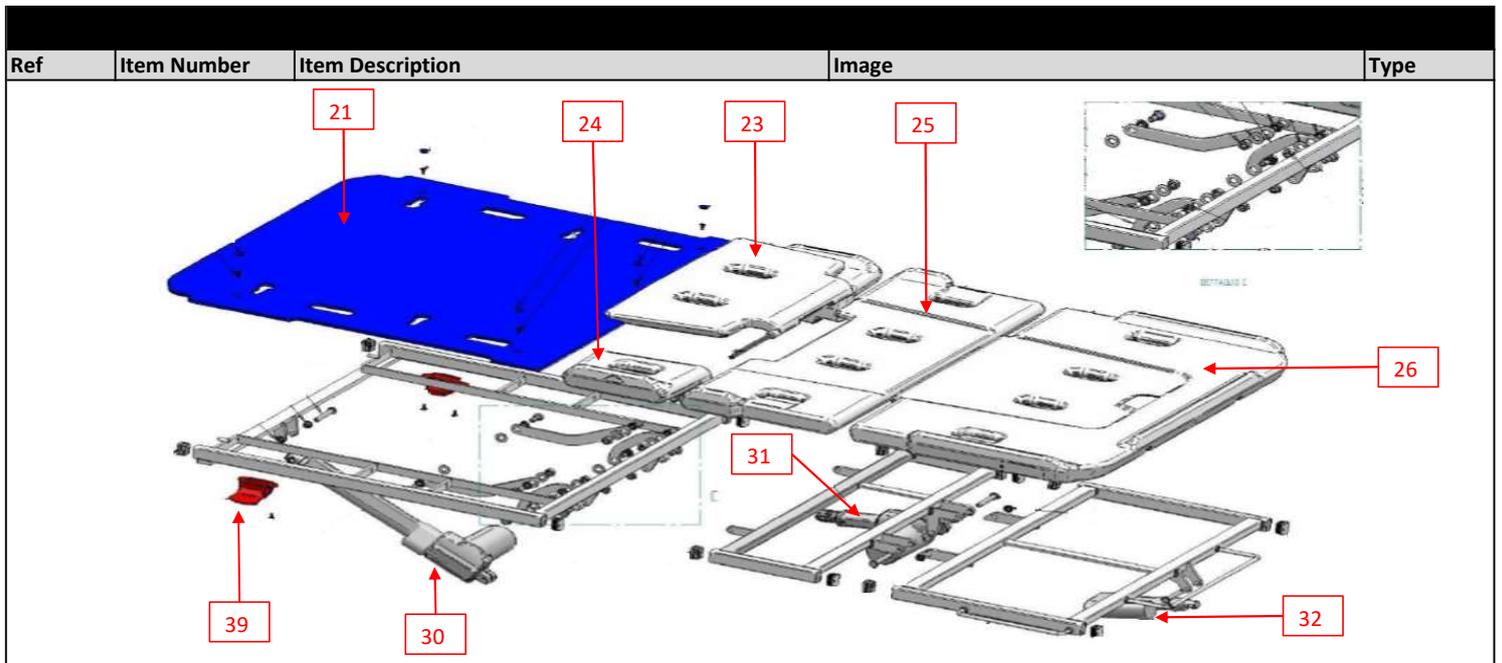


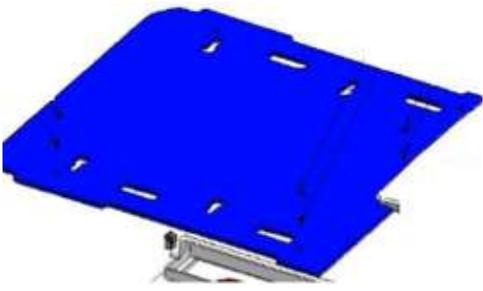
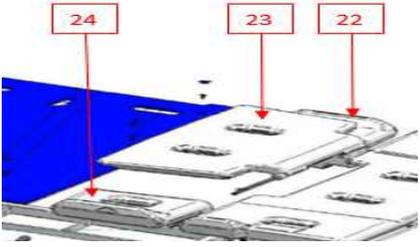
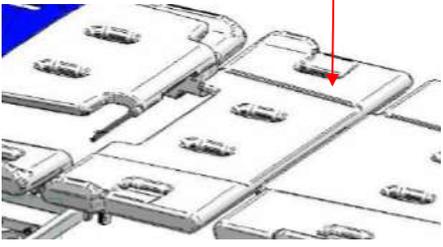
Ref	Item Number	Item Description	Image	Type
8	GSP872608	Complete Head Board - for Drive Wheel - Malvestio Aria Bed		Spare Part
9	GSP872572	Shock Absorbing Buffer for Backrest - Malvestio Aria Bed		Spare Part
10	GSP872600	Control Box - Malvestio Aria Bed		Spare Part
10A	GSP872602	Power Cord - Malvestio Aria Bed	Not On Drawing	Spare Part

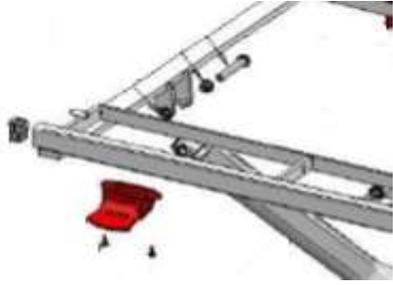
Ref	Item Number	Item Description	Image	Type
11	GSP497410	Wall Bumper Complete - Malvestio Aria / Delta 4 Bed		Spare Part

Ref	Item Number	Item Description	Image	Type

Ref	Item Number	Item Description	Image	Type
16	GSP872624	Lifting Column w/ Mounting Bolts - suits Head / Foot Side - Malvestio Aria Bed Does not incl Hardware & Brackets		Spare Part
18	GSP872630	Tilting Actuator - Malvestio Aria Bed		Spare Part

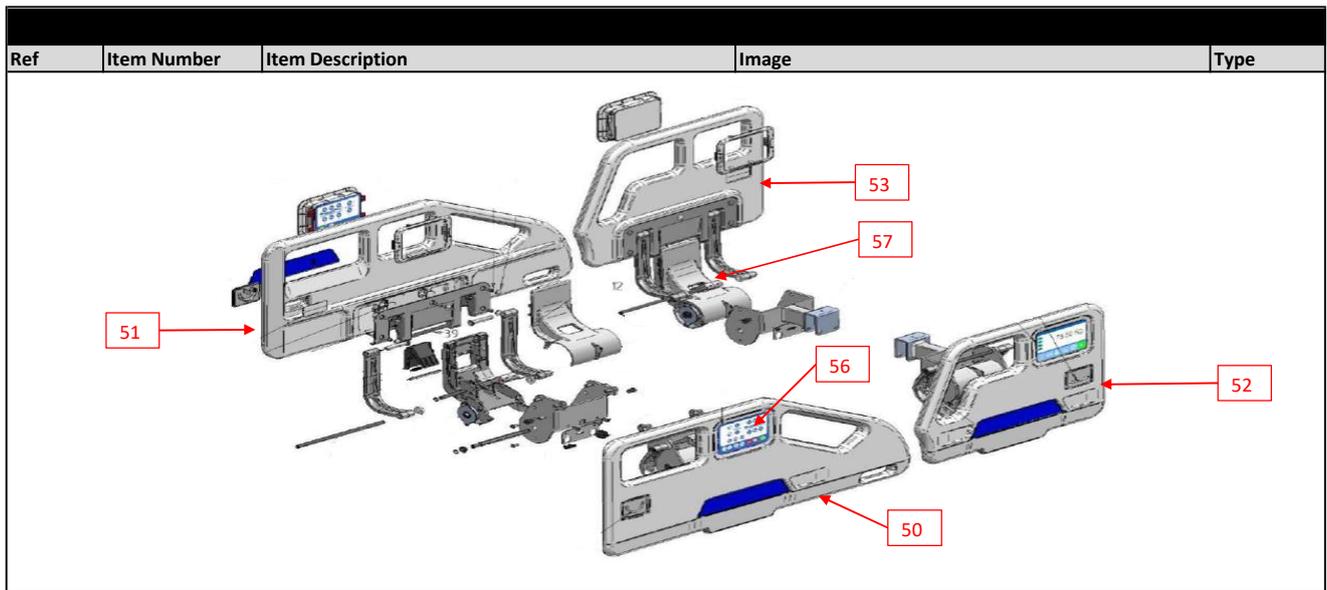


Ref	Item Number	Item Description	Image	Type
21	GSP872574	Backrest Platform - Laminated - Malvestio Aria Bed		Spare Part
23	GSP872636	Fixed Mid Section Platform - Malvestio Aria Bed		Spare Part
25	GSP872634	Knee-Break Section Platform - Malvestio Aria Bed		Spare Part
26	GSP872632	Legrest Section Platform - Malvestio Aria Bed		Spare Part

Ref	Item Number	Item Description	Image	Type
30	GSP872622	Backrest Actuator - Malvestio Aria Bed	N/A	Spare Part
31	GSP872620	Knee-Break Actuator - Malvestio Aria Bed	NA	Spare Part
32	GSP872576	Legrest Actuator - Malvestio Aria Bed	N/A	Spare Part
39	GSP872642	Backrest Quick Release Handle - Malvestio Aria Bed		Spare Part
40	GSP872578	Backrest Release Cables - Pair - Malvestio Aria Bed	Not on Drawing	Spare Part

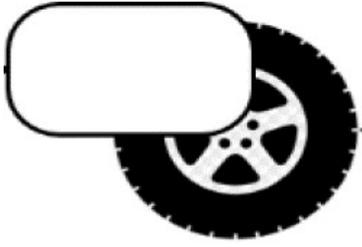
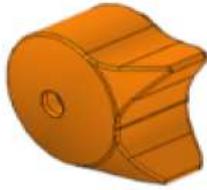
Ref	Item Number	Item Description	Image	Type

Ref	Item Number	Item Description	Image	Type
41	GSP872580	Complete Foot Board - Malvestio Aria Bed		Spare Part
45	GSP872614	Bed Extension Handle - Malvestio Aria Bed		Spare Part
46	GSP872612	Bed Extension - Malvestio Aria Bed		Spare Part



Ref	Item Number	Item Description	Image	Type
50	GSP872645	Side Rail - w/mounting mechanism - Right Head - Ref - 3960B - Malvestio Aria Bed	Please see above	Spare Part
51	GSP872646	Side Rail - w/mounting mechanism - Left Head - Ref - 3960B - Malvestio Aria Bed	Please see above	Spare Part
52	GSP872647	Side Rail - w/mounting mechanism - Right Foot - Ref - 3960B - Malvestio Aria Bed	Please see above	Spare Part
53	GSP872648	Side Rail - w/mounting mechanism - Left Foot - Ref - 3960B - Malvestio Aria Bed	Please see above	Spare Part

56	GSP872616	Control Panel - Right - w/ Nurse Call and Tilt (Side Rails - Head End) - Malvestio Aria Bed	Please see above	Spare Part
57	GSP497497	Unhooking Handle Kit for Side Rails - for Malvestio Delta 4 / Aria Beds		Spare Part

Ref	Item Number	Item Description	Image	Type
60	GSP872606	Drive Wheel - Malvestio Aria Bed		Spare Part
61	GSP872582	Control for Drive Wheel - Malvestio Aria Bed		Spare Part
62	GSP872584	Control Box for Drive Wheel - Malvestio Aria Bed	Picture N/A at this stage	Spare Part