

## Mangar Camel, Eagle, Elk & Airflo 24 Pump Spare Parts Manual



Product code: CD0100-24



Product code: HKA00500



Product code: HXA0100




Product code: HEA0033

Item Number	Item Description
GSP874521	Replacement Pump - Airflo 24 - Mangar ELK, CAMEL, EAGLE
LSS403810	Lift Cushion - Mangar CAMEL - 320kg
LSS403800	Lift Cushion - Mangar EAGLE - 320kg
LSS403820	Lift Cushion - Mangar ELK - 450kg

**This manual is for internal use only**

# 1 Airflo 24 Pump

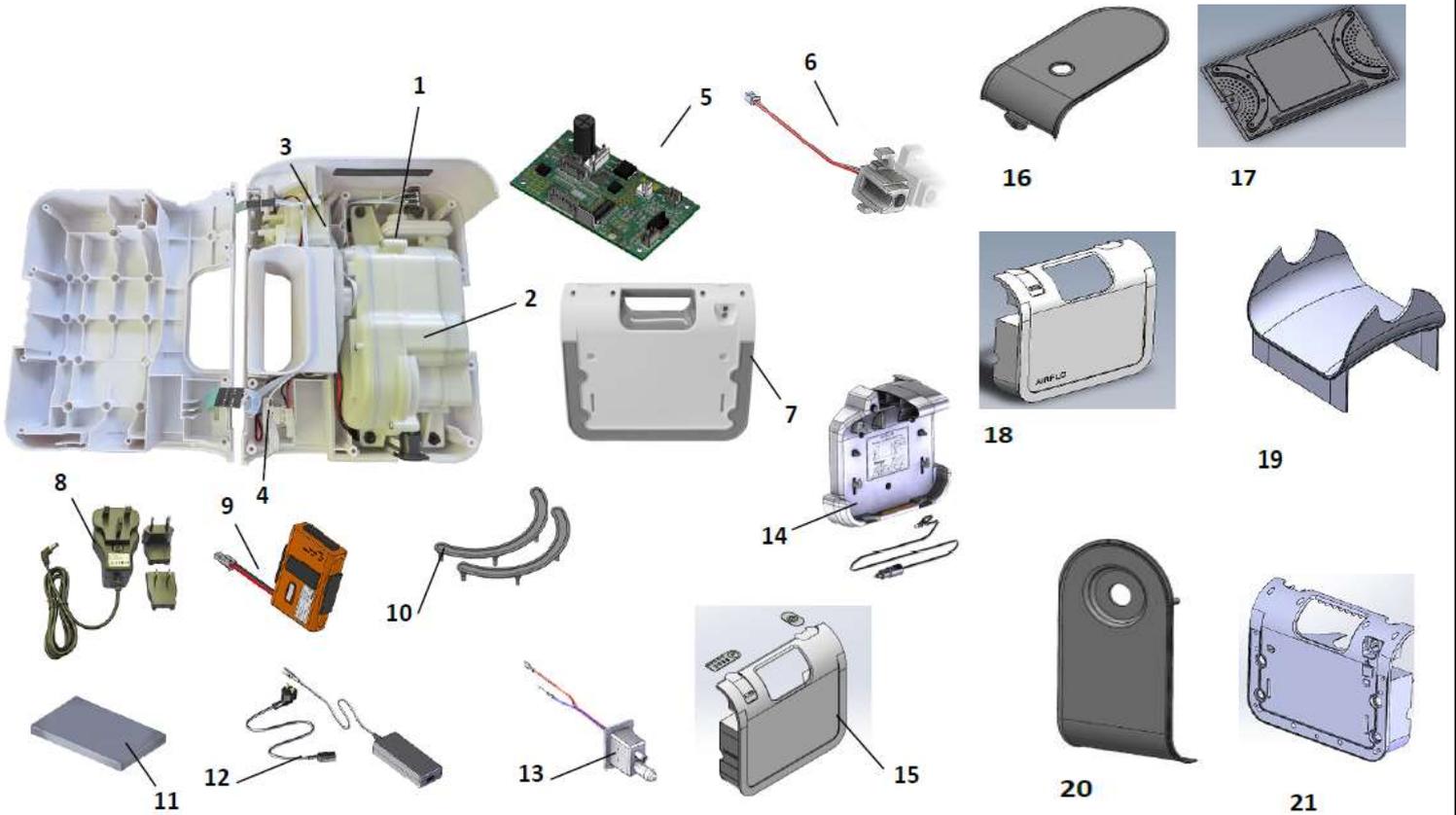

Ref	Item Number	Supplier Code	Description	Type
1	POA	CD0151-24	Outlet Valve - Airflo 24	Spare Part
2	POA	CD0190	Pump Assembly, Airflo 12 & 24	Spare Part
3	GSP874512	CD0244	Membrane (Control Side) - Airflo 24 - Mangar ELK, CAMEL & EAGLE	Spare Part
4	GSP874513	CD0242	Membrane (Battery Side) - Airflo 24 - Mangar ELK, CAMEL & EAGLE	Spare Part
5	GSP874511	CD0327	PCB Kit - Mangar - Airflo 24 - Mangar ELK, CAMEL & EAGLE	Spare Part
6	POA	CD0325	Airflo Charging Socket Kit	Spare Part
7	GSP874514	CD0310	Foot Casing Kit - Mangar - Airflo 24 - Mangar ELK, CAMEL & EAGLE	Spare Part
8	GSP874522	CD0332-24AU	Battery Charger - Airflo 24 - Mangar ELK, CAMEL & EAGLE	Spare Part
9	GSP874500	CD0314	Replacement Battery Pack - Airflo 24 - Mangar ELK, CAMEL & EAGLE	Spare Part
10	GSP874515	CD0326	Foot Case Kit - Mangar - Airflo 24 - Mangar ELK, CAMEL & EAGLE	Spare Part

<b>1 Airflo 24 Pump</b>				
11	POA	CD0181	Inlet Filter, Airflo 12 & 24	Spare Part
12	POA	CD0263-24	Airflo Mains Adaptor - Airflo 24	Spare Part
13	POA	CD0311	Charger Plug Kit	Spare Part
14	NA			Spare Part
15	POA	CD0330	Airflo 24 Front Case Kit	Spare Part
16	POA	CD0175-24	Battery Panel - Airflo 24	Spare Part
17	POA	CD0174-24	Base Panel - Airflo 24	Spare Part
18	GSP874516	CD0171	Front Pump Case - Mangar - Airflo 24 - Mangar ELK, CAMEL & EAGLE	Spare Part
19	POA	CD0177-24	Handle Panel - Airflo 24	Spare Part
20	POA	CD0176-24	Port Panel - Airflo 24	Spare Part
21	GSP874517	CD0306	Rear Pump Case - Mangar - Airflo 24 - Mangar ELK, CAMEL & EAGLE	Spare Part

## 2 Camel, Eagle & Elk Cushions



30



31



32




33



34




35

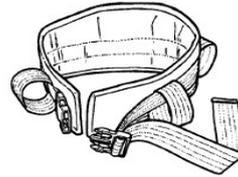

36



37

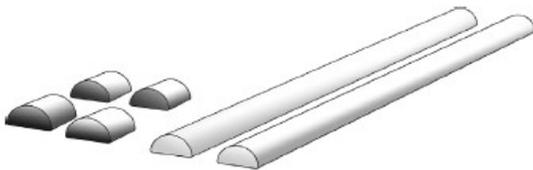

38



39

Ref	Item Number	Supplier Code	Description	Type
30	GSP874503	HK0064	Hand Control - Mangar CAMEL & EAGLE	Spare Part
30	GSP874502	HE0117	Hand Control - Mangar ELK	Spare Part
31	POA	CC0317	Coupling set - Eagle, Camel & Elk	Spare Part
32	POA	CC0299	Seal Quick Release Coupling - Eagle, Camel & Elk	Spare Part
33	POA	CA0446	Double Paddled Connector 11mm - Eagle, Camel & Elk	Spare Part
34	POA	HK0040	Camel Bag - Camel	Spare Part
34	POA	HX0040	Eagle Bag - Eagle	Spare Part
35	POA	KA0001	Slideways Full Body 120x180 long - Camel	Spare Part
36	POA	HG0024	Transfer Belt - Camel	Spare Part
37	POA	HE0073	Elk End Cap Set - Elk	Spare Part

<b>2 Camel, Eagle &amp; Elk Cushions</b>				
38	POA	HE0041	Elk Stabilising Bar Set - Elk	Spare Part
39	GSP874523	HE0032	Stretcher Bars - Mangar ELK	Spare Part