

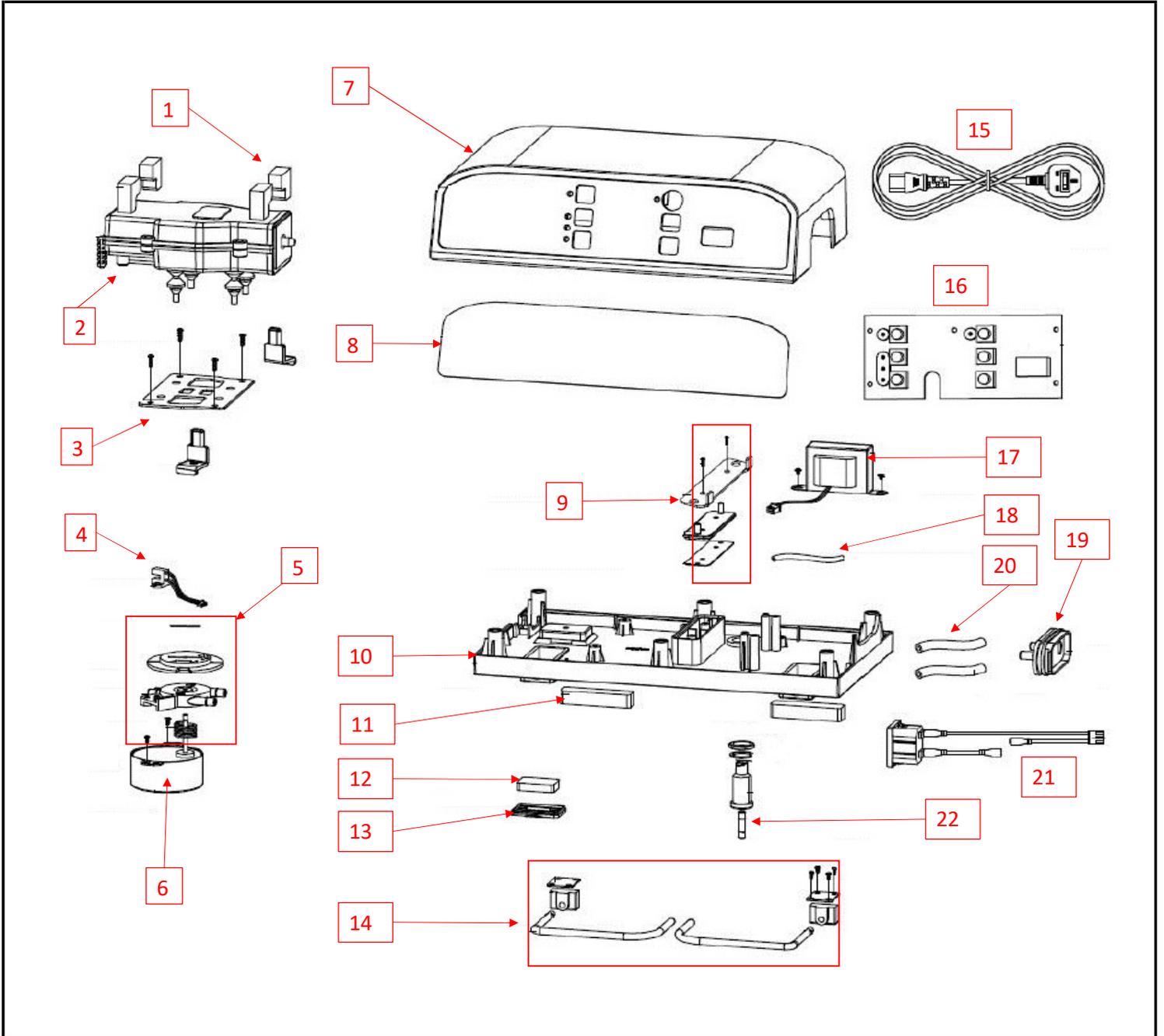
Spare Parts Manual



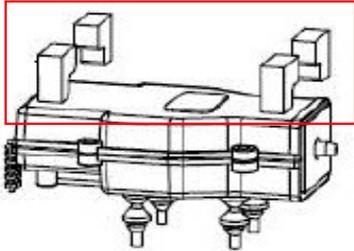
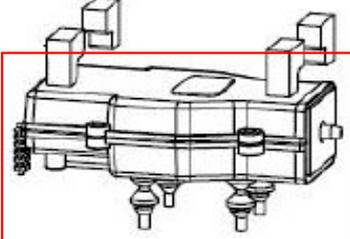
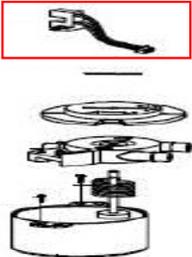
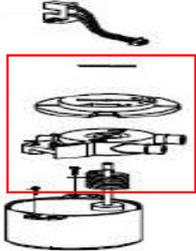
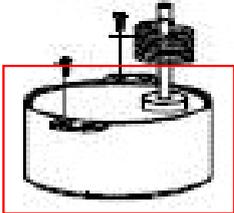
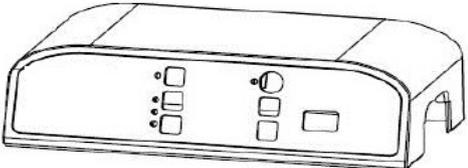
Item Number	Item Description
GSP494790	Mattress Pump - ULTIMATE 8
PTM638030	Mattress Pump - Aspire HYBRID COMFORT
PTM638032	Mattress Pump - Aspire HYBRID BARI

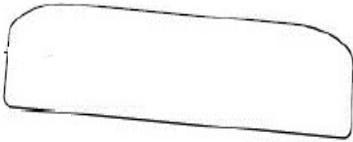
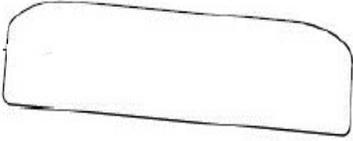
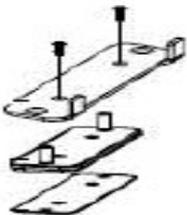
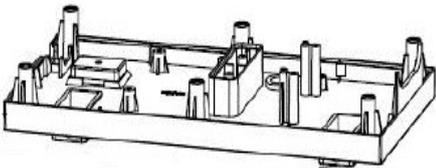
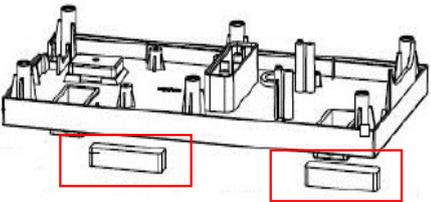
This manual is for internal use only

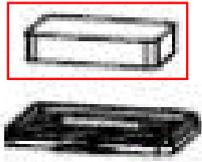
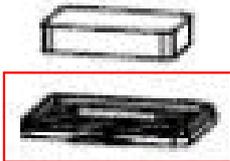
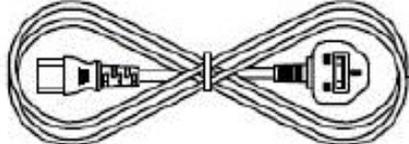
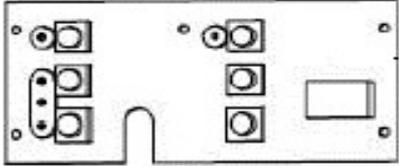
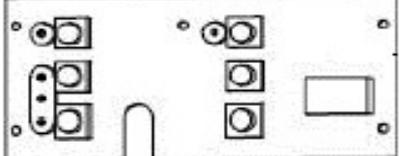
Exploded Diagram

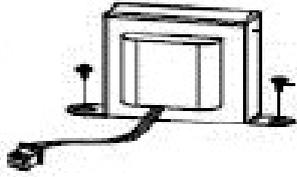
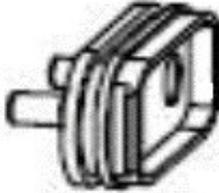
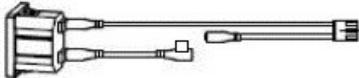
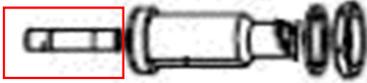


This manual is for internal use only

1 Spare Parts Listing				
Ref	Item Number	Item Description	Image	Type
1	GSP494640	Foam Noise Buffer - for Air Compressor - Aspire ULTIMATE 8 / Hybrid / Hybrid BARI Pump		Spare Part
2	GSP872804	Compressor - Aspire ULTIMATE 8 / Hybrid / Hybrid Acute / Hybrid BARI Pump - V2		Spare Part
4	GSP146610	Timer Switch w/Cable - Aspire ULTIMATE 8 / Hybrid / Hybrid BARI Pump		Spare Part
5	GSP494360	Air Valve - Aspire ULTIMATE 8 / Hybrid / Hybrid BARI Pump		Spare Part
6	GSP494150	Motor- Aspire - Ultimate 8 / Hybrid / Hybrid Acute / Hybrid BARI Pump - V1/V2		Spare Part
7	GSP494890	Top Case - Aspire -Ultimate 8 / Hybrid / Hybrid BARI Pump - V2		Spare Part

1 Spare Parts Listing				
8	GSP494650	Front Panel - Aspire - ULTIMATE 8 Pump - V2		Spare Part
8	GSP146612	Front Panel - Aspire - Hybrid Pump - V2		Spare Part
8	GSP146614	Front Panel - Hybrid BARI Pump - V2		Spare Part
9	GSP872802	Air Buffer - Aspire Ultimate 8 / Hybrid / Hybrid BARI Pump - V2		Spare Part
10	GSP494590	Bottom Case - Aspire - ULTIMATE 8 / Hybrid / Hybrid BARI Pump - V2		Spare Part
11	GSP872812	Rear Foam Pad - Aspire Ultimate 8 / Hybrid / Hybrid Acute / Hybrid BARI Pump - V2		Spare Part

1 Spare Parts Listing				
12	GSP146616	Filter - ULTIMATE 8 / Hybrid / Hybrid BARI Pump - V2		Spare Part
13	GSP872808	Pump Filter Cap - Aspire Ultimate 8 / Hybrid / Hybrid Acute / Hybrid BARI Pump - V2		Spare Part
14	GSP872810	Pump Bracket Assembly - Aspire Ultimate 8 / Hybrid / Hybrid Acute / Hybrid BARI Pump - V2		Spare Part
15	GSP494190	Main Power Cable - Aspire - ULTIMATE 8 / Hybrid / Hybrid BARI Pump - V1/V2		Spare Part
16	GSP494630	PCB - Aspire Ultimate 8 Pump - V2		Spare Part
16	GSP146618	PCB - Aspire Hybrid / Hybrid BARI Pump - V2		Spare Part

1 Spare Parts Listing				
17	GSP494155	Transformer - Aspire - Ultimate 8 / Hybrid / Hybrid BARI Pump - V1/V2		Spare Part
18	GSP494160	Internal Air Tubing (5.5mm Diameter) - Aspire - ULTIMATE 8 /Hybrid / Hybrid BARI Pump- V1/V2		Spare Part
19	GSP146620	Air Valve Assembly / Female Connector - ULTIMATE 8 /Hybrid / Hybrid BARI Pump- V2		Spare Part
20	GSP494165	Internal Air Tubing (7.2mm Diameter) - Aspire - Ultimate 8 / Hybrid / Hybrid BARI Pump - V1/V2		Spare Part
21	GSP494880	Power Socket - w/Terminal Wires - Aspire - Ultimate 8 / Hybrid / Hybrid BARI Pump - V2		Spare Part
22	GSP872806	Fuse - Aspire - Ultimate 8 / Hybrid / Hybrid Acute / Hybrid BARI Pump - V2		Spare Part

1 Spare Parts Listing				
23	GSP146622	Sticker - Weight Guide - Aspire - Ultimate 8 Pump - V2		Spare Part
23	GSP146624	Sticker - Troubleshooting - Aspire - Ultimate 8 / Hybrid / Hybrid BARI Pump - V2		Spare Part
23	GSP146626	Sticker - Aspire Logo - Aspire - Ultimate 8 / Hybrid / Hybrid BARI Pump - V2		Spare Part
23	GSP146628	Sticker - Safety Label - Aspire - Ultimate 8 Pump - V2		Spare Part
23	GSP146630	Sticker - Safety Lable - Aspire - Hybrid Pump - V2		Spare Part
23	GSP146632	Sticker - Safety Lable - Aspire - Hybrid BARI Pump - V2		Spare Part

1 Spare Parts Listing				
23	GSP146634	Sticker - Pump Compatibility - Aspire - Hybrid S/KS Pump - V2		Spare Part
23	GSP146636	Sticker - Pump Compatibility - Aspire - Hybrid BAIRI Pump - V2		Spare Part