

WHILL Model C2 Exploded view

Rev. 3

Item Code	Item Description
PWS655400	Power Wheelchair - Whill - Model C2 - (Black Armrests) - 450mm - 135kg
PWS655410	Power Wheelchair - Whill - Model C2 - (Black Armrests) - 500mm - 135kg

01. Revision history

Revision	Change details	Date
2.00	First release for global	May 18 th , 2021
2.01	Added HMI kit number / 04. Controller pad	June 8 th , 2021
3.00	Major update	

02. Disassembly of three major components

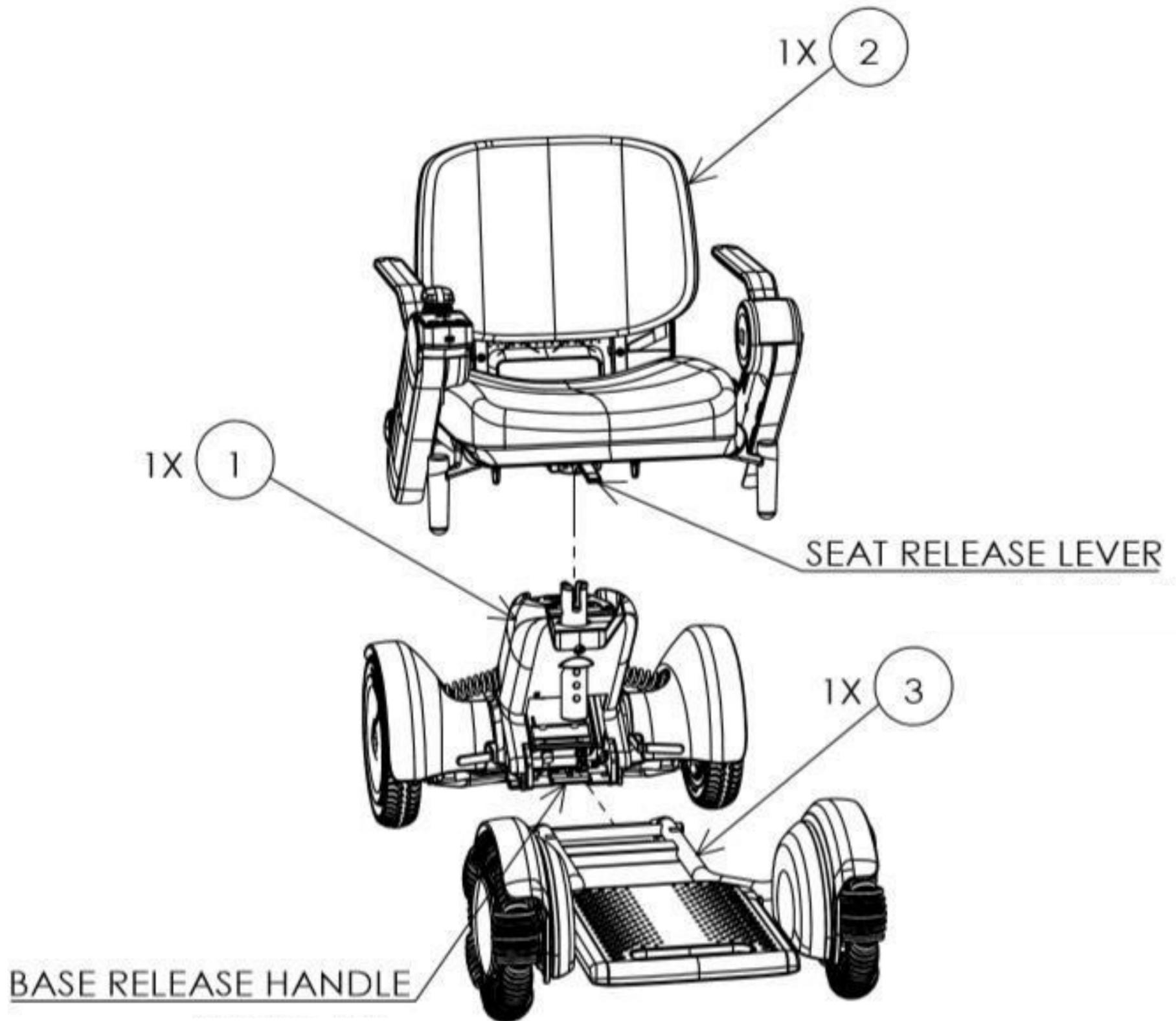


Figure	Part number	GP Code	Registered name	Item name	Notes
1	21-00019-0	MSP491832	Main_Body_Assy	Rear drive base	
2	-		Seat_Assy_HMI_Assy_16in	Seat assembly	
3	21-00003-0	MSP491708	Drive_Base_Assy	Front drive base	

03. Arm covers, control pad related parts

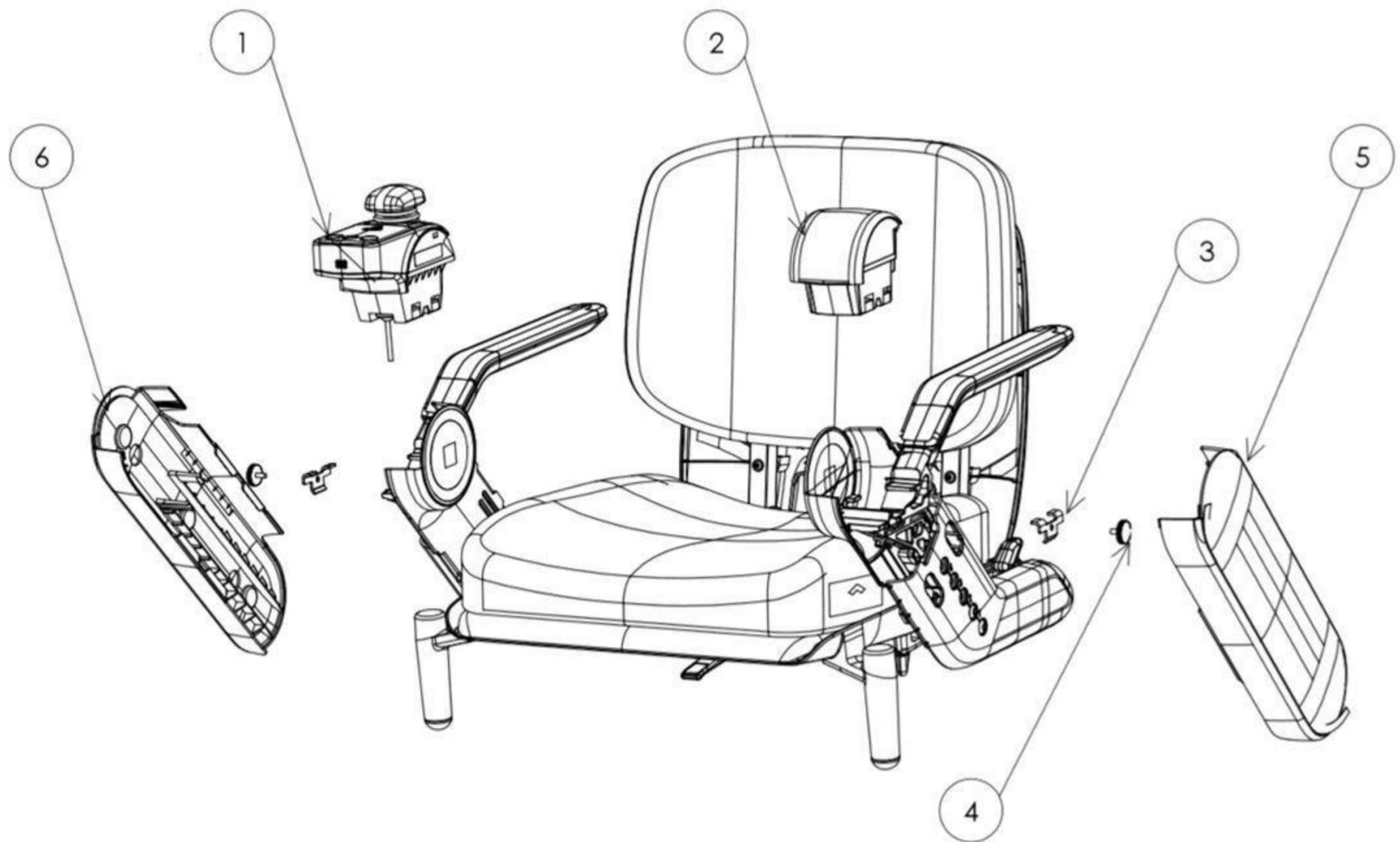


Figure	Part number	GP Code	Item name	Q'ty	Notes
1	21-00005-0		Control pad	1	Control pad for Model C2
2	21-00021-0	MSP492150	Hand rest	1	Hand rest
3	31-00299-0		Holder plate	1	LH/RH common
4	41-10137-0		Grip screw (M4-L10)	1	LH/RH common
5	22-00013-1	WCA870580	Arm cover - White (Left)	1	
6	22-00013-2	WCA870590	Arm cover - White (Right)	1	
5	22-00013-3	WCA870460	Arm cover - Black (Left)	1	
6	22-00013-4	WCA870470	Arm cover - Black (Right)	1	
5	22-00013-5	WCA870480	Arm cover - Gold (Left)	1	
6	22-00013-6	WCA870490	Arm cover - Gold (Right)	1	
5	22-00013-7	WCA870560	Arm cover - Pink (Left)	1	
6	22-00013-8	WCA870570	Arm cover - Pink (Right)	1	
5	22-00013-9		Arm cover - Blue (Left)	1	
6	22-00013-A		Arm cover - Blue (Right)	1	
5	22-00013-B	WCA870600	Arm cover - Red(Left)	1	
6	22-00013-C	WCA870610	Arm cover - Red (Right)	1	
5	22-00013-D		Arm cover - Light green (Left)	1	
6	22-00013-E		Arm cover - Light green (Right)	1	
5	22-00013-F	WCA870520	Arm cover - Light blue (Left)	1	
6	22-00013-G	WCA870530	Arm cover - Light blue (Right)	1	
5	22-00013-H	WCA870500	Arm cover - Gray (Left)	1	
6	22-00013-J	WCA870510	Arm cover - Gray (Right)	1	
5	22-00013-9	WCA870540	Arm cover - Navy (Left)	1	
6	22-00013-A	WCA870550	Arm cover - Navy (Right)	1	

04. Controller pad

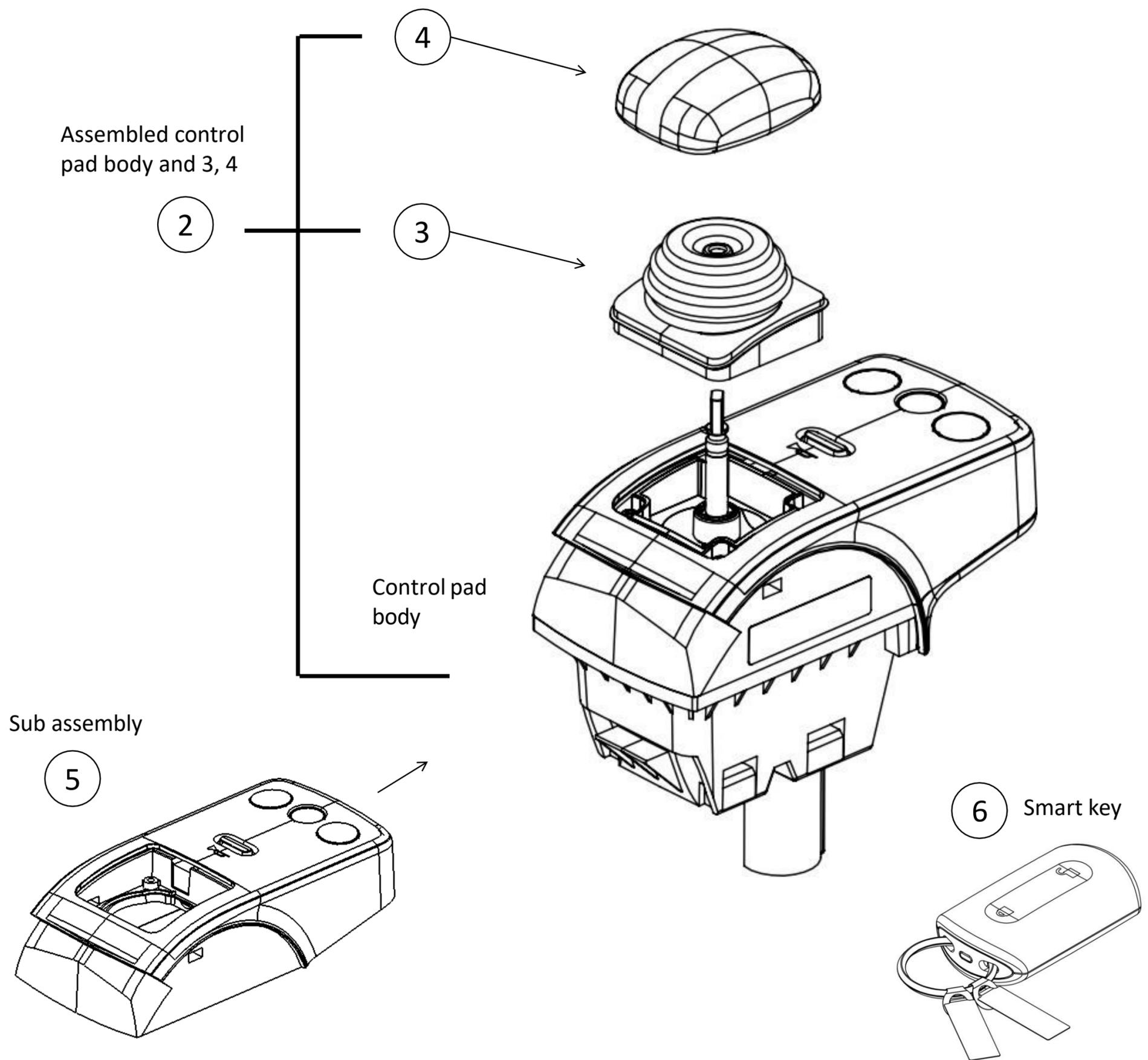


Figure	Part number	GP Code	Item name	Q'ty	Notes
1	24-10013-0		Control pad kit for JP[C2]	1	Including 2(HMI_Assy)+JP voice data + pass label
1	24-10013-1	MSP491780	Control pad kit for EN[C2]	1	Including 2(HMI_Assy)+EN voice data + pass label
1	24-10013-2		Control pad kit for CN[C2]	1	Including 2(HMI_Assy)+CN voice data + pass label
1	24-10013-3		Control pad kit for EU[C2]	1	Including 2(HMI_Assy)+EU voice data + pass label
1	24-10013-4		Control pad + Smart key kit for JP[C2]	1	Including 2(HMI_Assy),6(Smart key)+JP voice data + pass label
1	24-10013-5	MSP491790	Control pad + Smart key kit for EN[C2]	1	Including 2(HMI_Assy),6(Smart key)+EN voice data + pass label
1	24-10013-6		Control pad + Smart key kit for CN[C2]	1	Including 2(HMI_Assy),6(Smart key)+CN voice data + pass label
1	24-10013-7		Control pad + Smart key kit for EU[C2]	1	Including 2(HMI_Assy),6(Smart key)+EU voice data + pass label
2	21-00005-0		Control pad	1	Assembled controller body and 3, 4
3	31-00190-0		Joystick boot [C2/F]	1	
4	22-00044-0		Joystick knob assy [C2/F]	1	
5	22-00048-0		HMI Controller upper case [C2 / F]	1	Sub-assembly of control pad

05. Arm rest parts


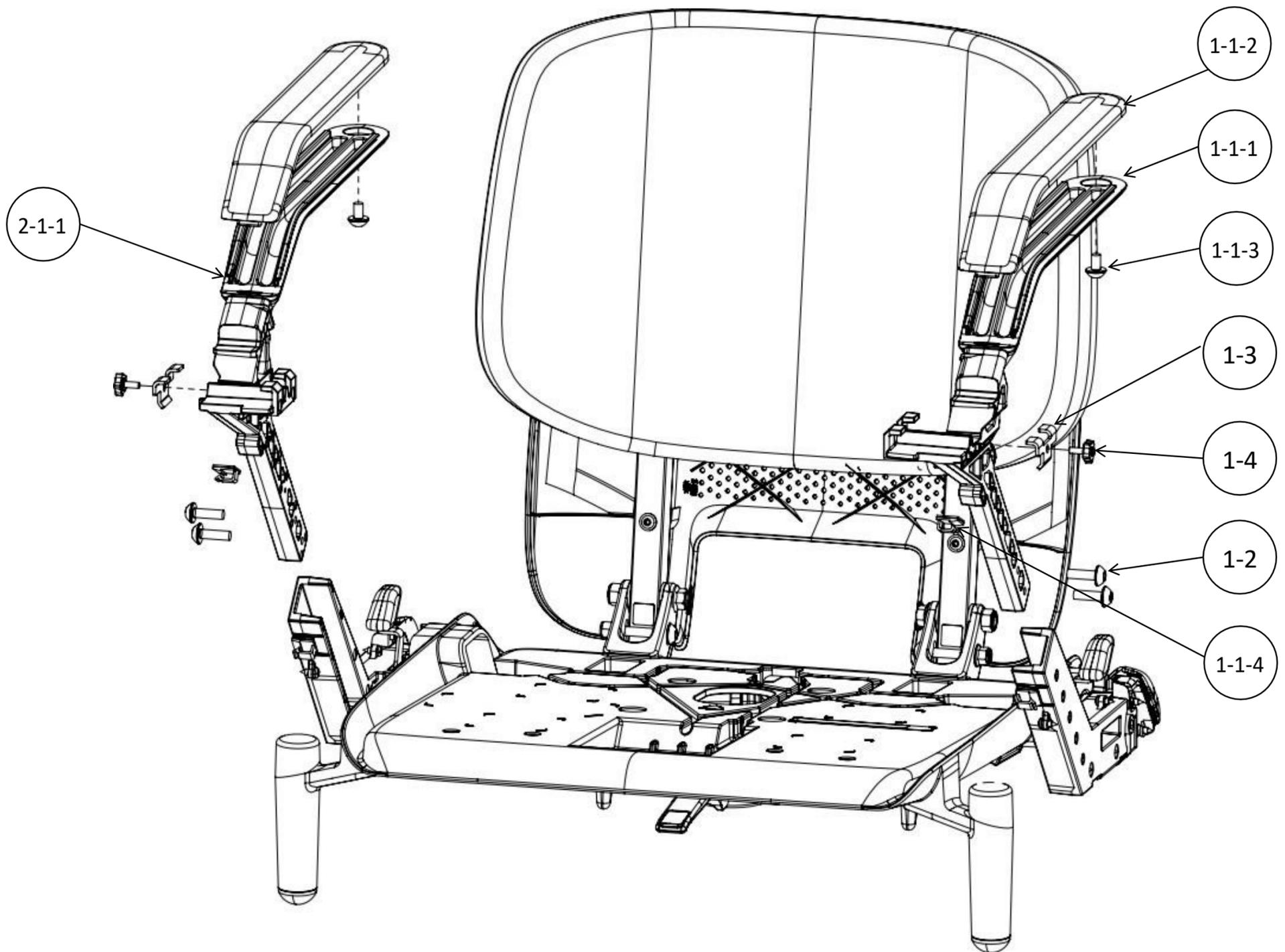


Figure	Part number	GP Code	Item name	Q'ty	Notes
1	24-10020-0	MSP491550	Arm rest frame kit (Left) [C2]	1	Including 1-1 to 1-4 for each Q'ty
1-1	21-00034-0		Arm rest frame assy (Left) [C2]	1	Including 1-1-1 to 1-1-4 for each Q'ty
1-1-1	31-00293-0		Arm rest frame (Left) [C2]	1	
1-1-2	31-00290-0		Arm rest cover [C2]	1	
1-1-3	31-00347-0		WHILL special bolt M6_L10	1	
1-1-4	41-00004-0		Cable clamp	1	
1-2	31-00347-1		WHILL special bolt M6_L42	2	LH/RH common
1-3	31-00299-0		Holder Plate [C2 / F]	1	LH/RH common
1-4	41-10137-0		Grip Screw M4_L10	1	LH/RH common
2	24-10020-1	MSP491560	Arm rest frame kit (Right) [C2]	1	Including 2-1, 1-2 to 1-4 for each Q'ty
2-1	21-00034-1		Arm rest frame assembly (Right) [C2]	1	Including 2-1-1, 1-1-2 to 1-1-4 for each Q'ty
2-1-1	31-00293-1		Arm rest frame (Right) [C2]	1	
3	24-10021-0	MSP491540	Arm rest cover kit [C2]	1	LH/RH common. Including 1-1-2, 1-1-3 for each Q'ty

06. Arm covers

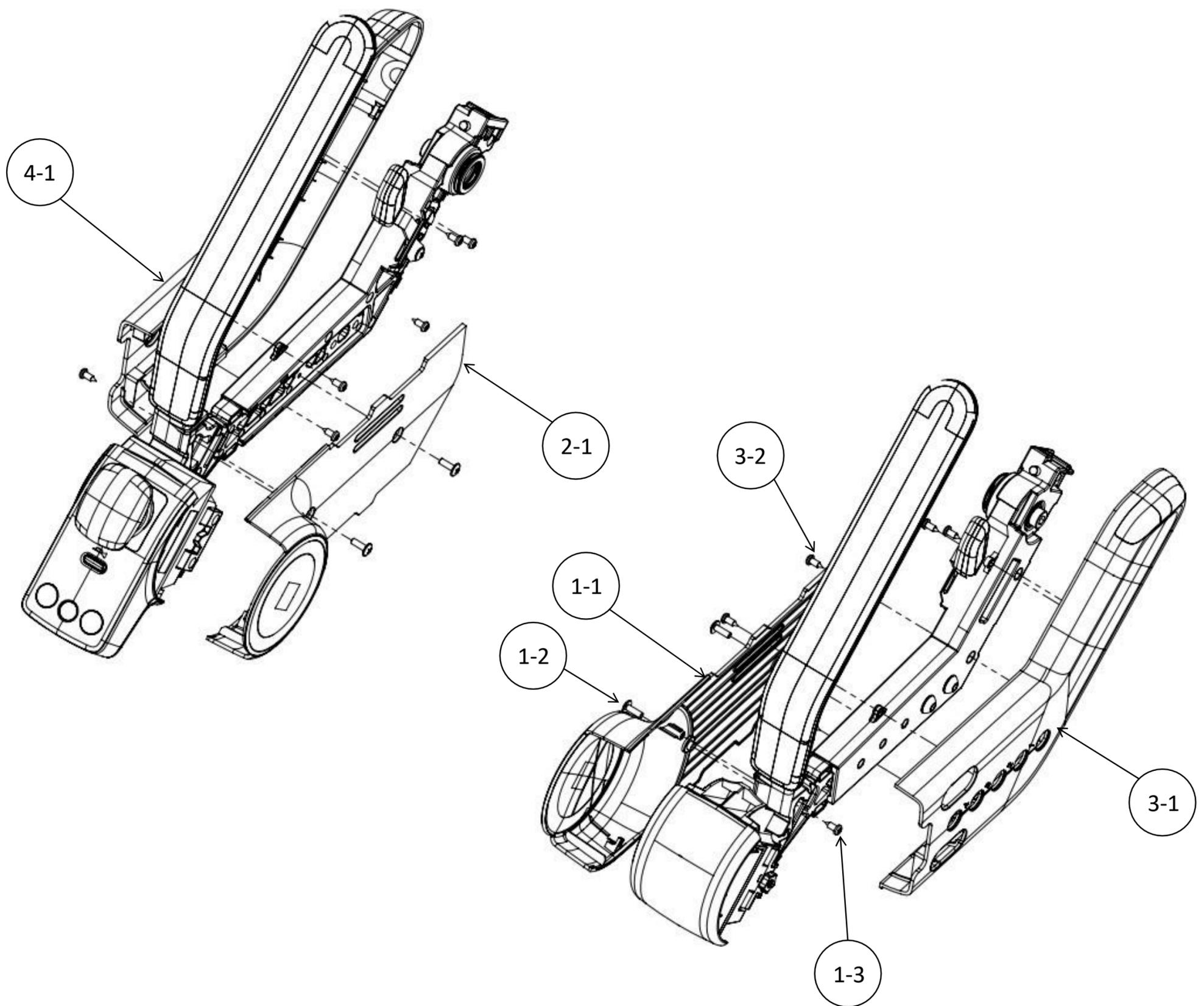

Top view of Arm

Figure	Part number	GP Code	Item name	Q'ty	Notes
1	24-10022-0	MSP491520	Arm inner cover kit (Left) [C2]	1	Including 1-1 to 1-3 for each Q'ty
1-1	31-00269-0		Arm cover inner (Left) [C2]	1	
1-2	41-00087-0		Truss bolt M4_L12	2	LH/RH common
1-3	41-00066-0		Pan Tapping screw type4_L10	1	LH/RH common
2	24-10022-1	MSP491530	Arm inner cover kit (Right) [C2]	1	Including 2-1, 1-2, 1-3 for each Q'ty
2-1	31-00269-1		Arm cover inner (Right) [C2]	1	
3	24-10023-0	MSP491570	Arm Main inside cover kit (Left) [C2]	1	Including 3-1, 3-2 for each Q'ty
3-1	21-00039-0		Arm Main inside cover (Left) [C2]	1	
3-2	41-00066-0		Pan Tapping screw type4_L10	5	LH/RH common
4	24-10023-1	MSP491580	Arm Main inside cover kit (Left) [C2]	1	Including 4-1, 3-2 for each Q'ty
4-1	21-00039-1		Arm Main inside cover (Left) [C2]	1	

07. Arm frame parts LH

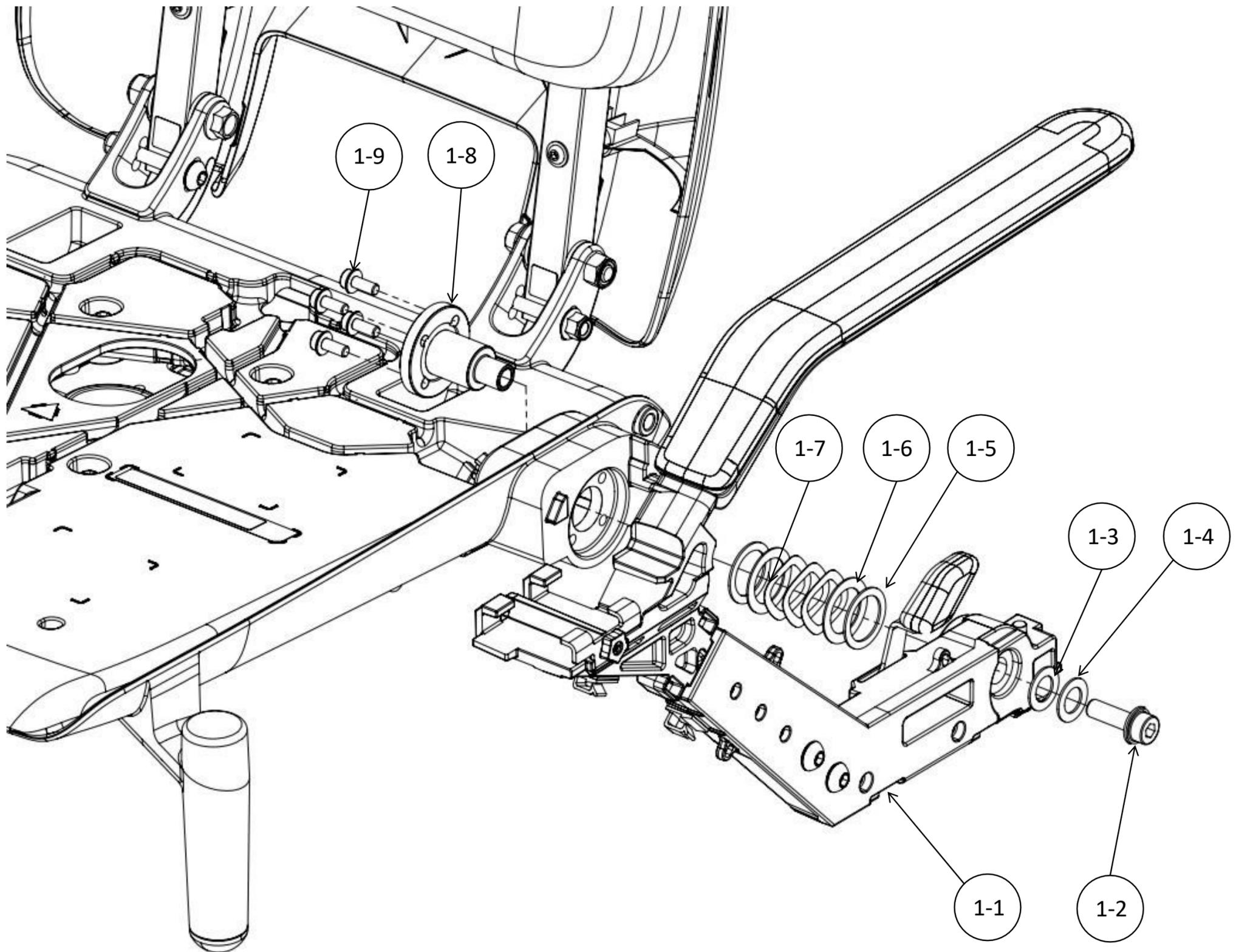


Figure	Part number	GP Code	Item name	Q'ty	Notes
1	24-10024-0	MSP491590	Arm frame assembly kit (Left)	1	Including 1-1 to 1-9 for each Q'ty
1-1	21-00031-0		Arm frame assembly (Left)	1	
1-2	41-00051-0		Screw (M8-L25)	1	LH/RH common
1-3	41-00061-0		POM Washer 20_21_1	1	LH/RH common
1-4	41-00062-0		SUS Washer 20_12_1	1	LH/RH common
1-5	41-00064-0		POM Washer 28_21_1	2	LH/RH common
1-6	41-00063-0		SUS Washer 30_21_0.8	2	LH/RH common
1-7	31-00341-1		Wave Washer WW20	3	LH/RH common
1-8	31-10467-0		Hinge shaft collar	1	LH/RH common
1-9	41-00081-0		Flange Cap M5_L12	4	LH/RH common

08. Arm frame parts RH

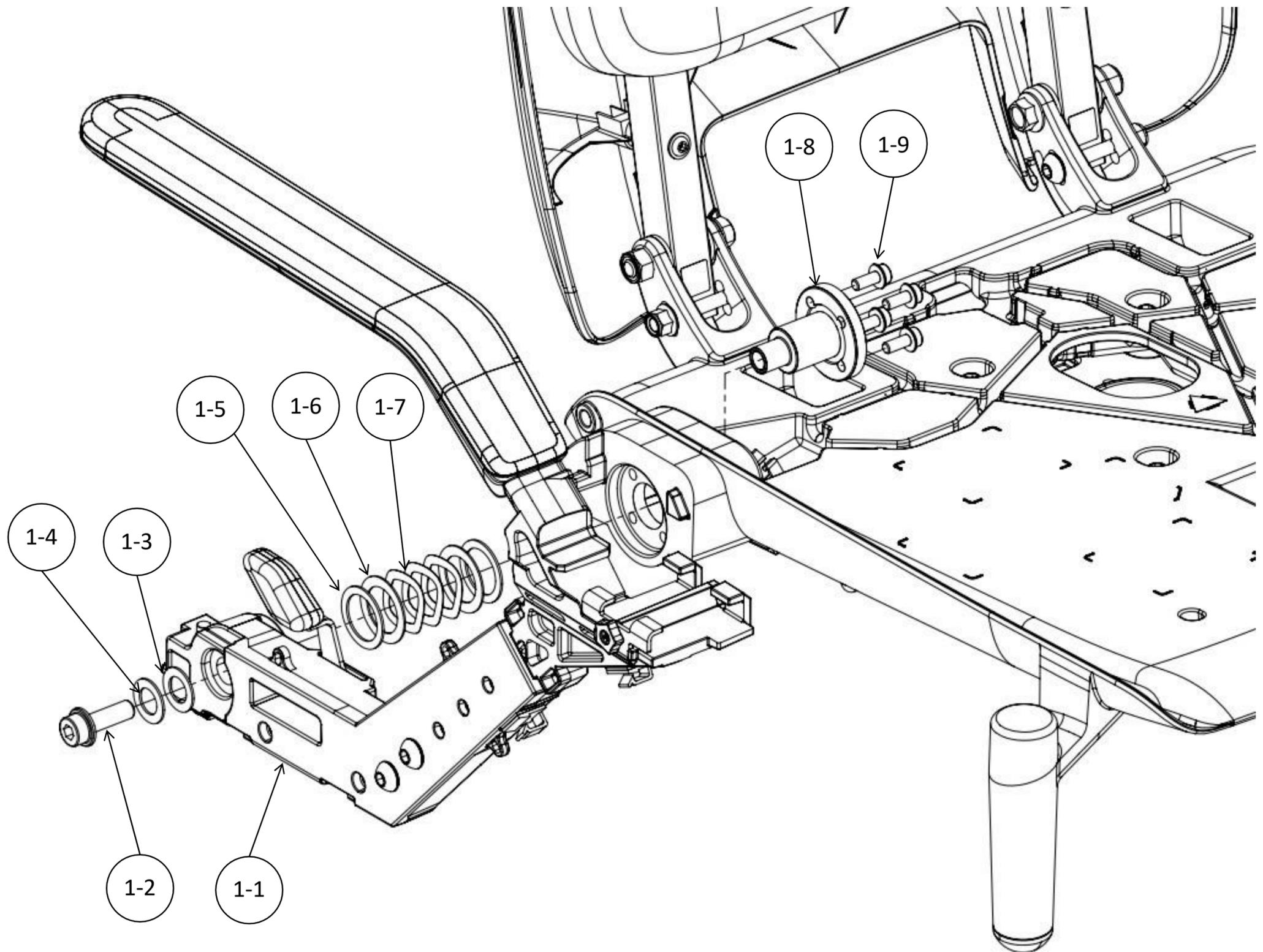


Figure	Part number	GP Code	Item name	Q'ty	Notes
1	24-10024-1	MSP491600	Arm frame assembly kit (Right)	1	Including 1-1 to 1-9 for each Q'ty
1-1	21-00031-1		Arm frame assembly (Right)	1	
1-2	41-00051-0		Screw (M8-L25)	1	LH/RH common
1-3	41-00061-0		POM Washer 20_21_1	1	LH/RH common
1-4	41-00062-0		SUS Washer 20_12_1	1	LH/RH common
1-5	41-00064-0		POM Washer 28_21_1	2	LH/RH common
1-6	41-00063-0		SUS Washer 30_21_0.8	2	LH/RH common
1-7	31-00341-1		Wave Washer WW20	3	LH/RH common
1-8	31-10467-0		Hinge shaft collar	1	LH/RH common
1-9	41-00081-0		Flange Cap M5_L12	4	LH/RH common

09. Arm release lever

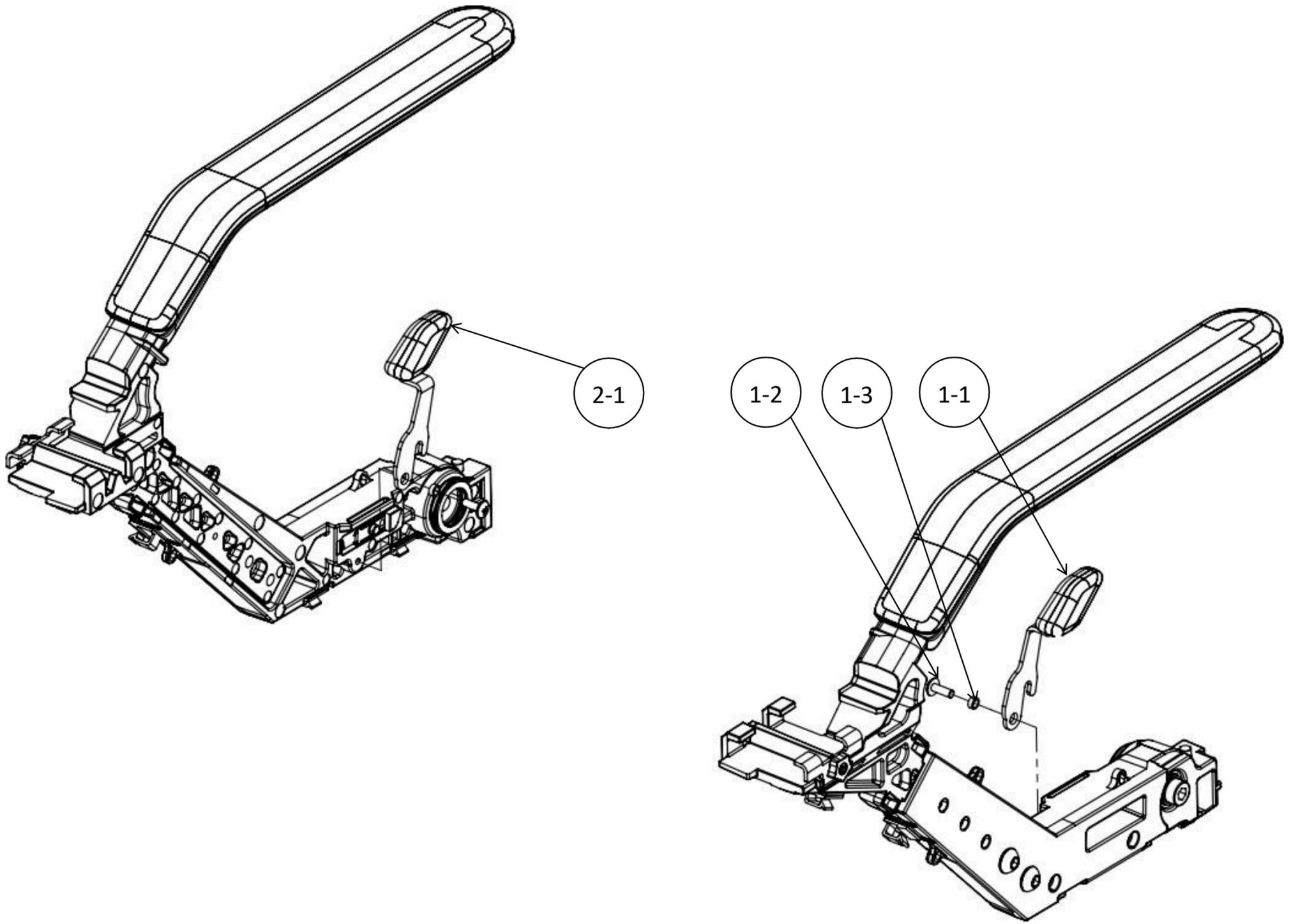


Figure	Part number	GP Code	Item name	Q'ty	Notes
1	24-10052-0	MSP491500	Arm release lever kit (Left)	1	Including 1-1 to 1-3 for each Q'ty
1-1	21-00007-0		Arm release lever assembly (Left)	1	
1-2	41-00087-1		Truss M5 L12 loctite	1	LH/RH common
1-3	31-00307-0		Lever collar	1	LH/RH common
2	24-10052-1	MSP491510	Arm release lever kit (Right)	1	Including 2-1, 1-2, 1-3 for each Q'ty
2-1	21-00007-1		Arm release lever assembly (Right)	1	

10. Back support

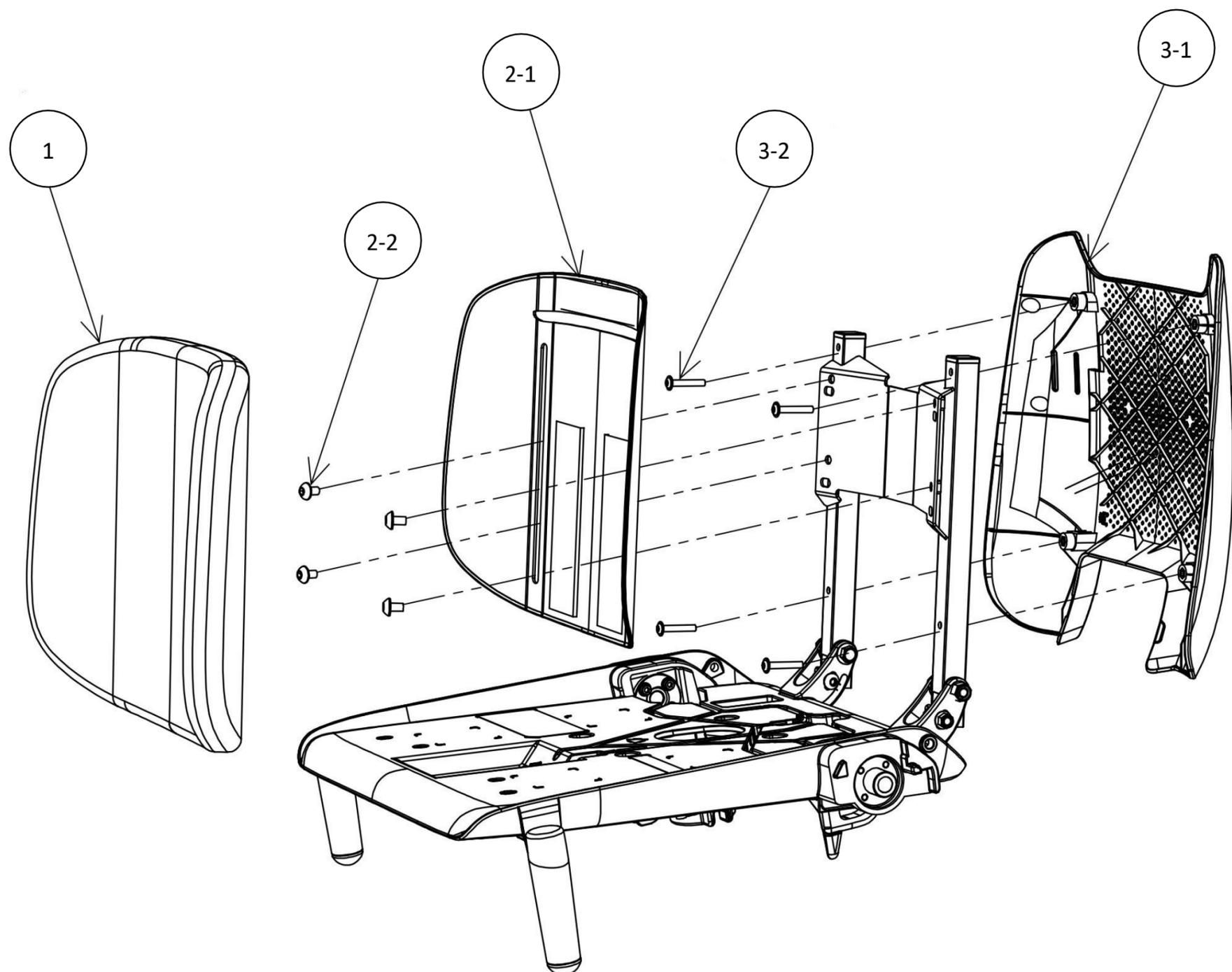


Figure	Part number	GP Code	Item name	Q'ty	Notes
1	22-00079-0		Back support cushion (normal)	1	Back support cushion (without frame)
1	22-00079-1	MSP491630	Back support cushion (flame retardancy)	1	Back support cushion (without frame) Regional regulation : Flame retardance
2	24-10028-0	MSP491640	Back support cushion frame kit (without cushion)	1	Including 2-1, 2-2 for each Q'ty
2-1	22-00047-0		Back support cushion frame	1	Back Support Cushion Frame(without cushion)
2-2	31-00347-0		Screw WHILL(M6-L10)	4	Screws to fix 2-1 to the frame (x4)
3	24-10026-0	MSP491610	Back Support Cover Kit	1	Including 3-1, 3-2 for each Q'ty
3-1	22-00037-0		Back support cover	1	
3-2	41-00038-1		Flange button bolt M5 L30	4	

11. Back support frame

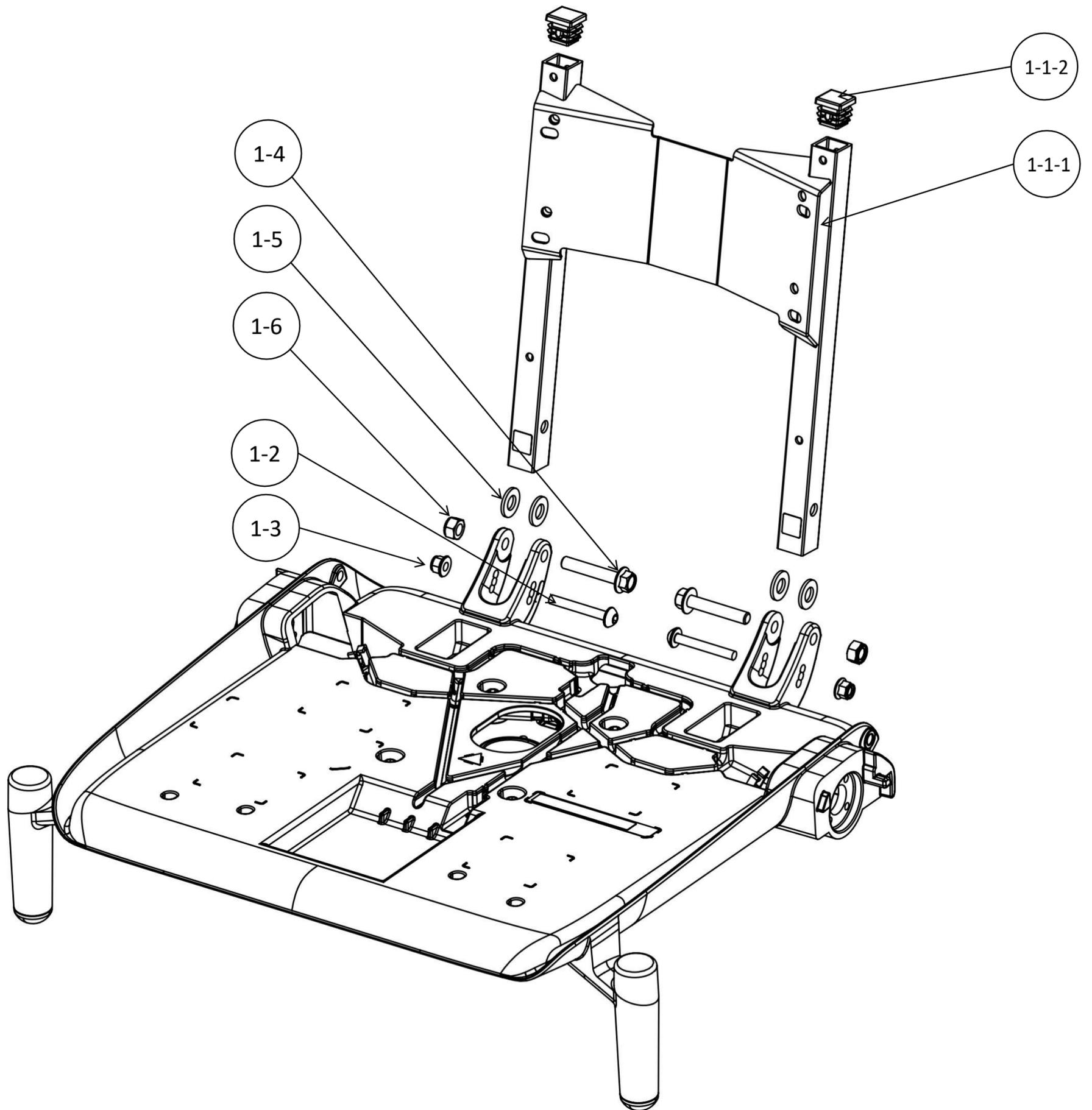


Figure	Part number	GP Code	Item name	Q'ty	Notes
1	24-10027-0	MSP491612	Back support frame kit	1	Including 1-1 to 1-6 for each Q'ty
1-1	21-00033-0		Back support frame cap label assembly	1	Including 1-1-1, 1-1-2 for each Q'ty and label
1-1-1	22-00038-0		Back support frame assembly	1	
1-1-2	31-00339-0		Pipe end cap square	2	
1-2	31-00347-3		Special bolt M6_L45	2	
1-3	41-00093-0		Flange Nylon nut M6	2	
1-4	41-00090-0		Flange Hex Bolt M8_L45	2	
1-5	41-00086-0		POM Washer 20_10.2_2	4	
1-6	41-00050-0		Nylon nut M8	2	

12. Seat frame

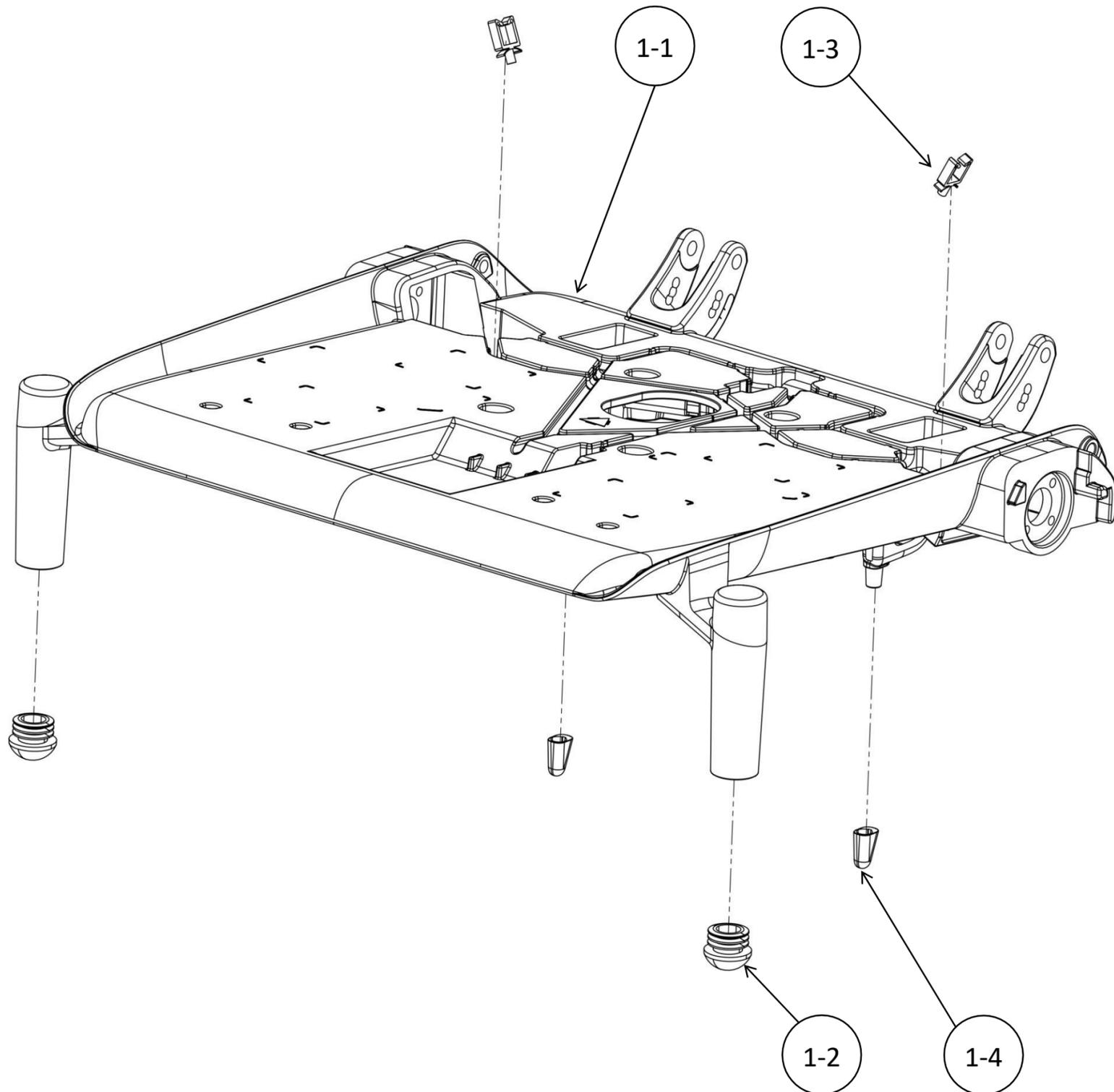


Figure	Part number	GP Code	Item name	Q'ty	Notes
1	24-10010-0		Seat frame 16in service parts (JP)	1	Including 1-1(16in), 1-2 to 1-4 for each Q'ty and JP label
1	24-10010-3		Seat frame 16in service parts	1	Including 1-1(16in), 1-2 to 1-4 for each Q'ty and label
1	24-10010-4	MSP492065	Seat frame 18in service parts	1	Including 1-1(18in), 1-2 to 1-4 for each Q'ty and label
1	24-10010-5	MSP492070	Seat frame 20in service parts	1	Including 1-1(20in), 1-2 to 1-4 for each Q'ty and label
1-1	31-00241-0		Seat_Frame_16in	1	
1-1	31-00200-0		Seat_Frame_18in	1	
1-1	31-00200-1		Seat_Frame_20in	1	
1-2	31-00338-0		Pipe_End_Cap_Sphere	2	
1-3	41-00084-0		Connector clasper	2	
1-4	22-00060-0		D-Ring end cover	2	

13. Seat mount and cable

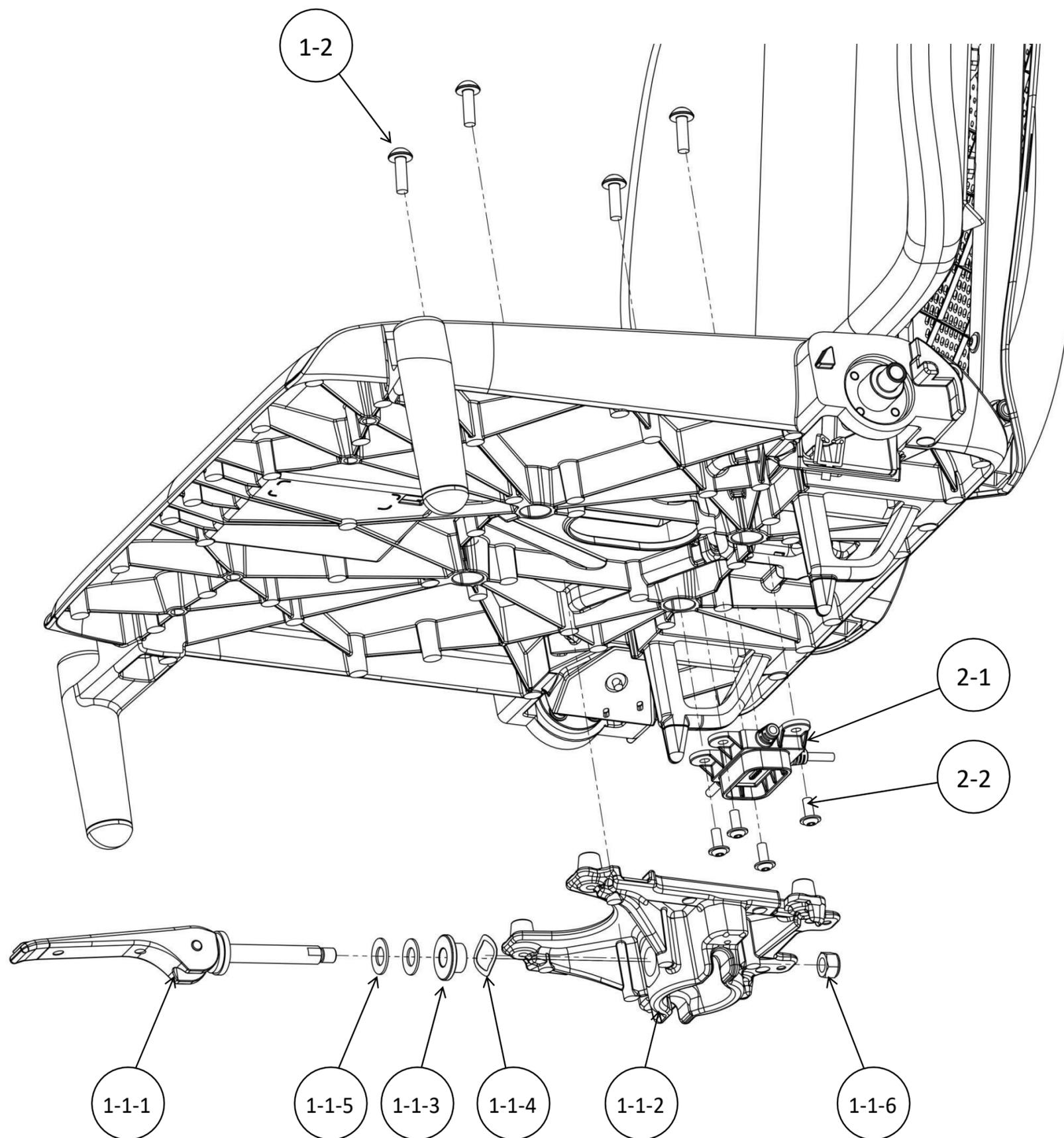


Figure	Part number	GP Code	Item name	Q'ty	Notes
1	24-10054-0	MSP492075	Seat mount kit	1	Including 1-1, 1-2 for each Q'ty
1-1	22-00068-0		Seat mount assembly	1	Including 1-1-1 to 1-1-6 for each Q'ty and label
1-1-1	22-00071-0		Seat release lever assembly	1	
1-1-2	31-00280-0		Seat frame mounting	1	
1-1-3	31-00340-0		Seat release lever lock block	1	
1-1-4	31-00341-0		Wave washer WW16	1	
1-1-5	31-00342-0		Camlever dish spring	2	
1-1-6	41-00050-0		Nylon nut M8	1	
1-2	31-00347-1		Flange cap bolt M6 L20	4	
2	24-10025-0	MSP492060	SB cable kit	1	Including 2-1, 2-2 for each Q'ty and zip tie
2-1	35-10033-0		SB cable	1	
2-2	41-00038-0		Screw (L5-L12)	4	

14. Seat cushion


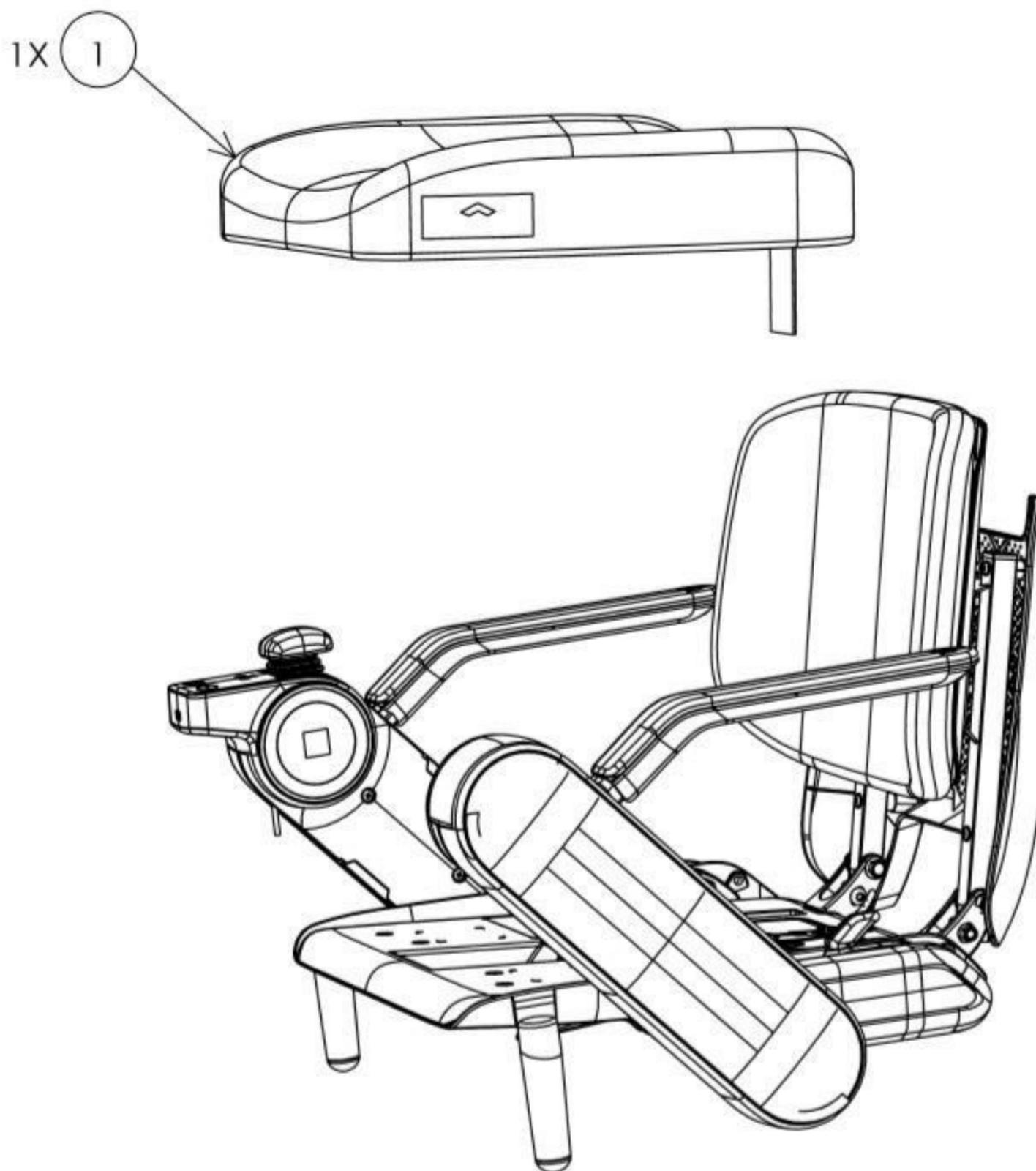


Figure	Part number	GP Code	Item name	Q'ty	Notes
1	22-00065-0		Seat cushion (16" x 16") (normal)	1	WHILL standard cushion 16" x 16" (40cm width x 40cm depth)
1	22-00078-0		Seat cushion (18" x 18") (normal)	1	WHILL standard cushion 18" x 18" (45cm width x 45cm depth)
1	22-00065-1		Seat cushion (16" x 16") (flame retardancy)	1	WHILL flame 15etardancy cushion 16" x 16" (40cm width x 40cm depth)
1	22-00078-1	MSP491650	Seat cushion (18" x 18") (flame retardancy)	1	WHILL flame 15etardancy 18" x 18" (45cm width x 45cm depth)
1	22-10092-0		Seat cushion (20" x 20") (flame retardancy)	1	WHILL flame 15etardancy 20" x 20" (50cm width x 50cm depth)

15. Rear wheel

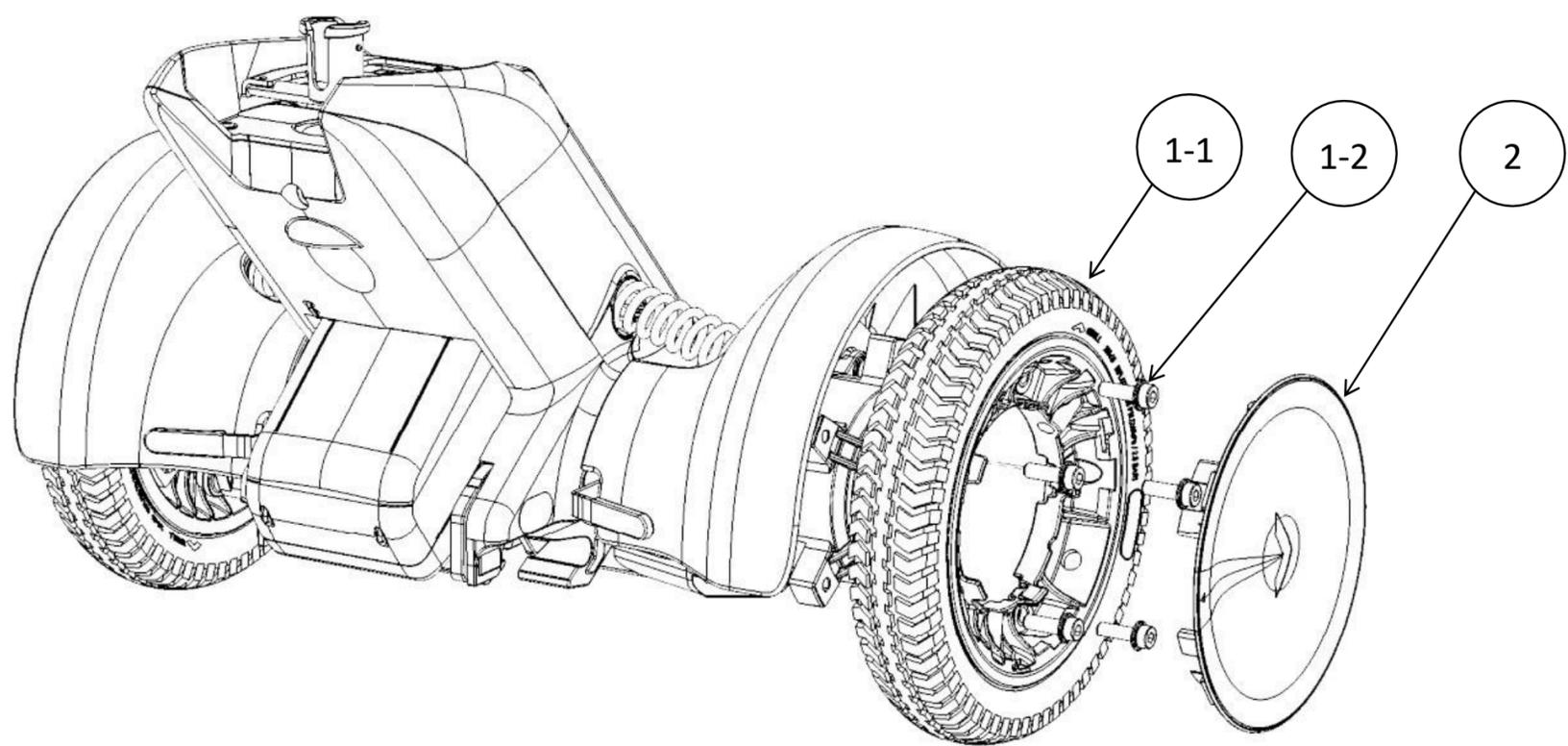

LH and RH are the same

Figure	Part number	GP Code	Item name	Q'ty	Notes
1	24-10029-0	MSP492040	Rear wheel kit [C2]	1	Including 1-1, 1-2 for each Q'ty, LH/RH common
1-1	22-00039-0		Rear wheel	1	LH/RH common
1-2	41-00051-0		Screw (M8-L25)	5	LH/RH common
2	22-00063-0	MSP492030	Rear wheel cap	1	LH/RH common

16. Motor

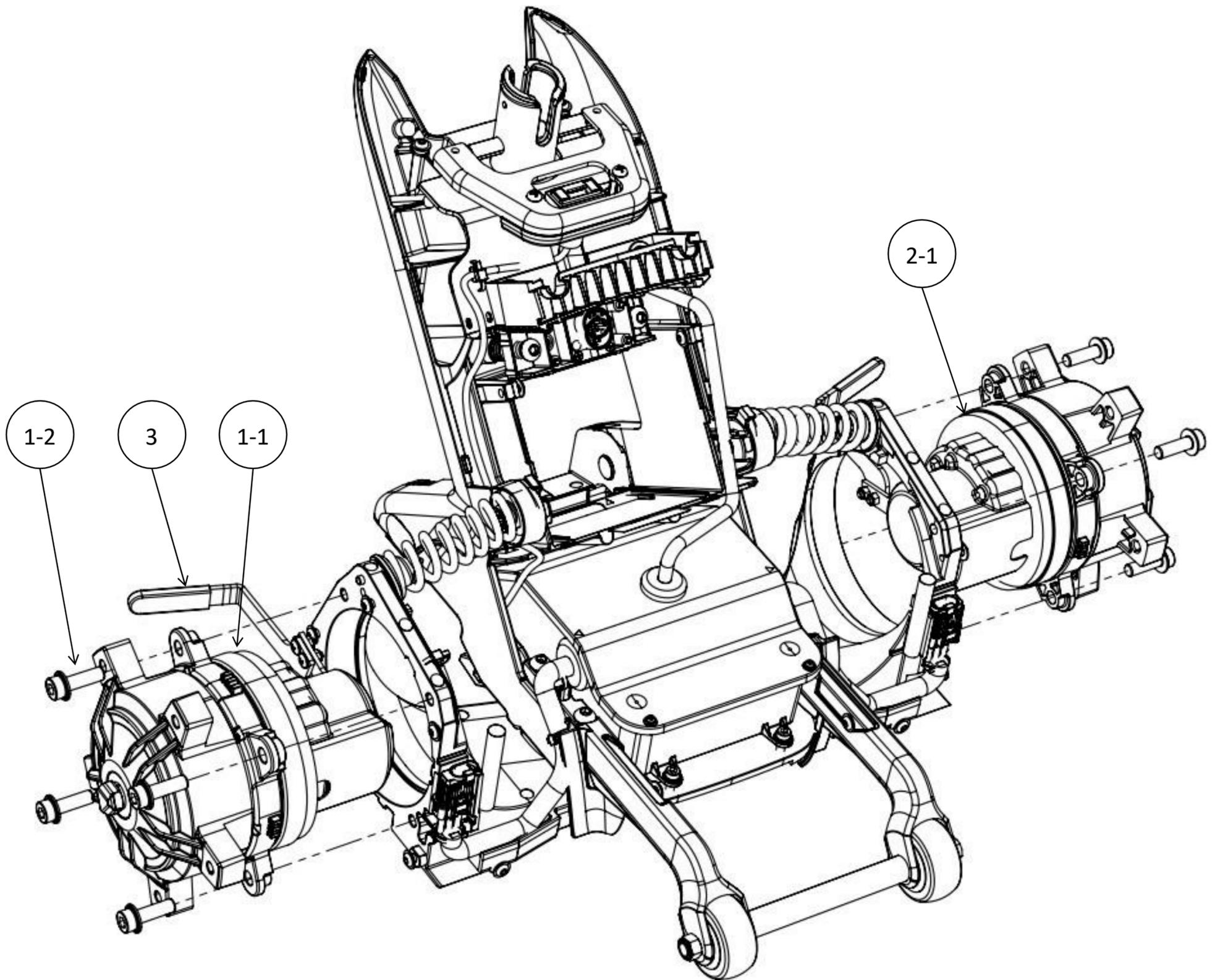


Figure	Part number	GP Code	Item name	Q'ty	Notes
1	24-10044-0	MSP491814	Motor kit (Left) [C2]	1	Including 1-1, 1-2 for each Q'ty
1-1	21-00008-0		Motor brake assembly (Left) [C2]	1	
1-2	41-00051-0		Flange cap bolt M8 L25	4	LH/RH common
2	24-10044-1	MSP491816	Motor kit (Right) [C2]	1	Including 2-1, 1-2 for each Q'ty
2-1	21-00008-1		Motor brake assembly (Right) [C2]	1	
3	31-00301-0		Brake lever cap	1	LH/RH common

17. Rear fender cover and chassis frame cover

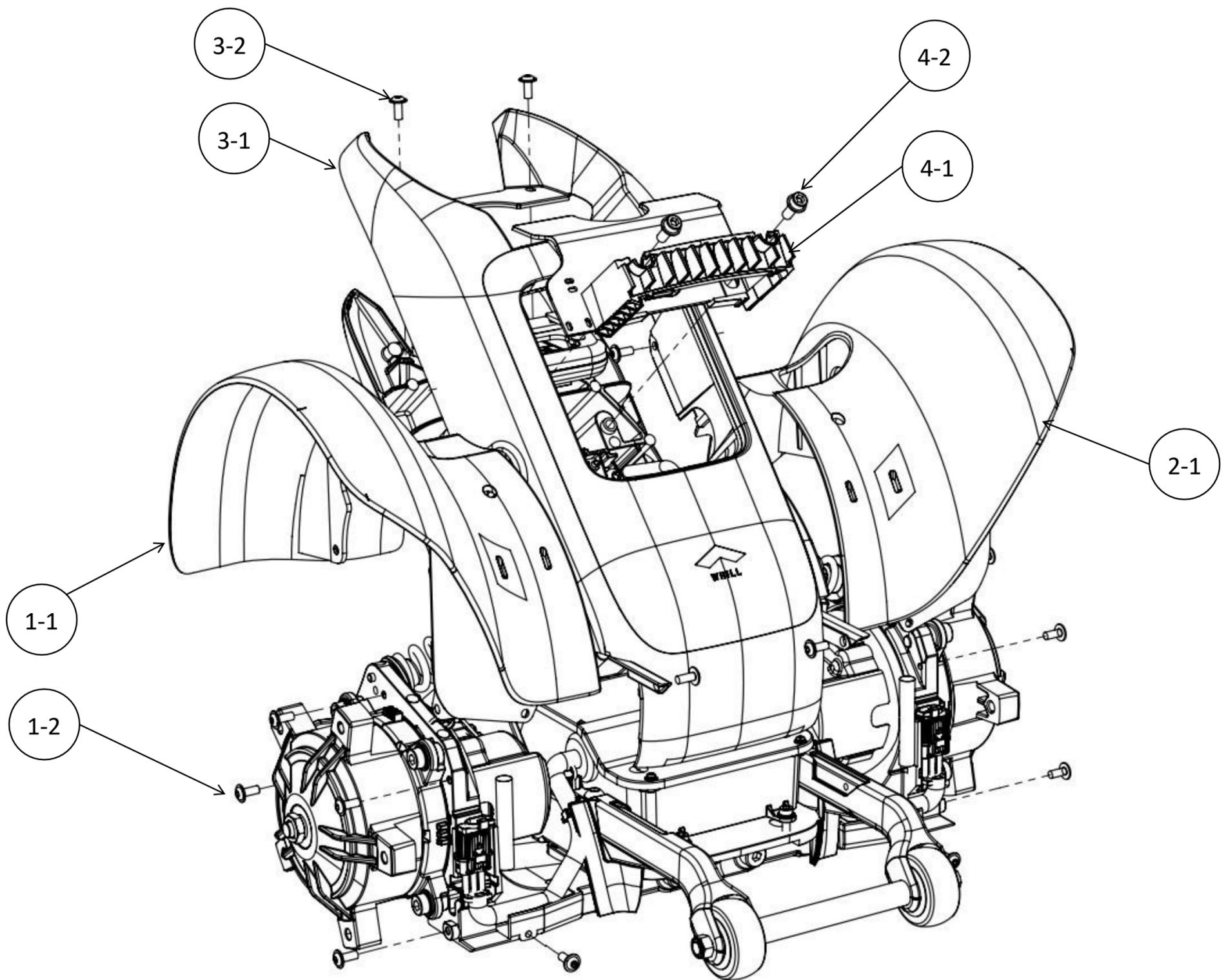


Figure	Part number	GP Code	Item name	Q'ty	Notes
1	24-10037-0	MSP491800	Rear fender kit (Left)	1	Including 1-1, 1-2 for each Q'ty
1-1	31-00197-0		Rear fender (Left)	1	
1-2	41-00038-0		Screw (L5-L12)	5	LH/RH common
2	24-10037-1	MSP491810	Rear fender kit (Right)	1	Including 2-1, 1-2 for each Q'ty
2-1	31-00197-1		Rear fender (Right)	1	
3	24-10038-0	MSP491820	chassis frame cover kit	1	Including 3-1, 3-2 for each Q'ty
3-1	31-00003-0		Chassis frame cover	1	
3-2	41-00038-0		Screw (L5-L12)	6	
4	24-10042-0	MSP491822	Frame_Handle_Kit	1	Including 4-1, 4-2 for each Q'ty
4-1	31-00132-0		Chassis frame handle	1	
4-2	41-00089-0		Flange cap bolt M6 L15	2	

18. Anti-tip lever and roller

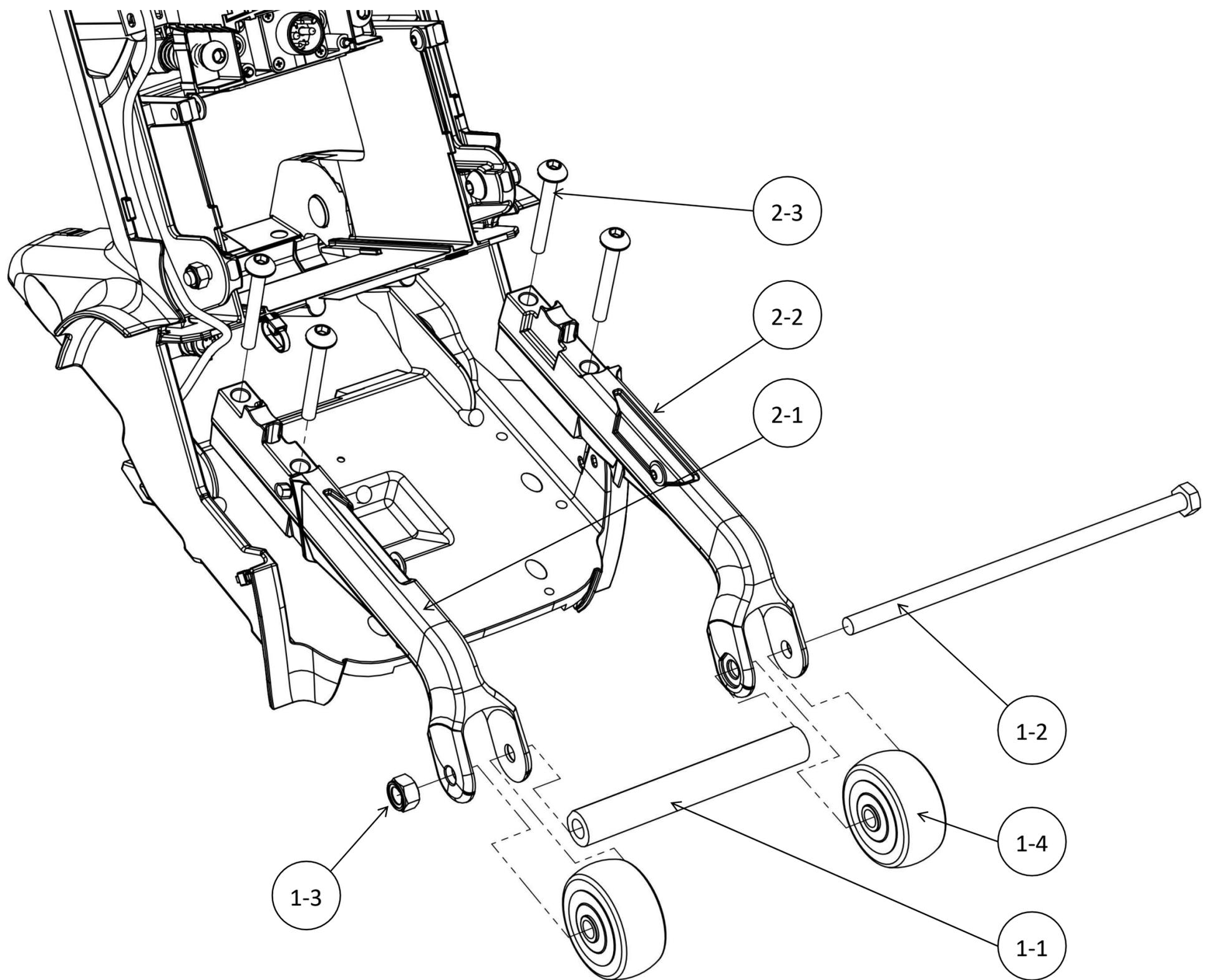


Figure	Part number	GP Code	Item name	Q'ty	Notes
1	24-10039-0	MSP491794	Anti-tip roller kit [C2]	1	Including 1-1 to 1-4 for each Q'ty
1-1	31-00275-0		Anti-tip bar	1	
1-2	41-00010-0		Hex m8 bolt M8 L190	1	
1-3	41-00050-0		Nylon nut M8	1	
1-4	31-00368-0		Anti-tip roller	2	
2	24-10040-0	MSP491792	Anti-tip lever kit	1	Including 2-1 to 2-3 for each Q'ty and zip tie
2-1	31-00274-0	MSP491860	Anti-tip lever LH	1	
2-2	31-00274-1	MSP491862	Anti-tip lever RH	1	
2-3	31-00347-3		Special bolt M6 L45	4	

19. Rear suspension spring

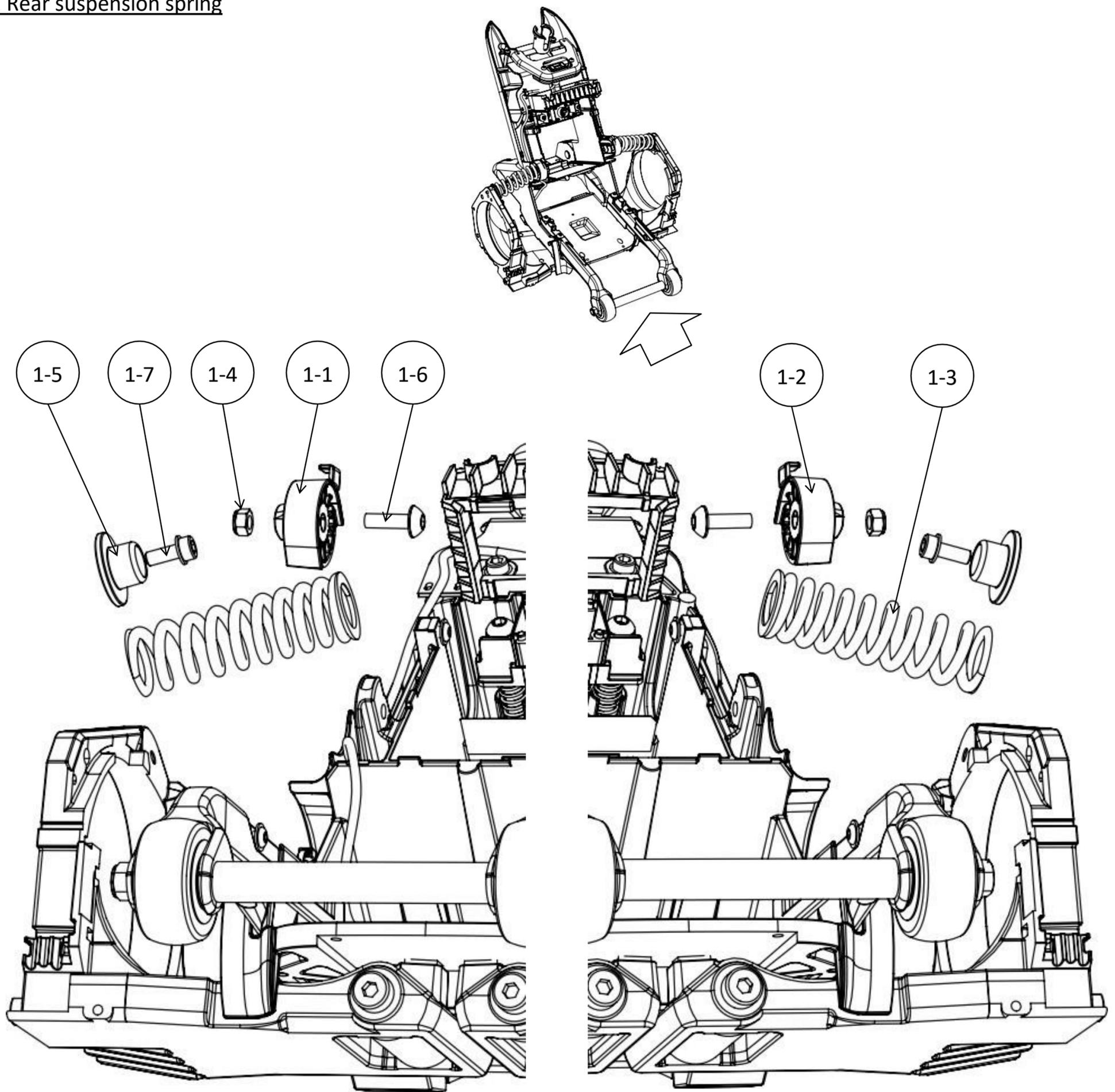


Figure	Part Number	GP Code	Item Name	QTY	Notes
1	24-10041-0	MSP491812	Rear suspension spring kit	1	Including 1-1 to 1-7 for each Q'ty
1-1	31-00302-1		Spring Holder upper (Right)	1	
1-2	31-00302-0		Spring Holder upper (Left)	1	
1-3	31-00196-0		Rear suspension spring	2	LH/RH common
1-4	41-00072-0		Nylon nut M6	2	LH/RH common
1-5	31-00303-0		Spring holder lower	2	LH/RH common
1-6	31-00347-1		Special bolt M6 L20	2	LH/RH common

20. Battery connector parts

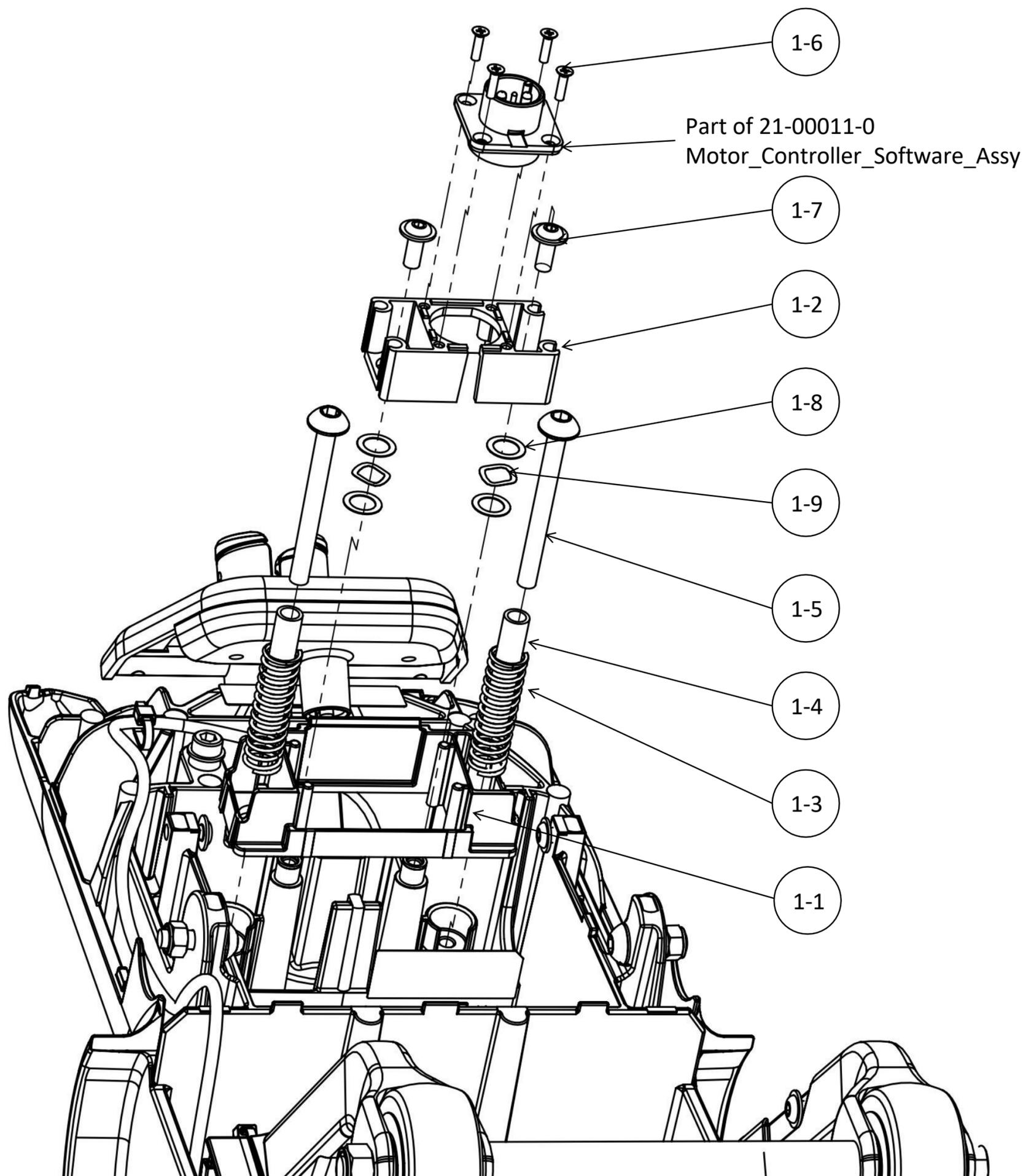


Fig	Part Number	GP Code	Item Name	QTY	Notes
1	24-10043-0	MSP491796	Battery Connector Repair Parts Kit [C2]	1	Including 1-1 to 1-9 for each Q'ty and zip tie
1-1	31-00225-0		Battery Bracket [C2/F]	1	
1-2	31-00224-0		Connector Bracket [C2/F]	1	
1-3	31-00227-0		Battery Bracket Spring [F]	2	
1-4	31-00267-0		Battery Bracket Collar [F]	2	
1-5	31-00347-0		Button bolt M6_L60	2	
1-6	41-00037-0		Countersunk Tapping screw Type 3_L10	4	
1-7	41-00038-0		Flange button bolts M5_L12	2	
1-8	41-10114-0		Washer 13.5_8.5_0.3	4	

21. Seat post parts and motor controller

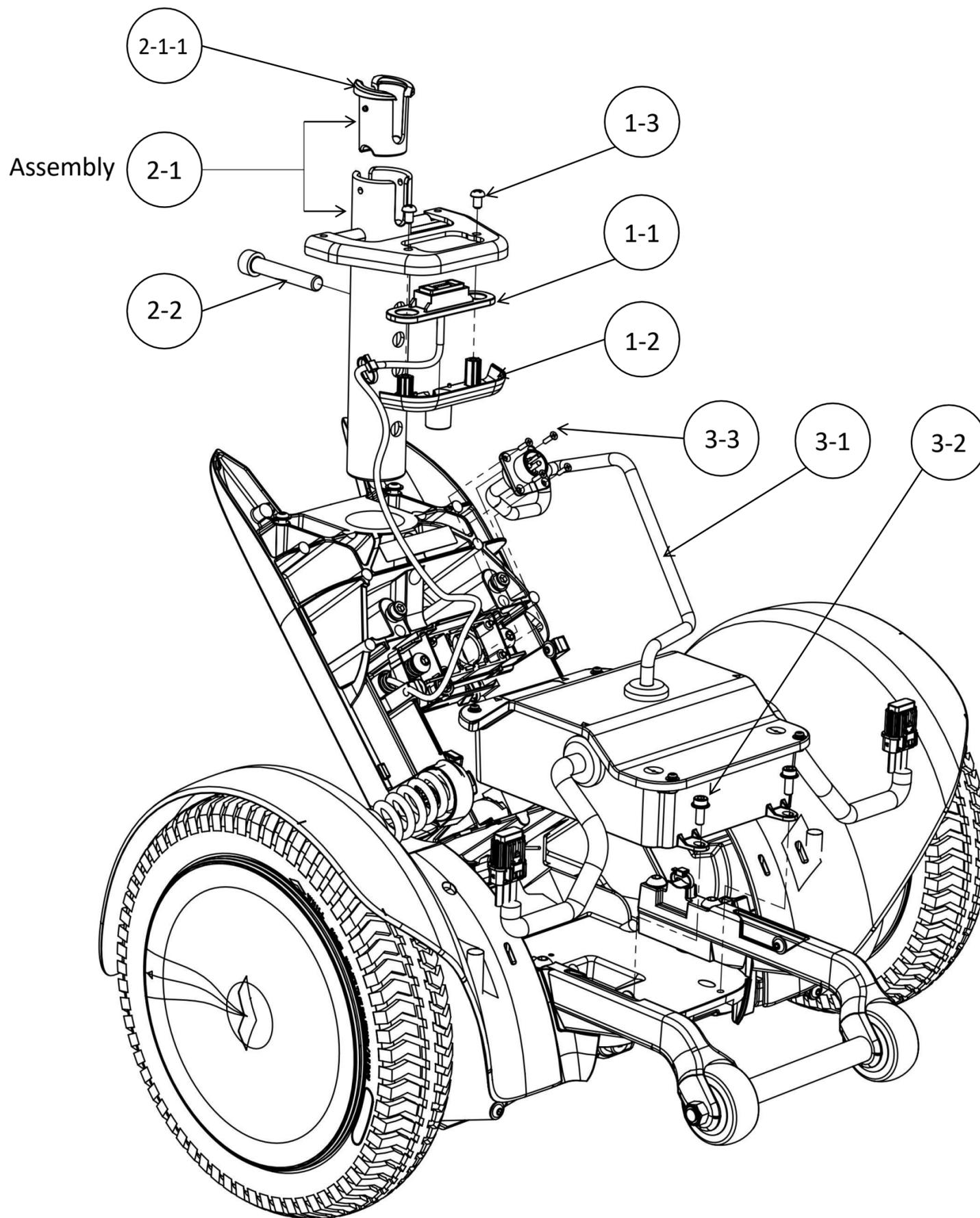


Figure	Part number	GP Code	Item name	Q'ty	Notes
1	24-10045-0	MSP491826	MB cable	1	Including 1-1 to 1-3 for each Q'ty and zip tie
1-1	35-10032-0		Connector bottom cover	1	
1-2	31-00101-0		Connector bottom cover	1	
1-3	41-00042-0		Pan tapping screw Type5 L8	2	
2	24-10046-0	MSP491830	Seat post kit	1	Including 1-2, 1-3, 2-1, 2-2 for each Q'ty
2-1	22-00056-0		Seat post protector assembly	1	Assembly of 2-1-1 and metal parts
2-1-1	31-00277-0		Seat post pipe protector	1	
2-2	41-00091-0		Cap bolt M10 L60	1	
3	24-10047-0	MSP491828	Motor controller kit	1	Including 3-1 to 3-3 for each Q'ty and zip tie
3-1	21-00011-1		Motor controller	1	
3-2	41-00081-0		Flange cap bolt M5 L12	2	
3-3	41-00037-0		Countersunk tapping_p_type bolt 3 L10	4	

22. Rear suspension frame parts and base release handle parts


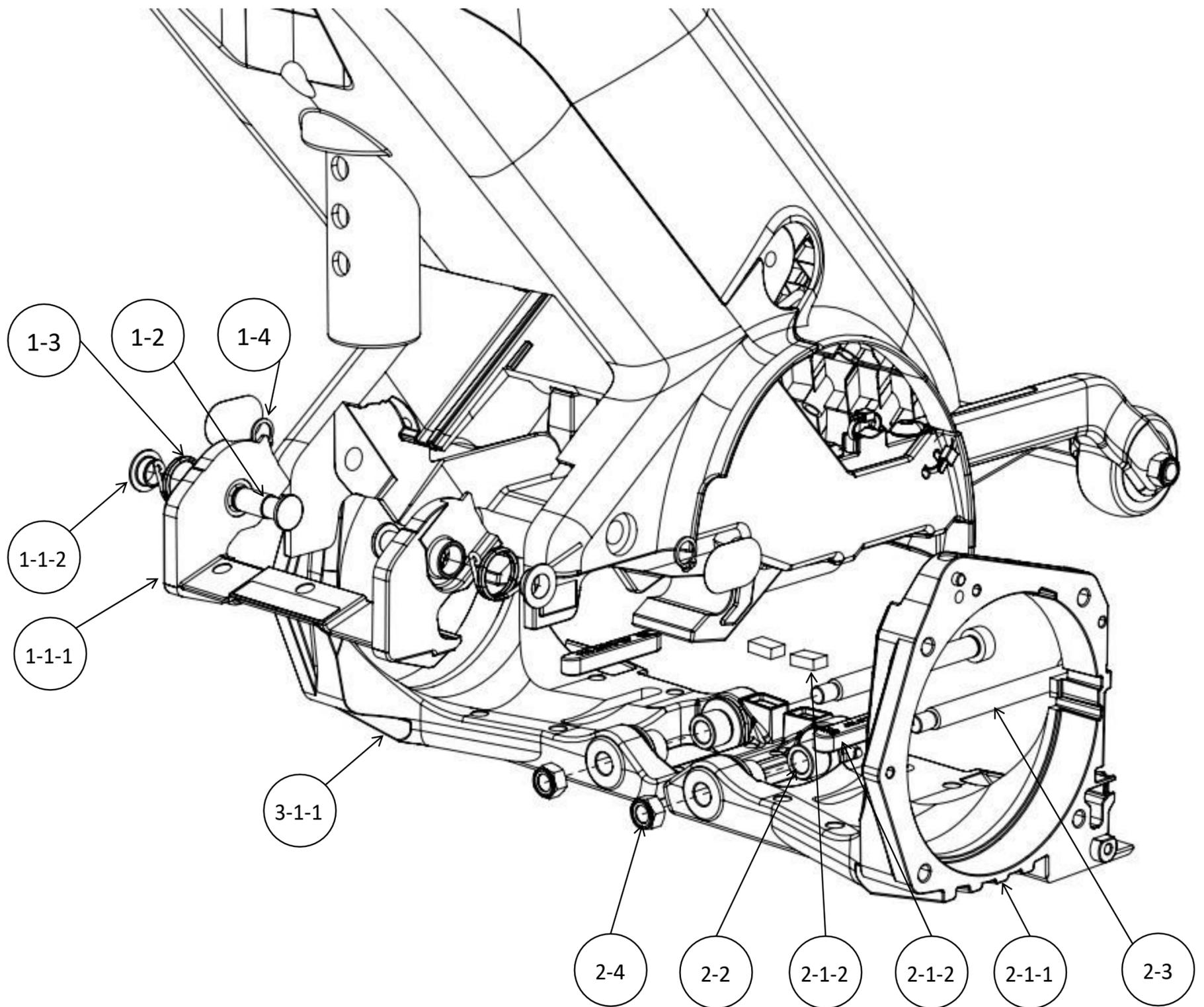


Figure	Part number	GP Code	Item name	Q'ty	Notes
1	24-10048-0	MSP491824	Base release handle kit	1	Including 1-1 to 1-4 for each Q'ty and label
1-1	21-10049-0		Base release handle assembly	1	Including 1-1-1, 1-1-2 for each Q'ty and label
1-1-1	31-00250-0		Base release handle	1	
1-1-2	41-00068-0		Flange collar	2	
1-2	41-00070-0		Hinge pin	2	
1-3	31-00313-0		Torsion spring	2	
1-4	41-00069-0		C clip	2	
2	24-10051-0		Rear suspension arm kit (Left)	1	Including 2-1 to 2-4 for each Q'ty
2-1	21-00036-0		Rear suspension arm stopper assembly (Left)	1	Including 2-1-1 to 2-1-3 for each Q'ty
2-1-1	31-00195-0		Rear suspension arm (Left)	1	
2-1-2	31-00369-0		Rubbar bar	1	LH/RH common
2-1-3	31-10399-0		Rubber block	1	LH/RH common
2-2	31-00125-0		Rear suspension bush	2	LH/RH common
2-3	41-00099-0		Stripper bolt M8 D10 L95	1	LH/RH common
2-4	41-00050-0		Nylon nut M8	1	LH/RH common
3	24-10051-1		Rear suspension arm kit (Right)	1	Including 3-1, 2-2, 2-3, 2-4 for each Q'ty
3-1	21-00036-1		Rear suspension arm stopper assembly (Right)	1	Including 3-1-1, 2-1-2, 2-1-3 for each Q'ty
3-1-1	31-00195-1		Rear suspension arm (Right)	1	

23. Omni wheel

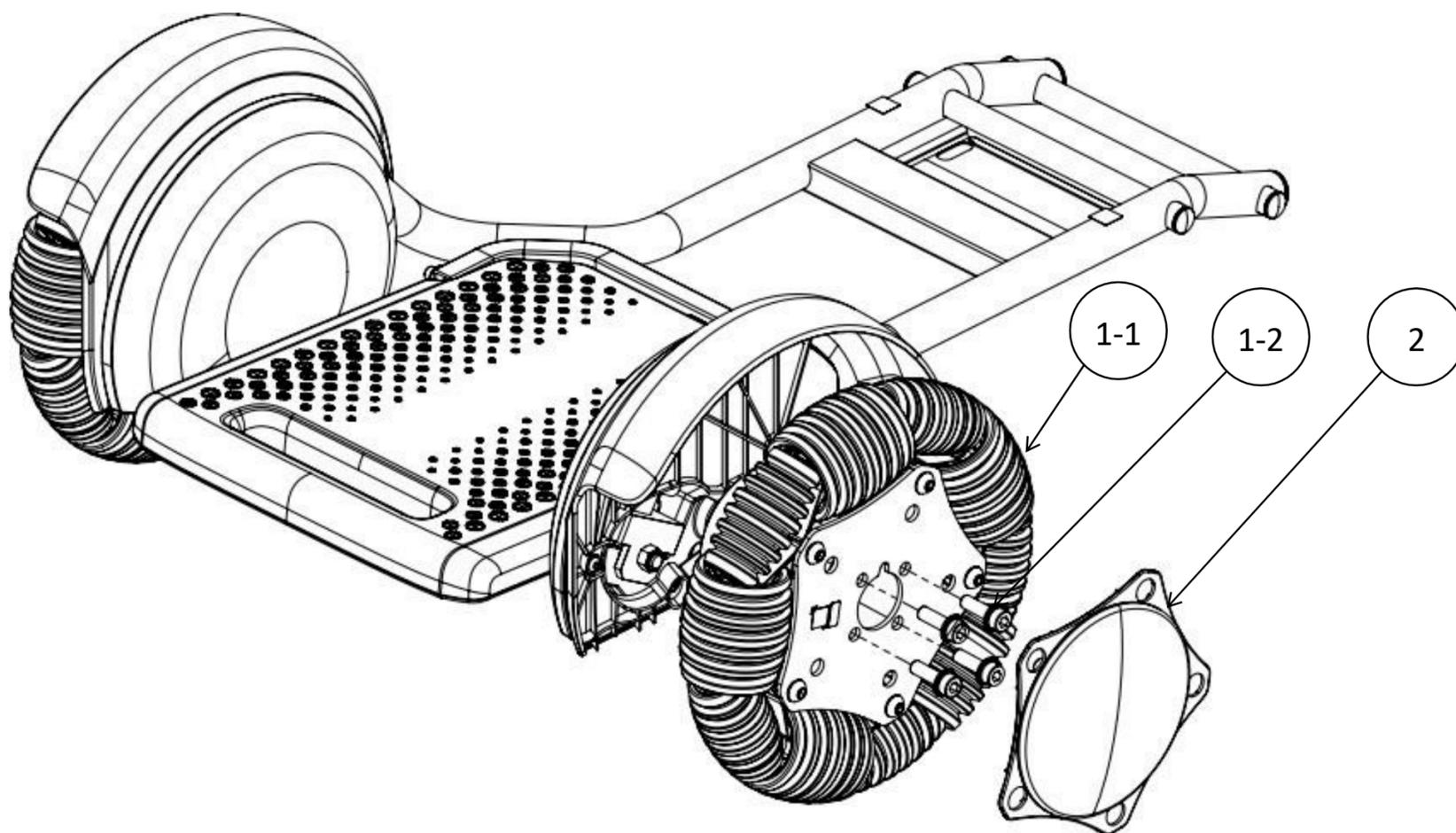

LH and RH are the same

Figure	Part number	GP Code	Item name	Q'ty	Notes
1	24-10030-0	MSP492050	Omni wheel kit [C2]	1	Including 1-1, 1-2 for each Q'ty, LH/RH common
1-1	22-00008-0		Omni wheel	1	LH/RH common
1-2	41-00051-0		Screw (M8-L25)	4	LH/RH common
2	22-00062-0	MSP492020	Omni wheel cap	1	LH/RH common

24. Front fender covers

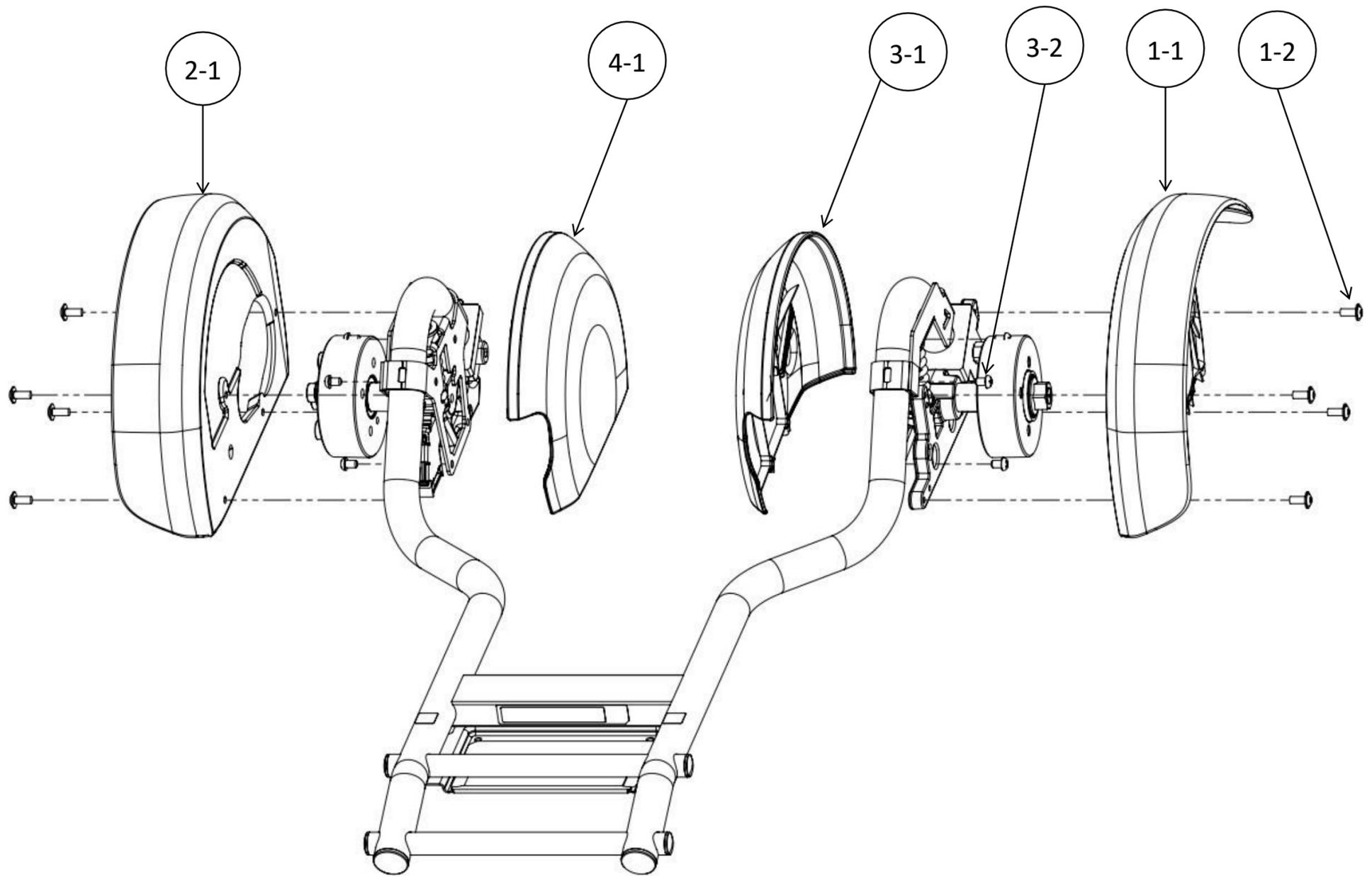


Figure	Part number	GP Code	Item name	Q'ty	Notes
1	24-10032-0	MSP491690	Front fender outer kit (Left) [C2]	1	Including 1-1, 1-2 for each Q'ty
1-1	31-00210-0		Front fender outer (Left) [C2]	1	
1-2	41-00038-0		Screw (L5-L12)	4	LH/RH common
2	24-10032-1	MSP491700	Front fender outer kit (Right) [C2]	1	Including 2-1, 1-2 for each Q'ty
2-1	31-00210-0		Front fender outer RL) [C2]	1	
3	24-10033-0	MSP491670	Front fender inner kit (Left) [C2]	1	Including 3-1, 3-2 for each Q'ty
3-1	31-00209-0		Front fender inner (Left) [C2]	1	
3-2	41-00042-0		Pan Tapping screw type5_L8	4	LH/RH common
4	24-10033-1	MSP491680	Front fender inner kit (Right) [C2]	1	Including 4-1, 3-2 for each Q'ty
4-1	31-00209-1		Front fender inner (Right) [C2]	1	

25. Front suspension parts

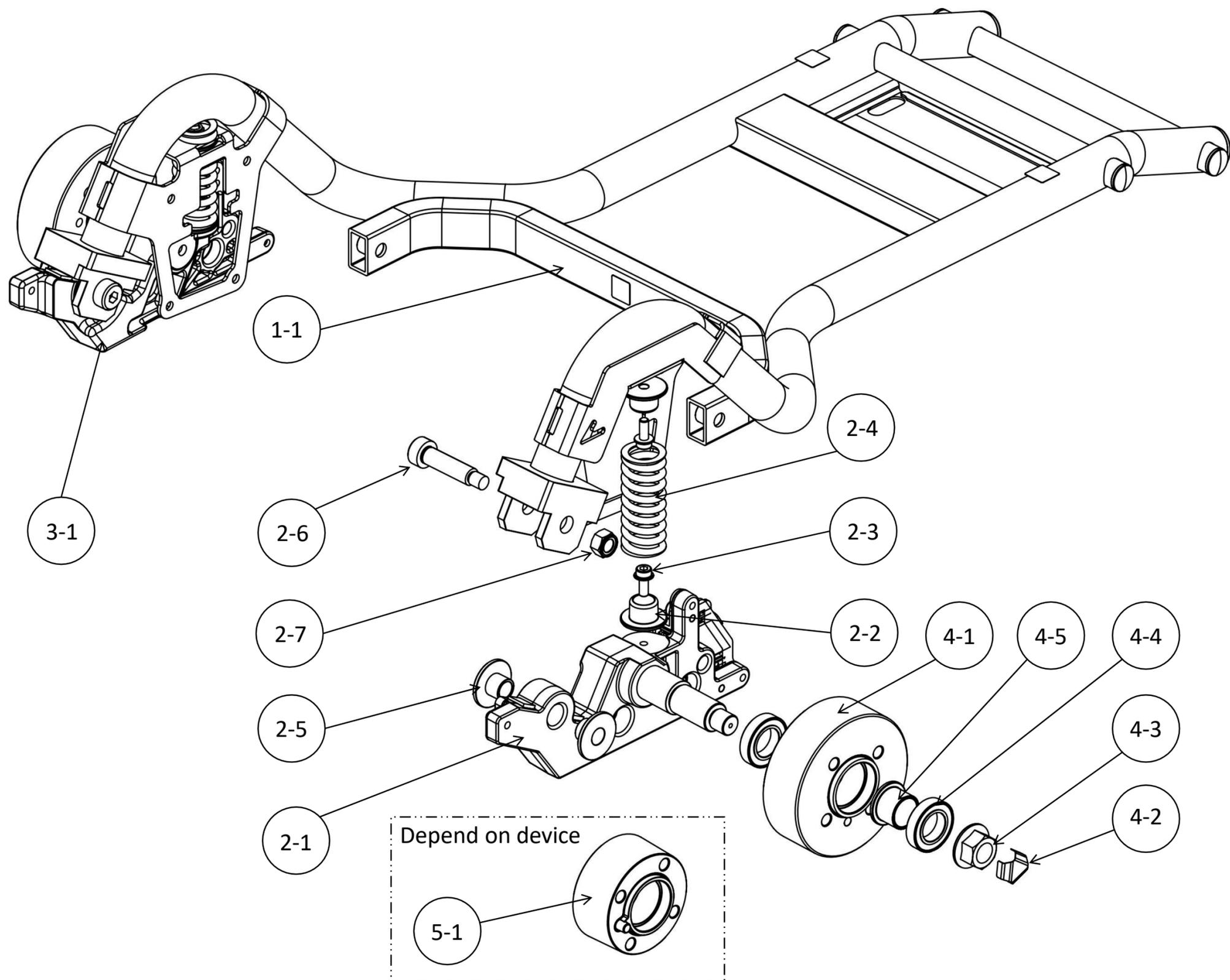


Figure	Part number	GP Code	Item name	Q'ty	Notes
1	24-10036-0		Front drive base frame repair kit	1	Including 1-1 for each Q'ty and label, cap
1-1	21-00023-0		Frame frame assembly	1	
2	24-10035-0	MSP491704	Front suspension repair kit (Left)	1	Including 2-1 to 2-7 for each Q'ty
2-1	21-00035-0		Front suspension arm stopper assembly (Left)	1	Including rubber sheet
2-2	31-00271-0		Spring holder	2	LH/RH common
2-3	41-00081-0		Flange cap bolt M5 L12	2	LH/RH common
2-4	31-00122-0		Front suspension spring	1	LH/RH common
2-5	31-00215-0		Brass bushing	2	LH/RH common
2-6	41-00031-0		Stripper bolt M8 D10 L35	1	LH/RH common
2-7	41-00050-0		Nylon nut M8	1	LH/RH common
3	24-10035-1	MSP491706	Front suspension repair kit (Right)	1	Including 3-1, 2-2 to 2-7 for each Q'ty
3-1	21-00035-1		Front suspension arm stopper assembly (Right)	1	Including rubber sheet
4	24-10031-0	MSP492052	Omni wheel hub kit (Large)	1	Including 4-1 to 4-5 for each Q'ty
4-1	31-00104-0		Omni wheel hub (Large)	1	LH/RH common
4-2	32-10040-0		Seal motor center nut	1	LH/RH common
4-3	41-00048-0		Flange nut M12 P1.25	1	LH/RH common
4-4	41-00049-0		Ball Bearing 6903	2	LH/RH common
4-5	31-00288-0		Collar 6903	1	LH/RH common
5	24-10031-1	MSP492054	Omni wheel hub kit (Small)	1	Including 5-1, 4-2 to 4-5 for each Q'ty
5-1	31-00104-2		Omni wheel hub (Small)	1	LH/RH common

26. Foot rest cover

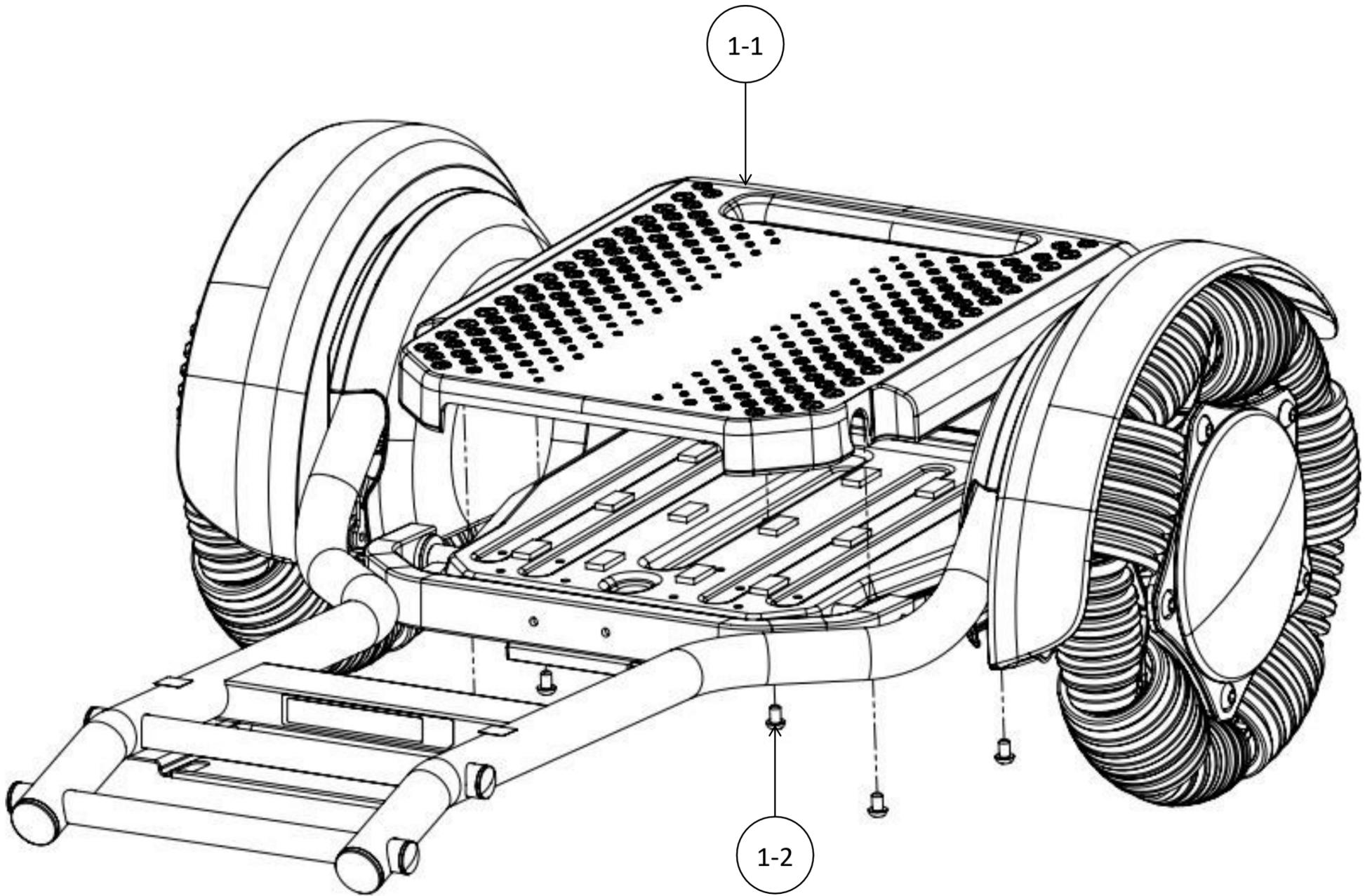


Figure	Part number	GP Code	Item name	Q'ty	Notes
1	24-10034-0	MSP491710	Foot rest cover assembly	1	Including 1-1, 1-2 for each Q'ty
1-1	31-00254-0		Foot rest cover	1	
1-2	41-00042-0		Pan Tapping screw type5_L8	8	

27. Foot rest parts

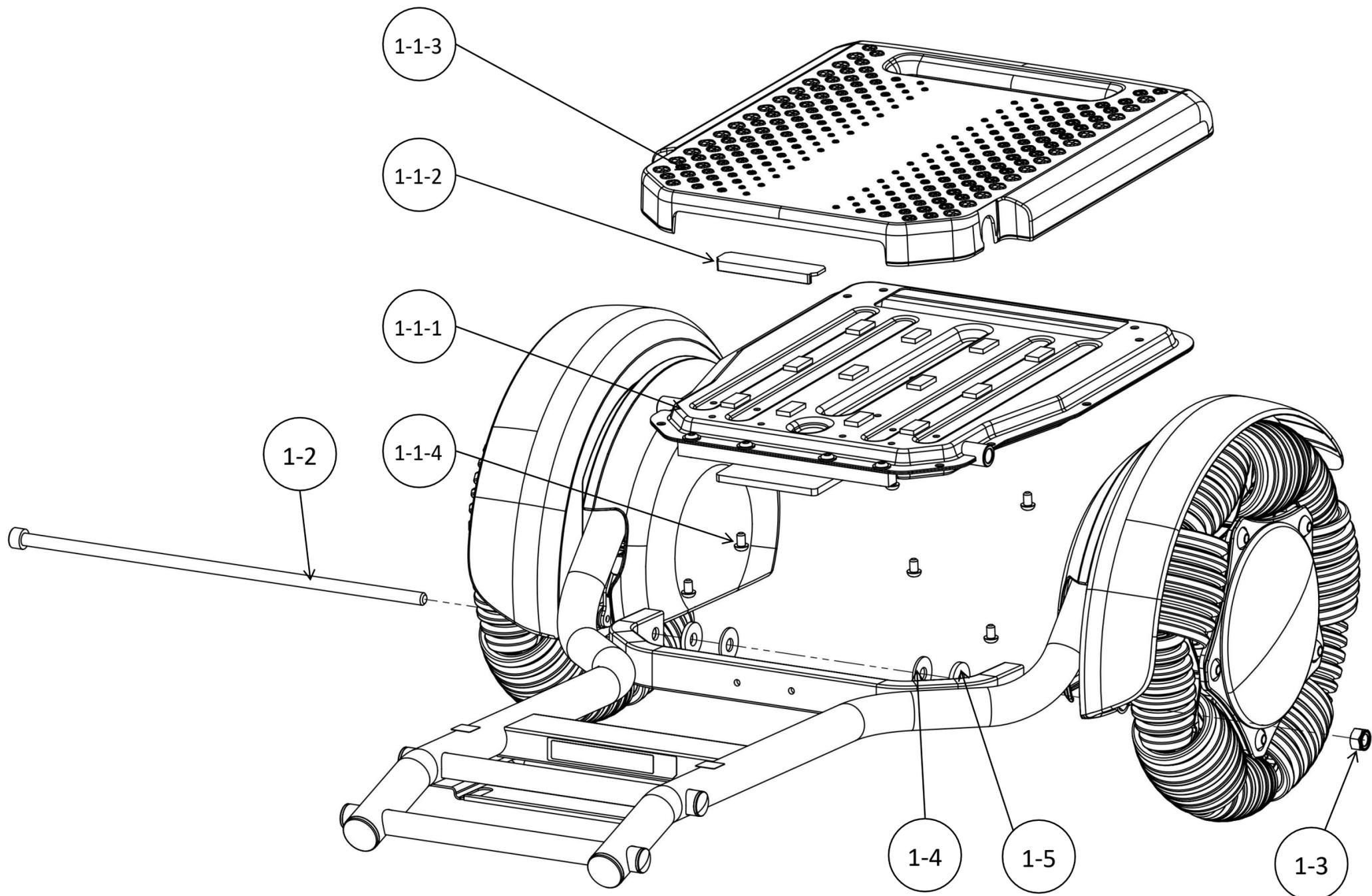


Figure	Part number	GP Code	Item name	Q'ty	Notes
1	24-10053-0	MSP491730	Foot rest assembly kit	1	Including 1-1 to 1-5 for each Q'ty
1-1	21-00024-0		Foot rest assembly	1	Including 1-1-1 to 1-1-4 for each Q'ty and sponge
1-1-1	21-10048-0	MSP491720	Foot rest frame assembly	1	Metal parts without sponge
1-1-2	31-00367-0		Stopper plate	1	
1-1-3	31-00254-0		Foot rest cover	1	
1-1-4	41-00042-0		Pan Tapping screw type5_L8	8	
1-2	41-00032-0		Cap bolt M8_L295	1	
1-3	41-00050-0		Nylon nut M8	1	
1-4	31-00273-0		Washer 22_8.5_1.6	2	
1-5	31-00272-0		POM Washer 22_8.2_3	2	
2	21-10052-0	MSP491702	Stopper plate repair kit	1	Including 1-1-2 and double side tape for repair purpose

28. Battery

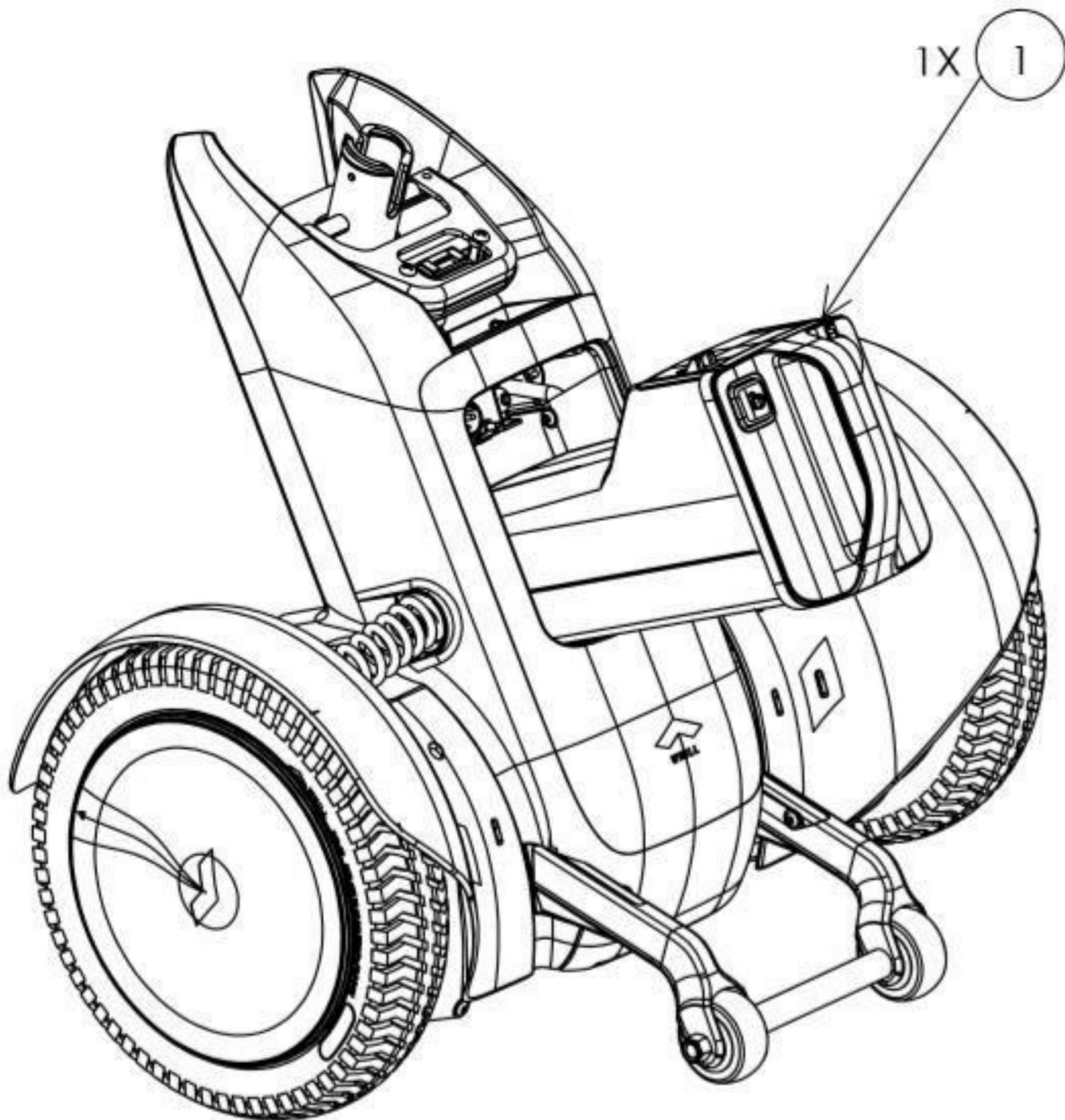


Figure	Part number	GP Code	Item name	Notes
1	23-00006-0	WCA870400	Li-Ion Battery	Removable lithium-ion battery for Model C2
1	21-00014-0		Li-Ion Battery	Removable lithium-ion battery for Model C2 Add JP label at warehouse

29. Basket



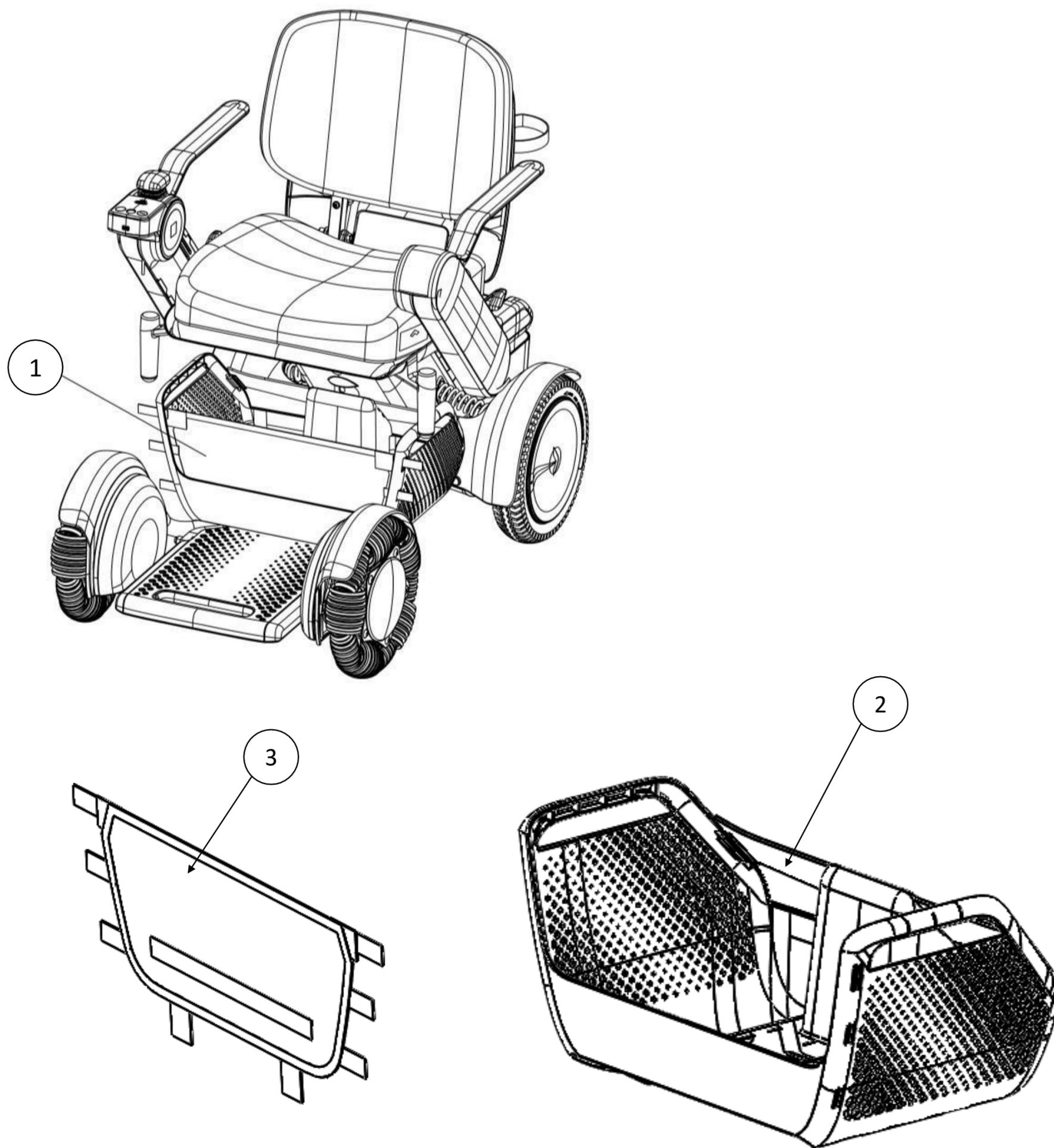


Figure	Part number	GP Code	Item name	Notes
1	21-10041-0		Storage basket assy	Basket for Model C2 (with 2 and 3 assembled)
1	21-10041-1		Storage basket assy	Basket for Model C2 (with 2 and 3 assembled)
1	21-10041-2		Storage basket assy	Basket for Model C2 (with 2 and 3 assembled)
2	-	WCA870440	Storage basket	Plastic basket case
3	-		Storage basket net	Basket net

30. Cane holder

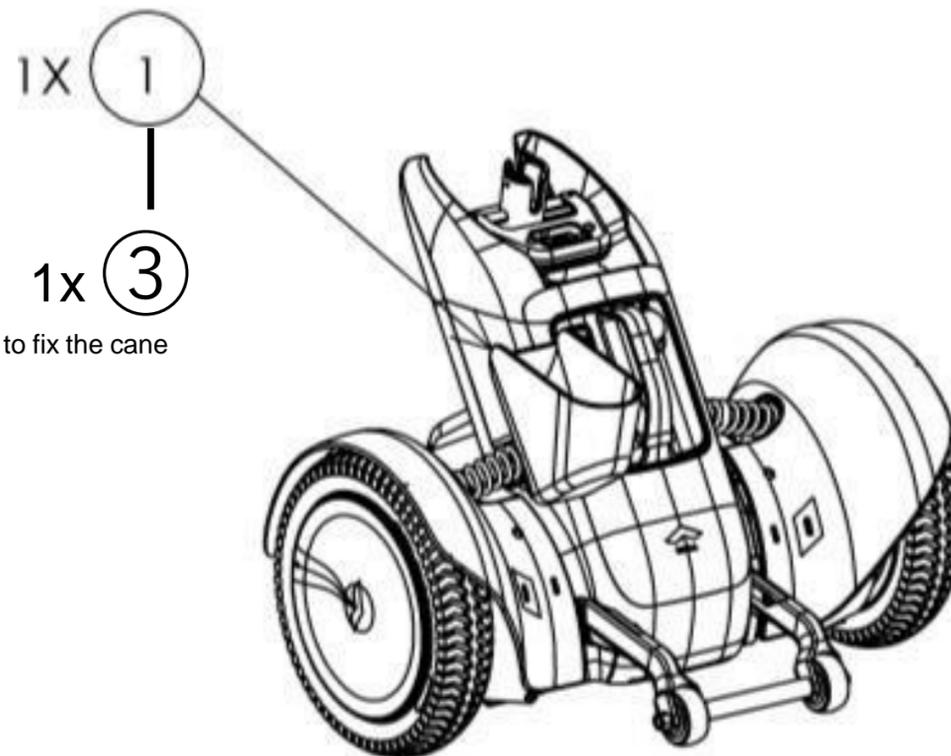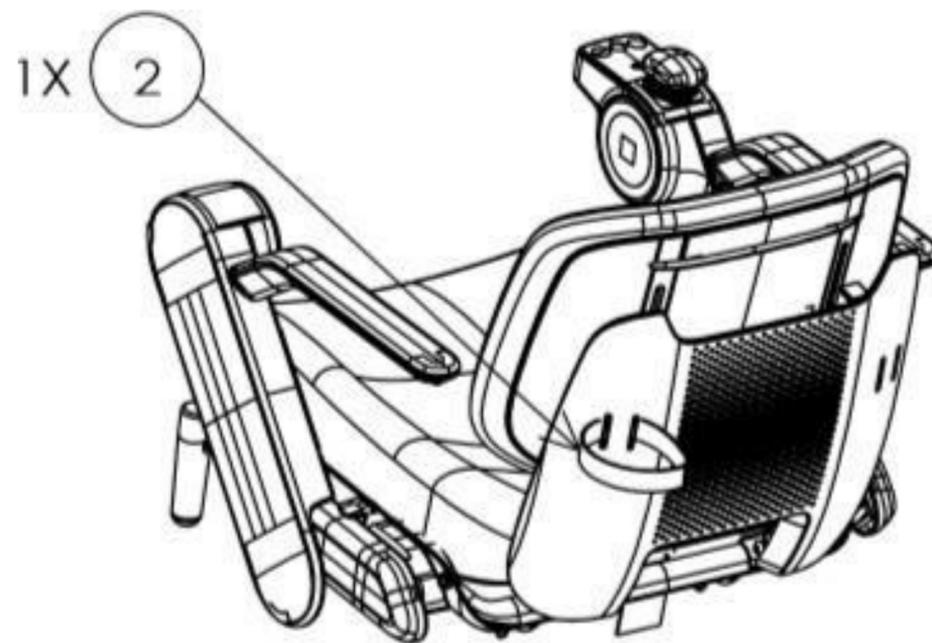

Plastic rivet to fix the cane holder

Figure	Part number	GP Code	Item name	Notes
1	22-10083-0		Cane holder	Cane holder (lower part) *This can be assembled on left and right side
2	31-00361-0		Cane holder band	Upper band to secure the cane (Velcro) *This can be assembled on left and right side
3	41-00100-0		Plastic rivet	Plastic rivet to fix fig.1 (x1)
-	24-10003-0		Cane holder kit	Kit of fig.1,2 and 3.

31. Smart key / Lap belt / Charger



Figure	Part number	GP Code	Item name	Notes
1	21-00037-0		Smart key	When you place an order, please provide us with the serial number of the Model C2.
2	24-10007-X	WCA870420	Lap belt	Lap belt is default in EU/AUS/NZ. -X is different from country or region, please check with your regional distributor.
3	23-00010-0		Charger JP plug	
3	23-00010-1		Charger CN plug	
3	23-00010-3		Charger US plug	
3	23-00010-4		Charger EU plug	
3	23-00010-5		Charger UK plug	
3	23-00010-6	WCA870410	Charger AUS/NZ plug	
3	23-00010-7		Charger Asia-Pacific	3 plugs (BF, C, O type) included.

32. Other repair parts



①



②

Figure	Part number	GP Code	Item name	Notes
1	31-10491-0	MSP491834	Battery connector cap	Battery connector cap for repair purpose.
2	24-10017-0	MSP491838	Button sticker service kit	Stickers to put on the control pad button when the marks are wared